

COMFORT FOOT: BOLT-ON

WARNING: Make sure all screws are tightly clamped upon completion.

Diagram 1


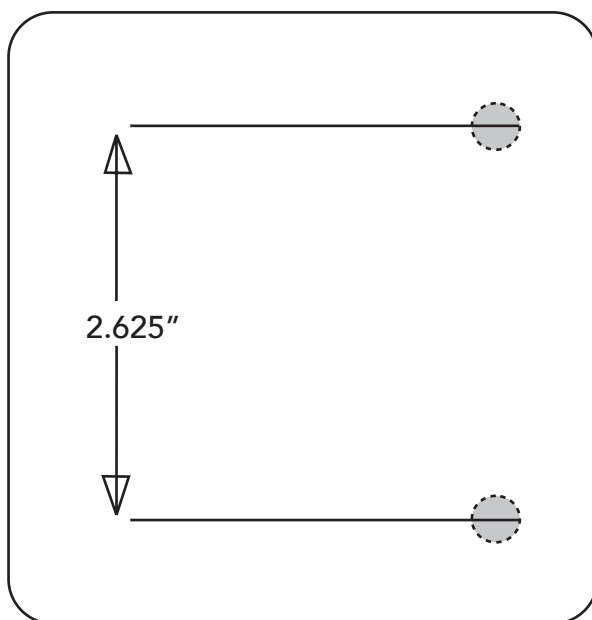


Diagram 2

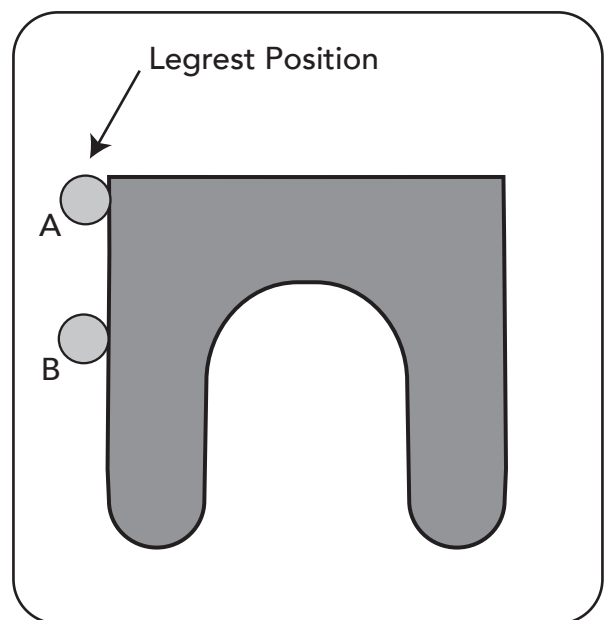

1 Place template (included) on the wheel-chair footplate in the desired position.

2 *Step 2 shown on diagram 1

Screw two $\frac{3}{4}$ " x $1\frac{1}{4}$ " or $\frac{3}{4}$ " x 1" bolts (depending on the size of footplate) from the bottom of the footplate into the Comfort Foot base.

3 *Step 3 shown on diagram 2

Release cam buckle and secure tightly around legrest tubing. Depending on wheel-chair, common loop will have to be tightened around legrest tubing (A), or legrest position (B).