

COMFORT FOOT: CLAMP-ON

WARNING: Make sure all screws are tightly clamped upon completion.

Diagram 1 (steps 1-3)


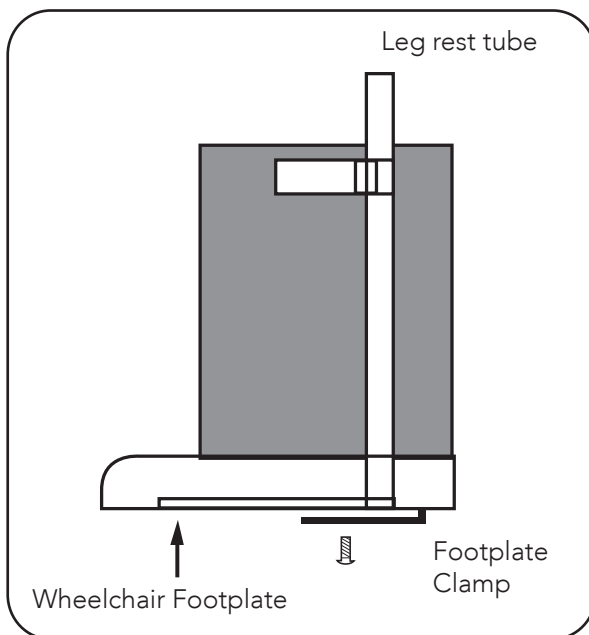


Diagram 2 (step 4)

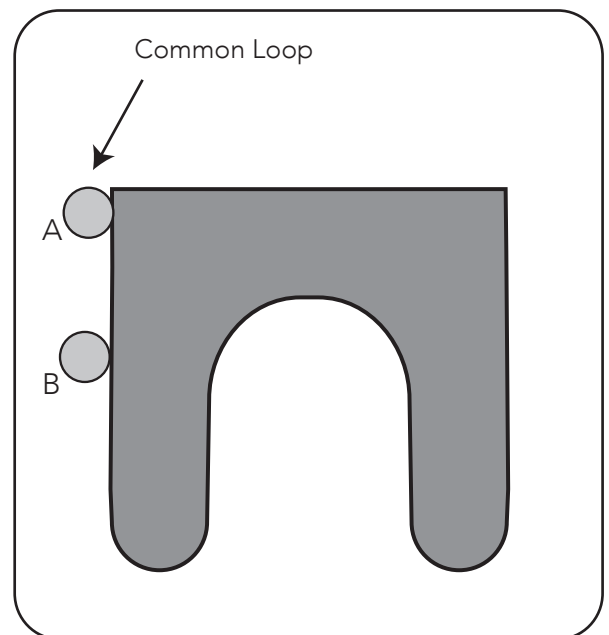

- 1** Loosen, but do not remove, footplate clamp using Phillips head screwdriver.
- 2** Slide clamp over wheelchair footrest plate.
- 3** Adjust Comfort Foot to desired position. Tighten screw securely.
- 4** Release common loop and secure tightly around legrest tubing. Depending on the wheelchair, common loop will have to be tightened around either position (A), or position (B).