




311 Otterson Drive, Suite 10
Chico, CA 95928
800.497.2003 Fax 530.893.1560
www.SafePathProducts.com

EZ Edge™ & StoneCap™ Threshold Ramp Installation Instructions

1. It is imperative that your substrate is CLEAN & FREE OF DEBRIS before beginning installation. A clean, dry substrate ensures maximum adhesion.
2. Dry fit the product by placing it flush against the threshold.
3. Use painter's tape to outline the ramp.
4. Remove ramp.
5. Generously apply adhesive to substrate. (We strongly recommend *SikaFlex 1a* brand adhesive). For best results, apply with a caulking gun, then use a notched cement/grout hand trowel to spread evenly.
6. Replace ramp onto adhesive surface.
7. Apply pressure to ramp evenly. Some glue may leak out onto tape.
8. After ramp is in place with adhesive, carefully remove painter's tape.
9. Excess adhesive can be cleaned from the substrate or product by applying acetone or paint thinner.

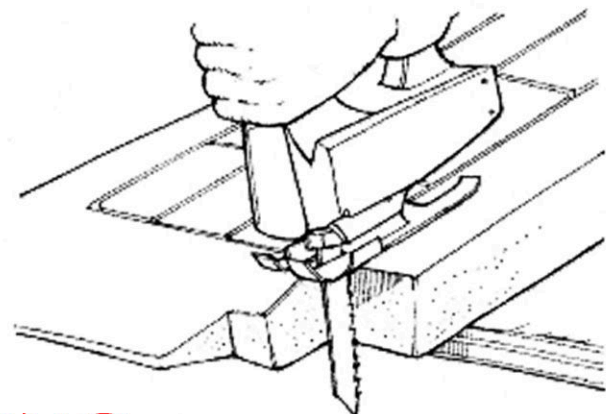

****Note: Excess adhesive will stain clothing, shoes and carpet - immediately clean any excess.****

WARNING

DO NOT use a power circular saw to cut ramp product.
The use of any electrical, battery, or pneumatic powered circular saw may result in serious injury due to unexpected binding, grabbing, or pinching of ramp material.