
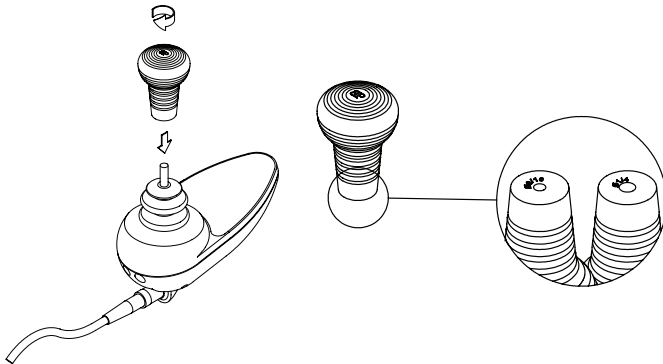
# Joystick Handle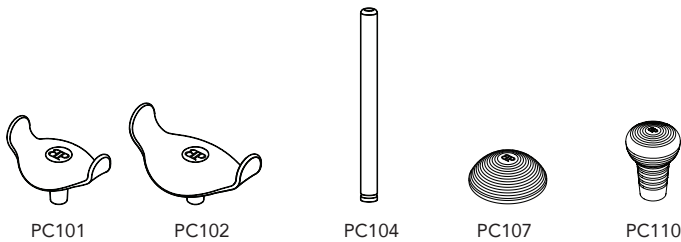

**en** All Bodypoint Joystick Handles with flexible shafts will, by design, slide off of the steel joystick stem if subjected to undue pulling. The U-shaped Joystick Handle design also allows rotation on its shaft. These design features are meant to prevent damage to the joystick box. Altering the product to circumvent these features is not recommended. Our Joystick Handle is designed with a flexible shaft that will fit tightly on any 3/16" (5 mm) or 1/4" (6mm) diameter steel joystick stem.

## INSTALLATION INSTRUCTIONS:

1. Clean the steel joystick stem to remove any dirt or oil.
2. Moisten the steel joystick stem to facilitate mounting the handle.
3. Grasp the steel joystick stem near the base with pliers to keep from pushing it into the sensitive electrical components of the joystick box.
4. Press the Joystick Handle onto the steel joystick shaft with a twisting motion.
5. To permanently affix joystick handle apply 2-3 drops of Cyanocrylate adhesive (super glue) to joystick mounting hole before installation.

**WARRANTY:** This product carries a limited lifetime warranty against defects in workmanship and materials arising under normal use by the original consumer. Contact your supplier or Bodypoint for warranty claims.

For more information on Bodypoint products, and a list of distributors outside the USA, go to [www.bodypoint.com](http://www.bodypoint.com).

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