


(Discontinued 1) Focus CR Power Tilt

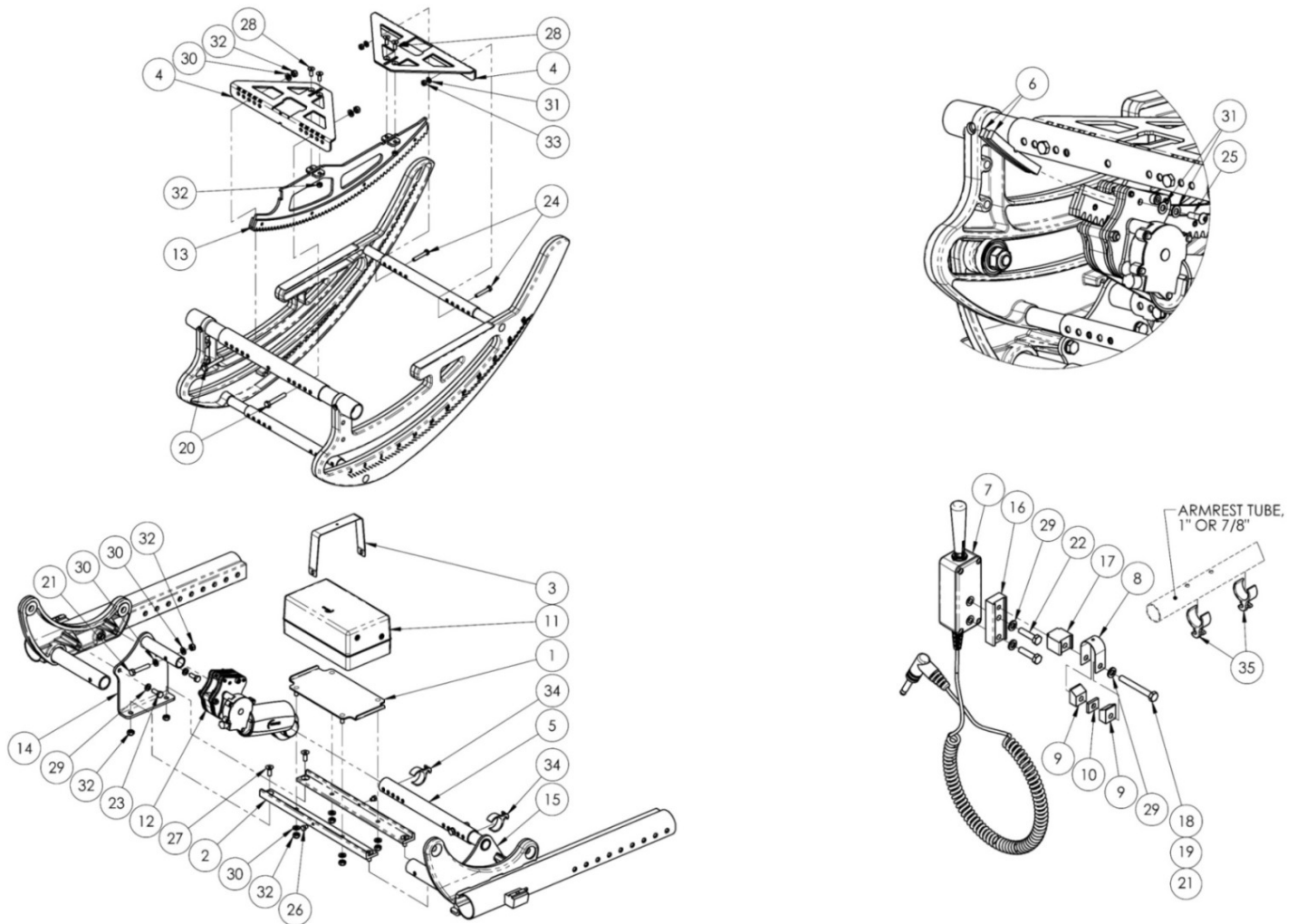

This Focus CR Power Tilt is discontinued and was used on serial numbers prior to FCHD002253 and FCR0020156.

Item Number	Part Number	Description	Quantity	Retail
1	002819	Power Tilt Tray RCB Focus	2	\$16.50
2a	002820	Power Tilt Bracket RCB 14W Focus	2	\$41.00
2b	002821	Power Tilt Bracket RCB 15W Focus	2	\$43.00
2c	002822	Power Tilt Bracket RCB 16W Focus	2	\$45.50
2d	002823	Power Tilt Bracket RCB 17W Focus	2	\$46.50
2e	002824	Power Tilt Bracket RCB 18W Focus	2	\$48.50
2f	002825	Power Tilt Bracket RCB 19W Focus	2	\$50.50
2g	002826	Power Tilt Bracket RCB 20W Focus	2	\$52.50

Item Number	Part Number	Description	Quantity	Retail
2h	002827	Power Tilt Bracket RCB 21W Focus	2	\$54.00
2i	002828	Power Tilt Bracket RCB 22W Focus	2	\$56.00
3	002829	Power Tilt Strap RCB Focus	1	\$36.50
4a	002892	Power Tilt Mounting Bracket Small	2	\$83.50
4b	002893	Power Tilt Mounting Bracket Medium	2	\$94.50
4c	002894	Power Tilt Mounting Bracket Large	2	\$106.00
5a	002895	Power Tilt Strut Tube Small Focus	1	\$22.50
5b	002896	Power Tilt Strut Tube Medium Focus	1	\$28.00
5c	002897	Power Tilt Strut Tube Large Focus	1	\$33.50
6	002898	Power Tilt Glide Block Focus	2	\$39.00
7	002900	Power Tilt Control Switch Focus	1	\$372.50
8	002902	Power Tilt Band Clamp Strap 7/8"	1	\$49.00
9	002903	Power Tilt Band Clamp Mandrel	2	\$20.00
10	002904	Power Tilt Clamp Spacer <i>Spacer only used with the Angle Adjustable Locking Extendable Flip Up Armrest</i>	1	\$18.50
11	002905	Power Tilt Battery RCB10 Unit, Focus <i>(Motor Cord is not shown)</i>	1	\$433.50
12	002908	Power Tilt Motor Arm Assembly Focus	1	\$611.50
13	002909	Power Tilt Rack Assembly Focus	1	\$83.50
14	002910	Power Tilt Mounting Plate Right Focus	1	\$44.50
15	002911	Power Tilt Mounting Plate Left Focus	1	\$44.50
16	003090	Power Tilt Control Switch Mount Block	1	\$27.00
17	003091	Power Tilt Control Switch Spacer <i>Spacer not used with Angle Adjustable Locking Extendable Flip Up Armrest with Foam Grip</i>	1	\$21.50
18	101827	M6 x 55 HHCS Blk Zn <i>Used with the Angle Adjustable Locking Extendable Flip Up Arm with Standard Arm Pad. If wanting to replace this hardware, use part# 100676.</i>	1	\$1.50
19	101826	M6 x 50 HHCS Blk Zn <i>Used with the Height Adjustable T-Arm, Height Adjustable Low T-Arm and Height Adjustable Dual Post Flip Back Arm</i>	1	\$2.00

Item Number	Part Number	Description	Quantity	Retail
20	101825	M6 x 40 HHCS Blk Zn	2	\$1.00
21	102386	M6 x 35mm HHCS Blk Zc <i>Used with the Angle Adjustable Locking Extendable Flip Up Armrest with Foam Grip</i>	3	\$1.00
22	102792	M6 x 25, HHCS, Blk Zn	2	\$0.50
23	102327	M6 x 16 HHCS Blk Zn	4	\$1.00
24	102933	M5 x 30 HHCS, BLK ZN	2	\$4.00
25	101716	M5 x 16 SHCS Blk Zn	2	\$1.00
26	102935	M5 X 6 HHCS w/Patch, Blk Zn	2	\$13.00
27	102936	M6 X 16 FHCS, Blk Zn	4	\$1.00
28	102937	M5 X 14 FHCS, Blk Zn	4	\$1.50
29	101835	M6 Lock Washer Blk Zn	7	\$0.50
30	100746	M6 Flat Washer Blk Zn	10	\$1.50
31	101856	Flat Washer, M5, Blk Zn	10	\$0.50
32	100658	M6 Nylock Nut Blk Zn	12	\$1.00
33	100657	M5 Nylock Nut Blk Zn	6	\$0.50
34	001117	Cable Clip 1" <i>Additional part used on seat tube (seat tube not shown in image)</i>	3	\$3.64
35	001118	Cable Clip 7/8"	2	\$3.64
	002906	Power Tilt Battery Charger RCB 120V (Mascot 2740) <i>Charger not shown in image</i>	1	\$139.00
4a, 5a, 6, 11-15, 20, 21, 23-25, 28-34	503396	Power Tilt Module Kit, Small, Focus	1	\$1,550.92
4b, 5b, 6, 11-15, 20, 21, 23-25, 28-34	503397	Power Tilt Module Kit, Medium, Focus	1	\$1,578.42
4c, 5c, 6, 11-15, 20, 21, 23-25, 28-34	503398	Power Tilt Module Kit, Large, Focus	1	\$1,606.92
1, 2a, 3, 26, 27, 30, 32	503399	Battery Tray Kit, Power Tilt, 14W, Focus	1	\$313.00
1, 2b, 3, 26, 27, 30, 32	503400	Battery Tray Kit, Power Tilt, 15W, Focus	1	\$315.00
1, 2c, 3, 26, 27, 30, 32	503401	Battery Tray Kit, Power Tilt, 16W, Focus	1	\$317.50
1, 2d, 3, 26, 27, 30, 32	503402	Battery Tray Kit, Power Tilt, 17W, Focus	1	\$318.50

Item Number	Part Number	Description	Quantity	Retail
1, 2e, 3, 26, 27, 30, 32	503403	Battery Tray Kit, Power Tilt, 18W, Focus	1	\$320.50
1, 2f, 3, 26, 27, 30, 32	503404	Battery Tray Kit, Power Tilt, 19W, Focus	1	\$322.50
1, 2g, 3, 26, 27, 30, 32	503405	Battery Tray Kit, Power Tilt, 20W, Focus	1	\$324.50
1, 2h, 3, 26, 27, 30, 32	503406	Battery Tray Kit, Power Tilt, 21W, Focus	1	\$326.00
1, 2i, 3, 26, 27, 30, 32	503407	Battery Tray Kit, Power Tilt, 22W, Focus	1	\$328.00
7-9, 16, 17, 19, 22, 35	503408	Controller Kit, Power Tilt, 7/8" Tube, Focus	1	\$520.28
4a, 13, 20, 24, 28, 30-33	503410	Power Tilt Rack Assembly, Small, Focus	1	\$283.50
4b, 13, 20, 24, 28, 30-33	503411	Power Tilt Rack Assembly, Medium, Focus	1	\$305.50
4c, 13, 20, 24, 28, 30-33	503412	Power Tilt Rack Assembly, Large, Focus	1	\$328.50