


Catalyst 5Ti Rear Frame

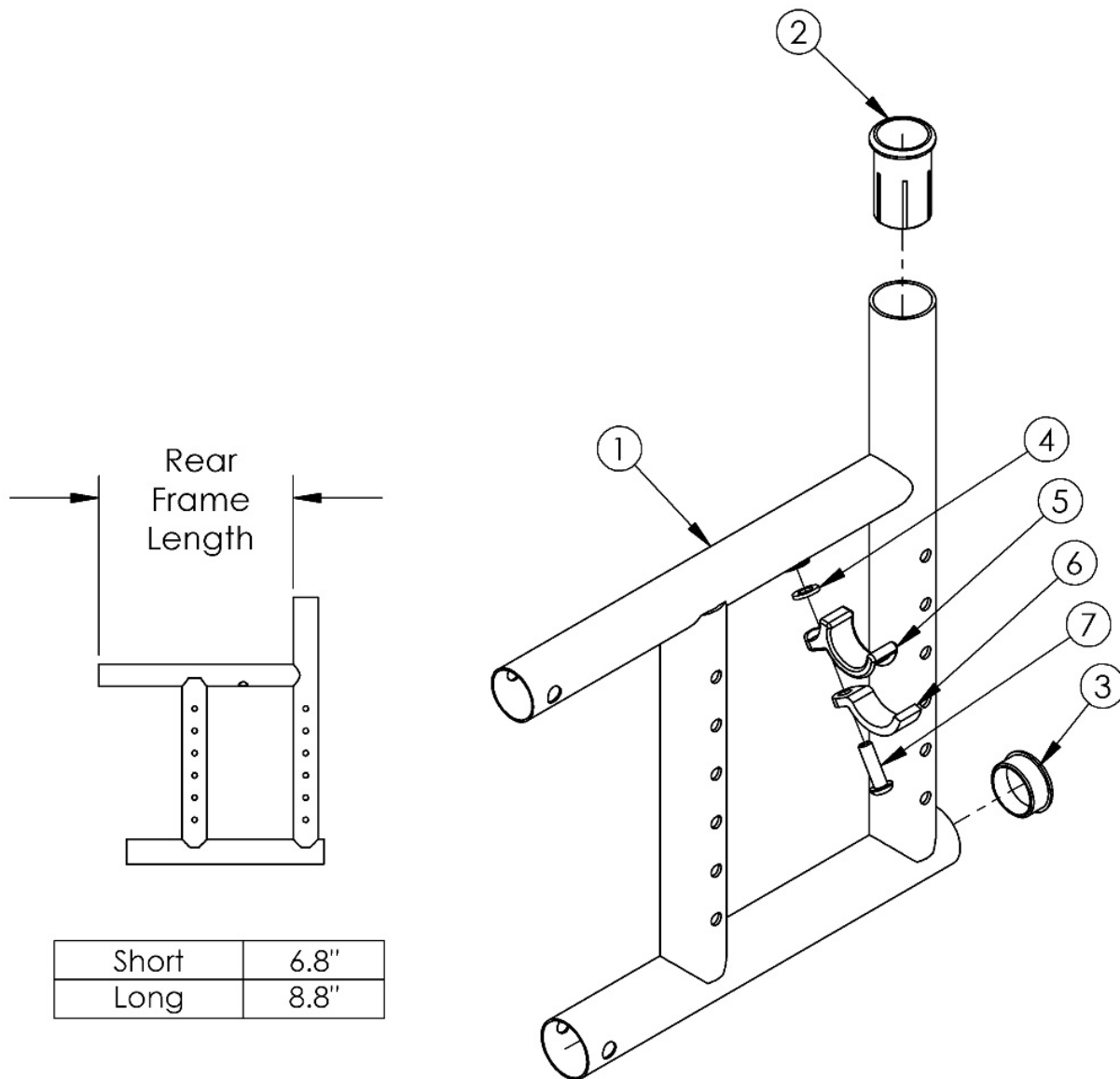

Use the short frame for 14-17" Seat Depths. Use long the frame for 18-20" Seat Depths.<p>
 If switching to a new Seat Depth, new Height Adjustable Flip Back T-Arms may be required (if present).

Item Number	Part Number	Description	Quantity	Retail
4, 5, 6, 7	501005	Seat Hook and Support Hardware Kit, Catalyst <i>This kit contains quantity of 1 each</i>	1	\$17.06
1a, 2, 3	100275	Frame Assm Cat TI Rear Right Short	1	\$549.20
1b, 2, 3	100276	Frame Assm Cat TI Rear Right Long	1	\$549.20

Item Number	Part Number	Description	Quantity	Retail
1c, 2, 3	100277	Frame Assm Cat TI Rear Left Short	1	\$549.20
1d, 2, 3	100278	Frame Assm Cat TI Rear Left Long	1	\$549.20
2	000030	Backrest Sleeve Cat	1	\$3.45
3	100532	Tube End Plug 1 1/8"	1	\$2.58
4	100831	1/2" Washer OD .040" Thick	1	\$1.55
5	000003	Seat Hook CAT	1	\$7.55
6	000004	Seat Hook Support CAT	1	\$6.42
7	100668	M6 x 20 CLS10.9 BTN SHCS ZC w/Patch	1	\$1.55