


Focus CR One Arm Drive Hex Style

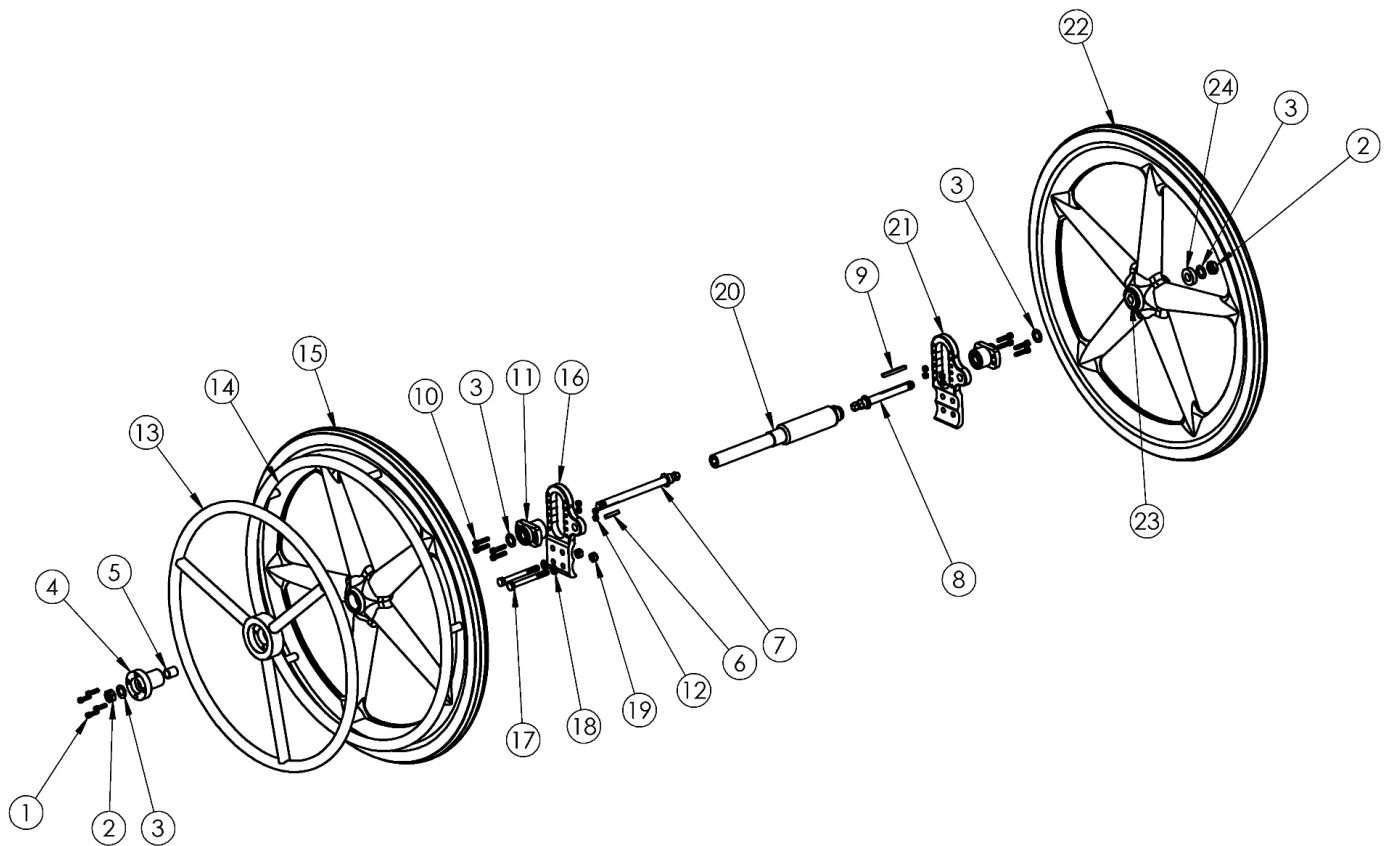

*One Arm Drive Kits do not include rear tires

Item Number	Part Number	Description	Quantity	Retail
1-12, 13a, 14-19, 20e, 21-24	503476	Focus One Arm Drive Retrofit Kit with Aluminum Drive Handrim, 16.69" Drive Shaft <i>Used on 18" wide chairs</i>	1	\$1,951.85
1-12, 13a, 14-19, 20f, 21-24	503477	Focus One Arm Drive Retrofit Kit with Aluminum Drive Handrim, 17.69" Drive Shaft <i>Used on 19" wide chairs</i>	1	\$1,951.85
1-12, 13a, 14-19, 20i, 21-24	503480	Focus One Arm Drive Retrofit Kit with Aluminum Drive Handrim, 20.69" Drive Shaft <i>Used on 22" wide chairs</i>	1	\$1,951.85
1-12, 13b, 14-19, 20e, 21-24	503485	Focus One Arm Drive Retrofit Kit with Poly Coated Drive Handrim, 16.69" Drive Shaft <i>Used on 18" wide chairs</i>	1	\$2,157.85
1-12, 13b, 14-19, 20f, 21-24	503486	Focus One Arm Drive Retrofit Kit with Poly Coated Drive Handrim, 17.69" Drive Shaft <i>Used on 19" wide chairs</i>	1	\$2,157.85
1-12, 13b, 14-19, 20i, 21-24	503489	Focus One Arm Drive Retrofit Kit with Poly Coated Drive Handrim, 20.69" Drive Shaft <i>Used on 22" wide chairs</i>	1	\$2,157.85
1-9	504339	One Arm Drive Axle Assembly, Focus	1	\$878.08

Item Number	Part Number	Description	Quantity	Retail
		<i>This kit contains 003211 and Installation Instructions.</i>		
11, 16, 21	003212	One Arm Drive Focus Axle Plate Assembly PAIR	1	\$568.00
10	101843	M5 x 20 SHCS Blk Zn	8	\$0.52
12	100657	M5 Nylock Nut Blk Zn	8	\$0.52
13a	003198	One Arm Drive Handrim Aluminum 24"	1	\$257.50
13b	003201	One Arm Drive Handrim Poly Coated 24"	1	\$406.85
14	100975	Handrim Aluminum Anodized 5 Rivet Nut 24"	1	\$56.65
15	100960	Wheel X Core Mag 5-Spoke 24" Black	1	\$133.90
17a	101831	M8 x 80 HHCS Blk Zn	4	\$3.01
17b	101832	M8 x 90 HHCS Blk Zn	4	\$3.61
17c	102384	M8 x 100 HHCS Blk Zc Part THD	2	\$3.61
18	101858	Washer Flat M8 Blk Zn	8	\$0.52
19	100659	M8 Nylock Nut Blk Zn	4	\$1.03
20e	504347	One Arm Drive - Drive Shaft 16.69", Focus <i>Used on 18" wide chairs. This kit contains 102645 Drive Shaft and Installation Instructions.</i>	1	\$551.78
20f	504348	One Arm Drive - Drive Shaft 17.69", Focus <i>Used on 19" wide chairs. This kit contains 102646 Drive Shaft and Installation Instructions.</i>	1	\$551.78
20i	504351	One Arm Drive - Drive Shaft 20.69", Focus <i>Used on 22" wide chairs. This kit contains 102752 Drive Shaft and Installation Instructions.</i>	1	\$551.78
22-24	003238	24" X Core Mag with One Arm Drive Hub	1	\$249.69