


# Catalyst 5Ti Swing Away Front Frame

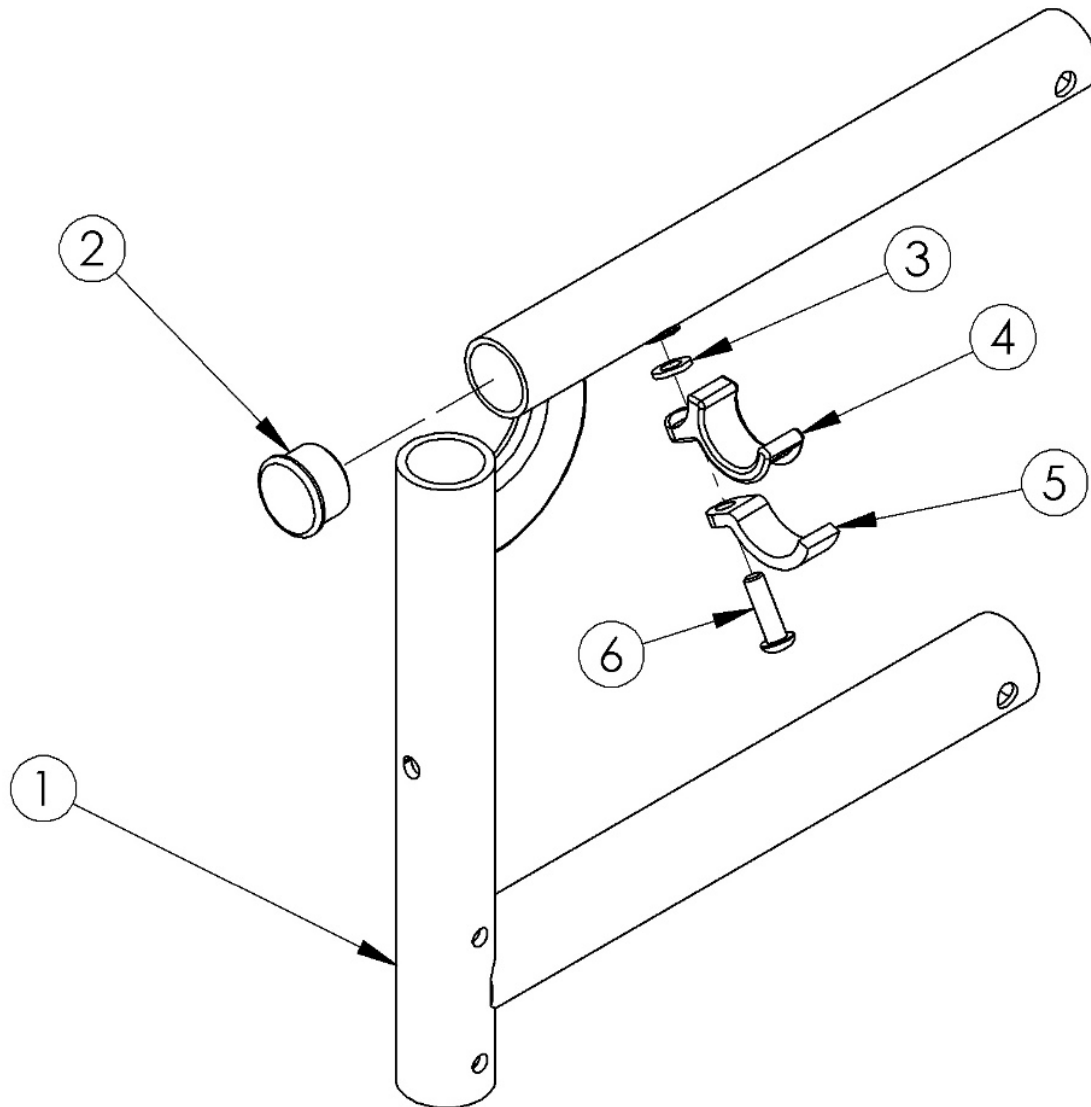

Frame Assemblies are based on Seat Depth. <p>

If switching to a new Seat Depth, new Height Adjustable Flip Back T-Arms may be required

Item Number	Part Number	Description	Quantity	Retail
3, 4, 5, 6	501005	Seat Hook and Support Hardware Kit, Catalyst <i>This kit contains quantity of 1 each</i>	1	\$17.06
1a, 2	100270	Frame Assm Cat TI Front Left 14	1	\$381.04
1b, 2	100271	Frame Assm Cat TI Front Left 15	1	\$514.16

Item Number	Part Number	Description	Quantity	Retail
1c, 2	100272	Frame Assm Cat TI Front Left 16	1	\$514.16
1d, 2	100273	Frame Assm Cat TI Front Left 17	1	\$514.16
1e, 2	501639	Frame Assm Cat Ti Front Left 18	1	\$386.94
1f, 2	501640	Frame Assm Cat Ti Front Left 19	1	\$422.81
1g, 2	100274	Frame Assm Cat TI Front Left 20	1	\$514.16
1h, 2	100279	Frame Assm Cat TI Front Right 14	1	\$514.16
1i, 2	100280	Frame Assm Cat TI Front Right 15	1	\$514.16
1j, 2	100281	Frame Assm Cat TI Front Right 16	1	\$514.16
1k, 2	100282	Frame Assm Cat TI Front Right 17	1	\$514.16
1l, 2	501641	Frame Assm Cat Ti Front Right 18	1	\$386.94
1m, 2	501642	Frame Assm Cat Ti Front Right 19	1	\$422.81
1n, 2	100283	Frame Assm Cat TI Front Right 20	1	\$514.16
2	100804	Tube End Plug 1"	1	\$1.55
3	100831	1/2" Washer OD .040" Thick	1	\$1.55
4	000003	Seat Hook CAT	1	\$7.55
5	000004	Seat Hook Support CAT	1	\$6.42
6	100668	M6 x 20 CLS10.9 BTN SHCS ZC w/Patch	1	\$1.55

<b>Code</b>	<b>Color Name</b>
<b>Matte</b>	
AGM	Army Green
EGM	Emerald Green
GGM	Graphite Gray
IBM	Iridescent Burgundy
INM	Iridescent Navy
RRM	Retro Red
RBM	Royal Blue
SBM	Shadow Black
SWM	Smooth White
<b>Gloss</b>	
BLK	Black
SKY	Blue Skies
PNK	Bubblegum Pink
CBL	Candy Blue
PUR	Candy Purple
RED	Candy Red
COP	Copper
FUS	Fuchsia
GSM	Granny Smith
ORG	Orange
FLM	Graphic Image Transfer
<b>Discontinued</b>	
BLU	Blue
BUR	Burgundy
EBL	Electric Blue
GRN	Forest Green
SAP	Sapphire
GRY	Slate Gray