


# (Discontinued 1) Focus CR Dual Hand Tilt Mechanism Stroller Handle Back

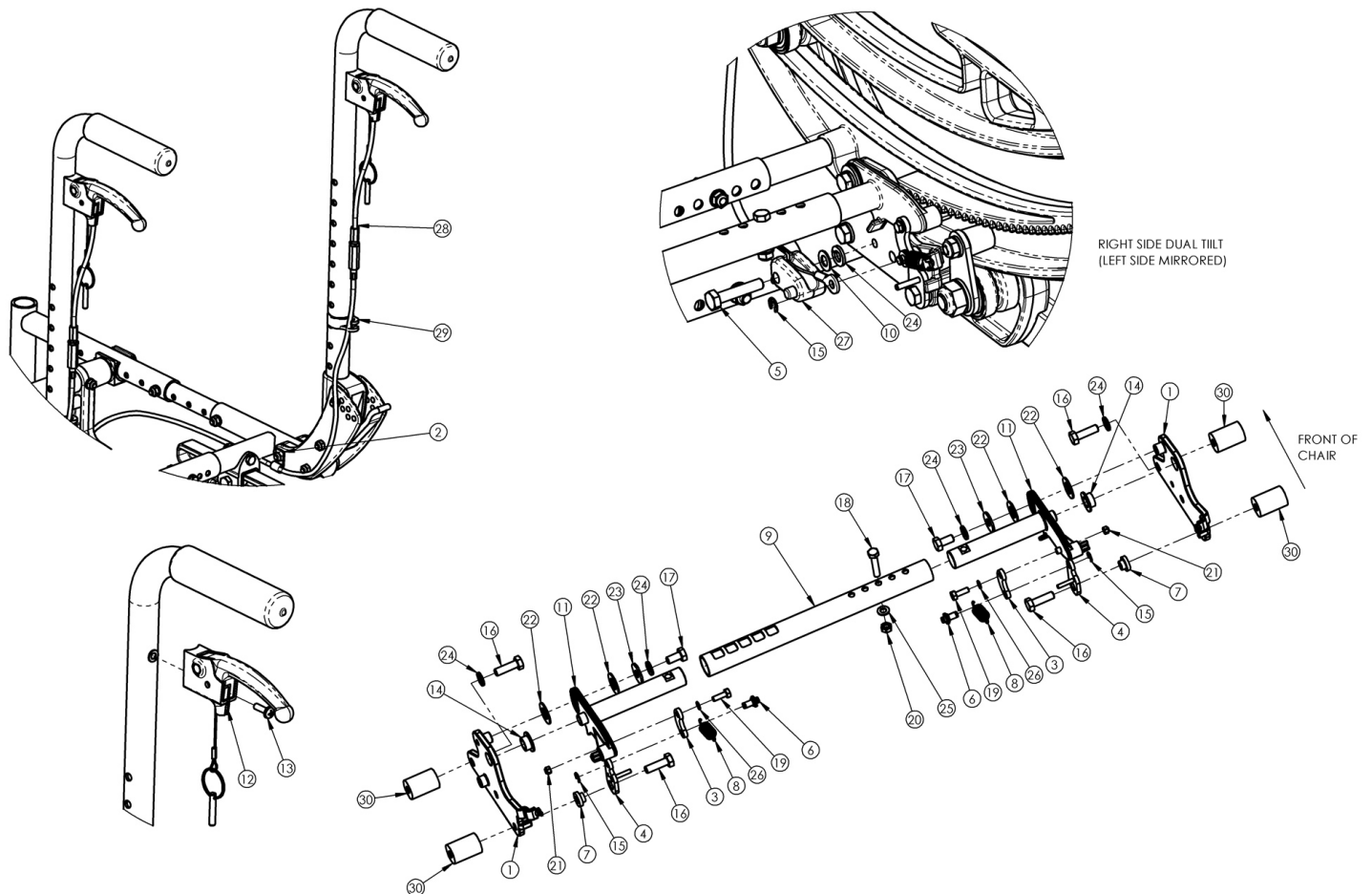

*This Focus CR Hand Tilt is discontinued and was used on chairs with serial numbers prior to FCR0025162 and FCHD002316.*

Item Number	Part Number	Description	Quantity	Retail
2, 12, 13, 28a, 29	503777	Dual Hand Tilt / Trigger Kit for Short Stroller Handle/Fixed Height/Recline Back, Focus	1	\$282.96
2, 12, 13, 28b, 29	503778	Dual Hand Tilt / Trigger Kit for Tall Stroller Handle/Fixed Height/Recline Back, Focus	1	\$285.06
1a, 14	503779	Tilt Lock Mount Bracket w/ Bearing Kit, Right, Focus, Dual Fixed Height	1	\$18.89
1b, 14	503780	Tilt Lock Mount Bracket w/ Bearing Kit, Left, Focus, Dual Fixed Height	1	\$18.89
2	001239	P-Clip 3/16"	2	\$6.18
3	003247	Link, Drive, Tilt	2	\$12.62
4a	002084	Toggle, Drive, Tilt, R	1	\$11.06
4b	002085	Toggle, Drive, Tilt, L	1	\$20.05

Item Number	Part Number	Description	Quantity	Retail
5	101824	M6 x 30 HHCS Blk Zn	2	\$2.06
6	003364	Pin, Toggle, Tilt	2	\$7.11
7	002088	Bushing, Toggle, Lower, Tilt	2	\$6.62
8	002089	Spring, Ext, 3/8ODx1.0LGx.041W	2	\$3.31
9a	002094	Growth Strut, Tilt Lock, Small	1	\$32.96
9b	002095	Growth Strut, Tilt Lock, Med	1	\$35.02
9c	002096	Growth Strut, Tilt Lock, Large	1	\$37.08
10	102803	Washer Flat .250 x .500 x .027 HX BLZ	2	\$1.55
11a	002411	Cog Lever, Tilt Lock, R	1	\$37.83
11b	002412	Cog Lever, Tilt Lock, L	1	\$37.83
12, 13	110139	Lever Assembly with Lanyard Vertical Tilt	2	\$33.33
13	100734	M5 x 16 PHMS ZC	2	\$1.55
16a	101823	M6 x 20 HHCS Blk Zn	4	\$1.03
16b	100674	M6 x 45 CLS10.9 DIN931 HHCS ZC <i>Used with Heavy Duty option/spacer</i>	2	\$1.55
16c	101826	M6 x 50 HHCS Blk Zn <i>Used with Heavy Duty option/spacer</i>	2	\$2.06
17	101822	M6 x 12 HHCS BLKZC w/Patch	2	\$1.03
18	101821	M5 x 25 HHCS Blk Zn	1	\$1.03
19	102783	M3 x 12 HHCS, CL8.8, Blk Zn	2	\$1.03
20	100657	M5 Nylock Nut Blk Zn	1	\$0.52
21	102784	Nut, Nylock, M3, Blk	2	\$1.03
22	102082	Washer Nylon .406IDX.75ODX.040	4	\$0.55
23	101853	M6 Washer Blk Zn	2	\$0.52
24	101835	M6 Lock Washer Blk Zn	5	\$0.52
25	101856	Flat Washer, M5, Blk Zn	1	\$0.52
26	102785	Flat Washer, M3, Blk Zn	2	\$0.52
27	003387	Cable Mount, Tilt	2	\$3.61
28a	002100	Cable, Tilt, Short	2	\$105.06

Item Number	Part Number	Description	Quantity	Retail
		<i>Used with short back height range</i>		
28b	002101	Cable, Tilt, Med <i>Used with tall back height range</i>	2	\$105.06
29	001117	Cable Clip 1"	2	\$3.75
30	001405	Spacer, .69OD x .24ID x .1.01Lg <i>Only used with Heavy Duty Option</i>	4	\$25.54