


# Discontinued Focus CR Height Adjustable T-Arm

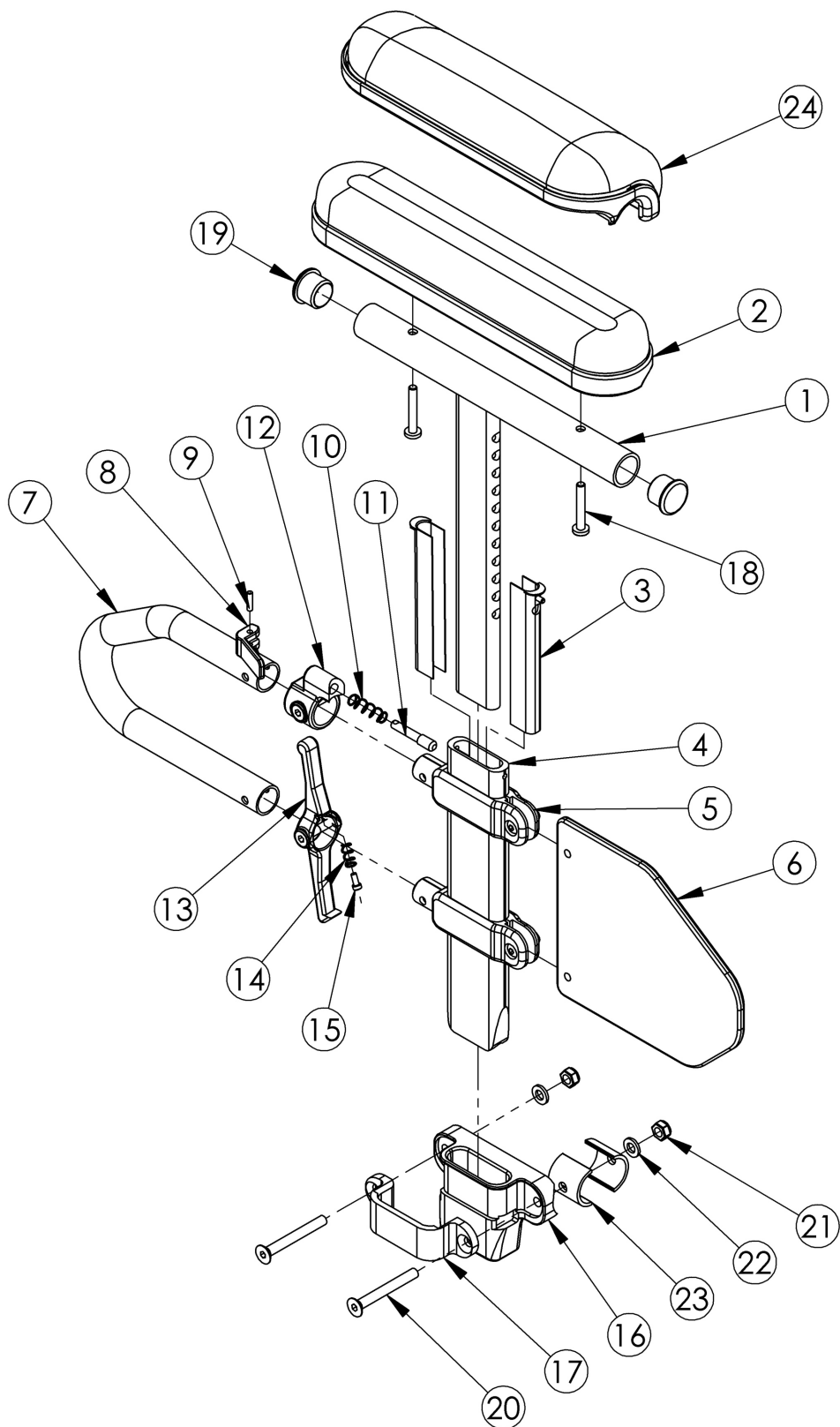

Item Number	Part Number	Description	Quantity	Retail
1a, 3-5, 6, 7-15, 19	100238	Height Adj T Armrest Short Desk Right <i>Replaced with 003463</i>	1	\$203.90
1a, 3-5, 6, 7-15, 19	100242	Height Adj T Armrest Short Desk Left <i>Replaced with 003463</i>	1	\$203.90
1b, 3-5, 6, 7-15, 19	100239	Height Adj T Armrest Short Full Right <i>Replaced with 003464</i>	1	\$203.90
1b, 3-5, 6, 7-15, 19	100243	Height Adj T Armrest Short Full Left <i>Replaced with 003464</i>	1	\$203.90
1a, 2b, 3-5, 6, 7-15, 18, 19	501682	Height Adj T-Arm Assm Short Desk w/ Std Pad Left <i>Replaced with 505646</i>	1	\$210.52
1a, 2b, 3-5, 6, 7-15, 18, 19	501683	Height Adj T-Arm Assm Short Desk w/ Std Pad Right <i>Replaced with 505646</i>	1	\$210.52
1b, 2a, 3-5, 6, 7-15, 18, 19	501684	Height Adj T-Arm Assm Short Full w/ Std Pad Left <i>Replaced with 505644</i>	1	\$210.52
1b, 2a, 3-5, 6, 7-15, 18, 19	501685	Height Adj T-Arm Assm Short Full w/ Std Pad Right <i>Replaced with 505644</i>	1	\$210.52
1a, 2d, 3-5, 6, 7-15, 18, 19	501690	Height Adj T-Arm Assm Short Desk w/ Waterfall Pad Left <i>Replaced with 505650</i>	1	\$219.96
1a, 2d, 3-5, 6, 7-15, 18, 19	501691	Height Adj T-Arm Assm Short Desk w/ Waterfall Pad Right <i>Replaced with 505650</i>	1	\$219.96
1b, 2c, 3-5, 6, 7-15, 18, 19	501692	Height Adj T-Arm Assm Short Full w/ Waterfall Pad Left <i>Replaced with 505648</i>	1	\$219.96
1b, 2c, 3-5, 6, 7-15, 18, 19	501693	Height Adj T-Arm Assm Short Full w/ Waterfall Pad Right <i>Replaced with 505648</i>	1	\$219.96
16, 17, 20-23	502298	Armrest T-Arm Receiver Assembly Focus	1	\$54.84
1a, 2b, 3-5, 6, 7-23	502299	Armrest Assembly T-Arm Short Desk w/ 1" Receiver Right Focus <i>Replaced with 506083</i>	1	\$272.84
1a, 2b, 3-5, 6, 7-23	502300	Armrest Assembly T-Arm Short Desk w/ 1" Receiver Left Focus <i>Replaced with 506083</i>	1	\$272.84
1b, 2a, 3-5, 6, 7-23	502303	Armrest Assembly T-Arm Short Full w/ 1" Receiver Right Focus <i>Replaced with 506084</i>	1	\$272.84
1b, 2a, 3-5, 6, 7-23	502304	Armrest Assembly T-Arm Short Full w/ 1" Receiver Left Focus <i>Replaced with 506084</i>	1	\$272.84
1a, 2b, 3-5, 6, 7-23	501140	Armrest Assembly T-Arm Short Desk w/ 1" Receiver Pair Focus <i>Replaced with 506087</i>	1	\$545.67

Item Number	Part Number	Description	Quantity	Retail
1b, 2a, 3-5, 6, 7-23	501141	Armrest Assembly T-Arm Short Full w/ 1" Receiver Pair Focus <i>Replaced with 506088</i>	1	\$545.67
1a, 2d, 3-5, 6, 7-23	502385	Height Adj T-Arm Assm Short Desk w/ Waterfall Pad and 1" Receiver Right Focus <i>Replaced with 506085</i>	1	\$293.25
1a, 2d, 3-5, 6, 7-23	502386	Height Adj T-Arm Assm Short Desk w/ Waterfall Pad and 1" Receiver Left Focus <i>Replaced with 506085</i>	1	\$293.25
1b, 2c, 3-5, 6, 7-23	502389	Height Adj T-Arm Assm Short Full w/ Waterfall Pad and 1" Receiver Right Focus <i>Replaced with 506086</i>	1	\$293.25
1b, 2c, 3-5, 6, 7-23	502390	Height Adj T-Arm Assm Short Full w/ Waterfall Pad and 1" Receiver Left Focus <i>Replaced with 506086</i>	1	\$293.25
1a, 2d, 3-5, 6, 7-23	502393	Height Adj T-Arm Assm Short Desk w/ Waterfall Pad and 1" Receiver Pair Focus <i>Replaced with 506089</i>	1	\$586.51
1b, 2c, 3-5, 6, 7-23	502394	Height Adj T-Arm Assm Short Full w/ Waterfall Pad and 1" Receiver Pair Focus <i>Replaced with 506090</i>	1	\$586.51
1a, 19	000091	Height Adj T Armrest Upper Desk	1	\$73.57
1b, 19	000101	Height Adj T Armrest Upper Full	1	\$73.57
2a	100500	Arm Pad Full	1	\$23.18
2b	100501	Arm Pad Desk	1	\$23.18
3	000090	Height Adj T Armrest Shim	2	\$2.50
4	000089	Height Adj T Armrest Post	1	\$24.61
5	000092	Height Adj T Armrest Clamp	2	\$8.79
6	000097	Height Adj T Armrest Side Guard Short	1	\$14.06
7	000095	Height Adj T Armrest Transfer Tube	1	\$27.52
8	000098	Height Adj Armrest Lever	1	\$3.51
9	200257	Dowel Pin 1/8 x 1/2	1	\$2.58
10	000134	Height Adj Armrest Lever Spring	1	\$1.09
11	000099	Height Adj Armrest Lever Pin	1	\$18.75
12	000096	Height Adj T Armrest Pin Lock Housing	1	\$14.42
13	000100	Height Adj T Armrest Release Lever	1	\$9.14
14	000135	Height Adj T Armrest Release Lever Spring	1	\$1.07
15	000152	Height Adj T Armrest Release Spring Button	1	\$1.07

Item Number	Part Number	Description	Quantity	Retail
16	000087	Height Adjustable T-Arm Receiver 1" CAT	1	\$14.95
17	000088	Height Adj T Armrest Bracket	1	\$26.88
18a	100696	M5 x 30mm PPH Capscrew <i>Used with Standard Arm Pads</i>	2	\$1.55
18b	100654	M5 x 40 DIN7985 PH Pan HD M/S ZC <i>Used with Waterfall Arm Pads</i>	2	\$1.55
19	100803	Tube End Plug 7/8"	2	\$1.55
20	101592	M6 x 45 CLS10.9 DIN7991 FHCS BLKZC	2	\$2.58
21	100658	M6 Nylock Nut Blk Zn	2	\$1.03
22	100746	M6 Flat Washer Blk Zn	2	\$1.55
23	001230	Tube Spacer 1 x .083	2	\$3.30
24a	102452	Waterfall Arm Pad Full	1	\$32.16
24b	102521	Waterfall Arm Pad Desk	1	\$32.16
8-12	500541	Arm Height Adj T Upper Lever Assembly	1	\$39.76