


Catalyst Angle Adjustable Backrest

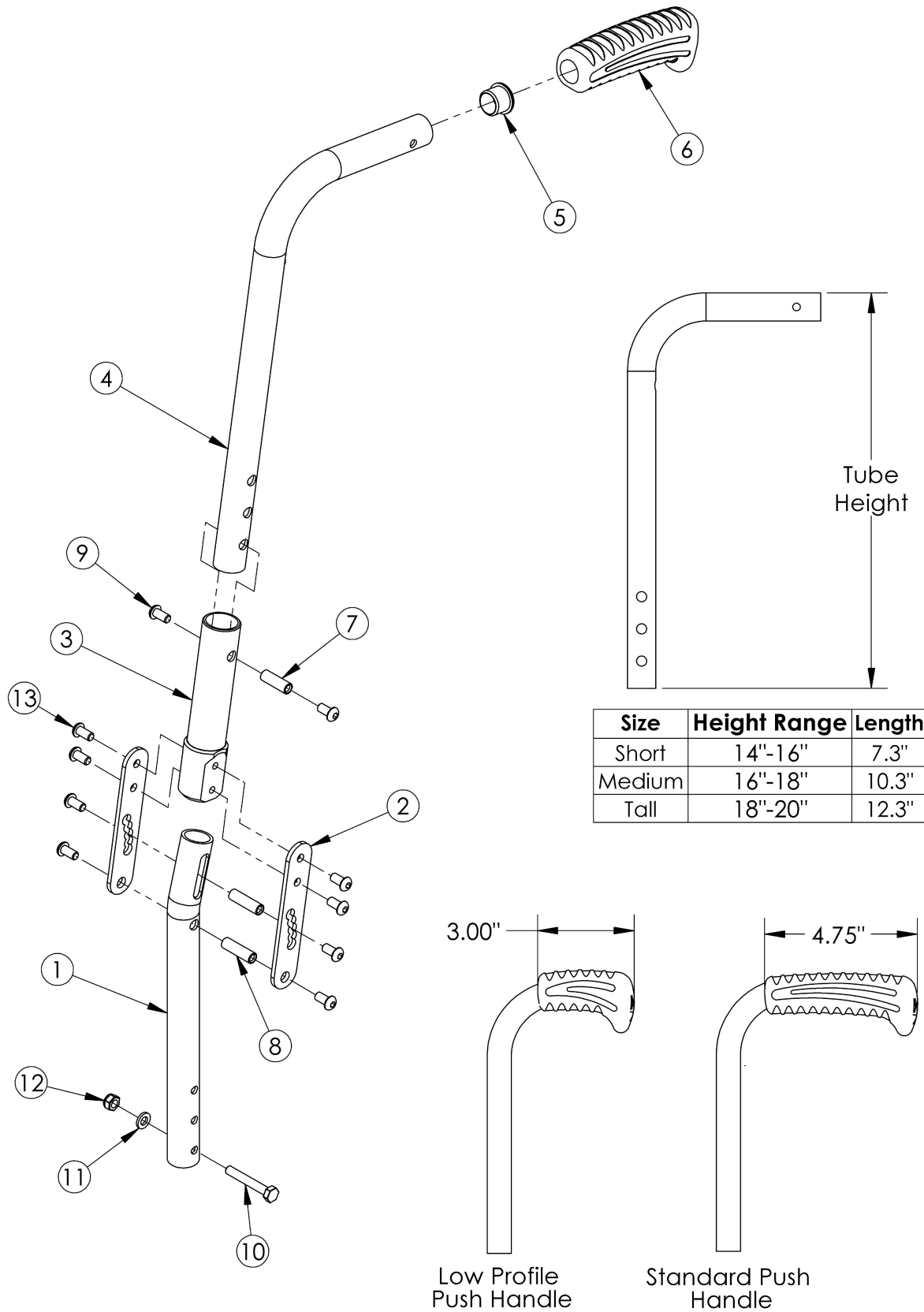

Item Number	Part Number	Description	Quantity	Retail
1	000146	Angle Adjustable Backrest Tube Lower CAT	1	\$21.47
2	000148	Angle Adjustable Backrest Mounting Plate CAT	2	\$7.95
3	000147	Angle Adjustable Backrest Housing CAT	1	\$99.41
4a, 5, 6a	500032	Angle Adjustable Back Post Short Upper Assembly with Standard Push Handle	1	\$66.80
4b, 5, 6a	500033	Angle Adjustable Back Post Medium Upper Assembly with Standard Push Handle	1	\$69.11
4c, 5, 6a	500034	Angle Adjustable Back Post Tall Upper Assembly with Standard Push Handle	1	\$71.43
4a, 5, 6b	002755	Angle Adjustable Back Post Short Upper Assembly with Low Profile Push Handle	1	\$66.80
4b, 5, 6b	002756	Angle Adjustable Back Post Medium Upper Assembly with Low Profile Push Handle	1	\$69.11
4c, 5, 6b	002757	Angle Adjustable Back Post Tall Upper Assembly with Low Profile Push Handle	1	\$71.43
7	000065	M6 x 23.68 Threaded Barrel	1	\$3.93
8	000066	M6 x 27 Threaded Barrel	2	\$4.24
9	101184	M6 x 10 BTN SHCS BLKZC w/Patch	2	\$1.50
10	100681	M6 x 40 CLS8.8 DIN931 HHCS ZC	1	\$1.50
11	100746	M6 Flat Washer Blk Zn	1	\$1.50
12	100658	M6 Nylock Nut Blk Zn	1	\$1.00
13	100667	M6 x 12 BTN SHCS BLKZC w/Patch	8	\$1.50
1 - 3, 7 - 13	500031	Angle Adjustable Back Post Lower Assm CAT	1	\$107.90