


Catalyst Depth Adjustable Backrest with Hardware

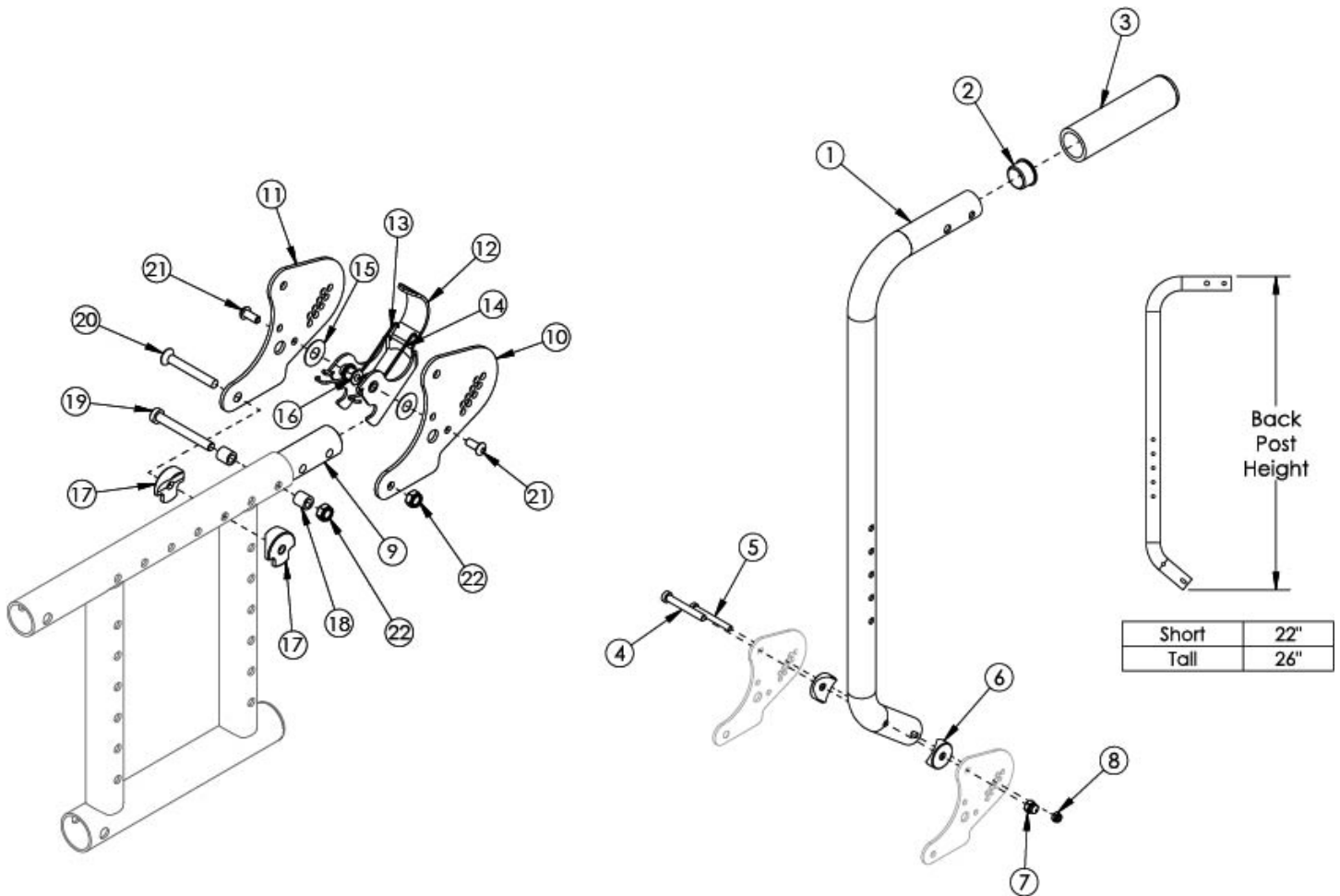

Item Number	Part Number	Description	Quantity	Retail
1a - 9, 10a, 11-22	111329	Back Post Depth Adjustable with Push Handle Short Assm with All Hardware CAT/ SPARK - Left Side	1	\$209.62
1a - 9, 10b, 11-22	111330	Back Post Depth Adjustable with Push Handle Short Assm with All Hardware CAT/ SPARK - Right Side	1	\$209.62
1b - 9, 10a, 11-22	111331	Back Post Depth Adjustable with Push Handle Tall Assm with All Hardware CAT/ SPARK - Left Side	1	\$209.62
1b - 9, 10b, 11-22	111332	Back Post Depth Adjustable with Push Handle Tall Assm with All Hardware CAT/ SPARK - Right Side	1	\$209.62
1a, 2, 3	500613	Back Post Depth Adjustable with Push Handle Short Assm CAT/ SPARK	1	\$47.78
1b, 2, 3	500614	Back Post Depth Adjustable with Push Handle Tall Assm CAT/ SPARK	1	\$47.78
1a - 8	501206	Back Post Depth Adjustable with Push Handle Short Assm with Hardware CAT/ SPARK	1	\$58.08
1b - 8	501207	Back Post Depth Adjustable with Push Handle Tall Assm with Hardware CAT/ SPARK	1	\$58.08

Item Number	Part Number	Description	Quantity	Retail
9, 10a, 11-22	500738	Depth Adjustable Backrest Hardware Assm Left CAT	1	\$145.13
9, 10b, 11-22	500739	Depth Adjustable Backrest Hardware Assm Right CAT	1	\$145.13
9, 11-22	501227	Depth Adjustable Backrest Hardware Assm SPARK/ OPEN SEAT	1	\$145.13
4-8	111164	Depth Adjustable Backrest Attaching Hardware Kit	1	\$9.29
2	100804	Tube End Plug 1"	1	\$1.55
4	101123	M6 x 45 Low SHCS BLK ZC	1	\$1.55
5	101186	M5 X 45 CLS12.9 DIN912 SHCS P/THD BLZ	1	\$3.09
6	000055	Saddle 1"	2	\$1.55
7	100558	M6 NYL INS Domed Cap L/N ZC	1	\$1.03
8	100657	M5 Nylock Nut Blk Zn	1	\$0.52
9	000605	Depth Adjustable Frame Extension Tube CAT/ SPARK	1	\$36.41
10a	000604L	Depth Adjustable Backrest Plate Inside Left CAT/ SPARK	1	\$22.75
10b	000604R	Depth Adjustable Backrest Plate Inside Right CAT/ SPARK	1	\$22.75
11	000611	Depth Adjustable Backrest Side Plate CAT/ SPARK	2	\$22.75
12	000670	Depth Adjustable Backrest Latch CAT/ SPARK	1	\$31.85
13	000613R	Depth Adjustable Backrest Latch Spring Right CAT/ SPARK	1	\$2.71
14	000613L	Depth Adjustable Backrest Latch Spring Left CAT/ SPARK	1	\$2.71
15	000428	Wear Washer	2	\$2.58
16	000420	Backrest Latch Pivot Bushing	2	\$2.50
17	000612	1" Half Saddle Washer	2	\$1.88
18	000240	Backrest Latch Strike Bushing	2	\$3.50
19	101124	M6 x 50 Low SHCS BLK ZC	1	\$1.55
20	101592	M6 x 45 CLS10.9 DIN7991 FHCS BLKZC	1	\$2.58
21	100669	M5 x 10 BTN SHCS BLKZC w/Patch	2	\$1.55
22	100658	M6 Nylock Nut Blk Zn	2	\$1.03