


# Catalyst 5Vx Side Frame Assembly

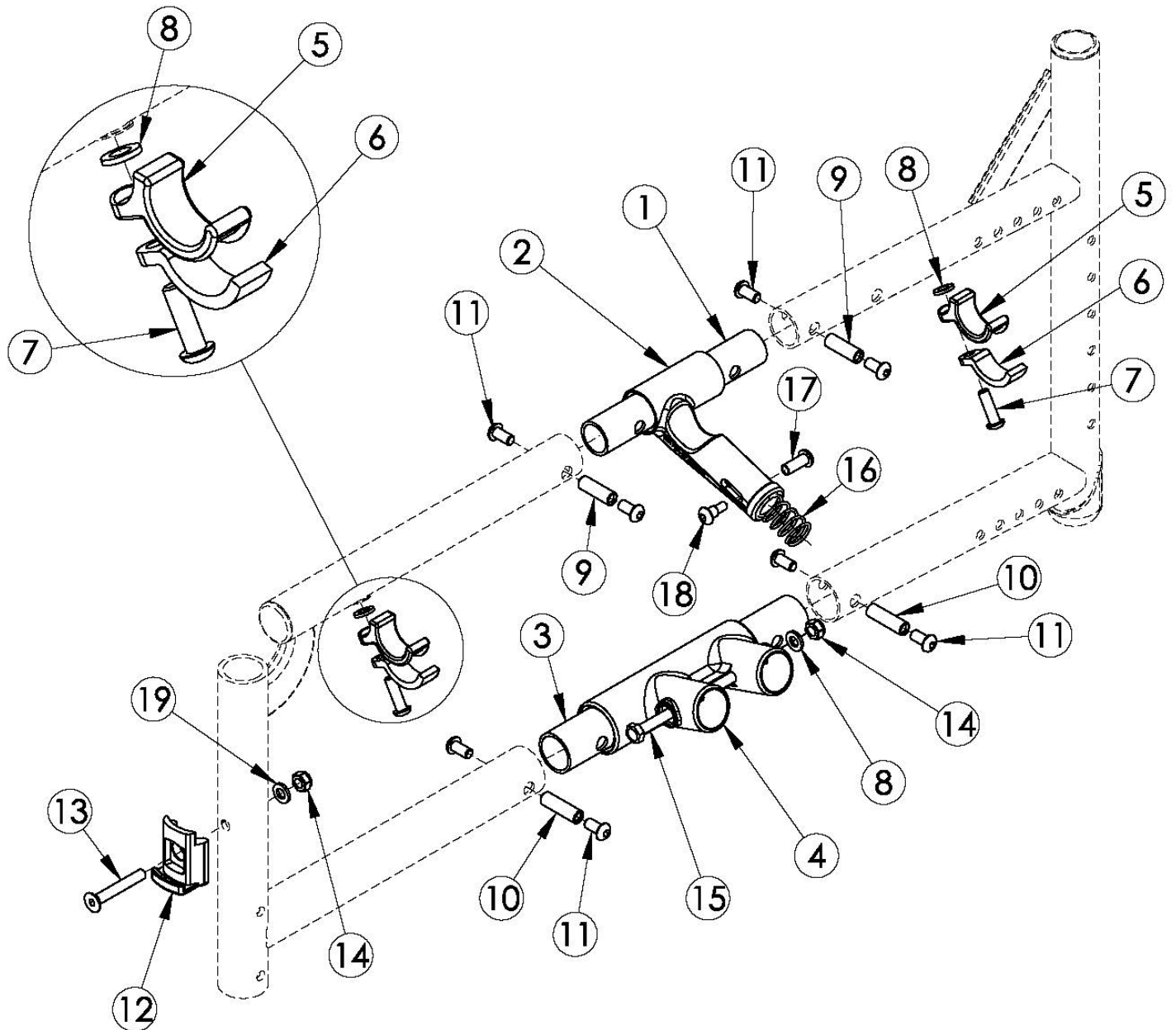

Item Number	Part Number	Description	Quantity	Retail
2a, 16 - 18	500024	Upper Pivot Assembly 14" <i>This kit also contains the 14" slider link (not pictured)</i>	1	\$43.89
2b, 16 - 18	500025	Upper Pivot Assembly 15" <i>This kit also contains the 15" slider link (not pictured)</i>	1	\$43.89
2b, 16 - 18	500026	Upper Pivot Assembly 16" <i>This kit also contains the 16" slider link (not pictured)</i>	1	\$43.89
2b, 16 - 18	500027	Upper Pivot Assembly 17"	1	\$43.89

Item Number	Part Number	Description	Quantity	Retail
		<i>This kit also contains the 17" slider link (not pictured)</i>		
2b, 16 - 18	500028	Upper Pivot Assembly 18" <i>This kit also contains the 18" slider link (not pictured)</i>	1	\$43.89
2b, 16 - 18	500029	Upper Pivot Assembly 19" <i>This kit also contains the 19" slider link (not pictured)</i>	1	\$43.89
2b, 16 - 18	500030	Upper Pivot Assembly 20" <i>This kit also contains the 20" slider link (not pictured)</i>	1	\$43.89
2b, 16 - 18	501479	Upper Pivot Assembly 21/22" <i>This kit also contains the 21/22" slider link (not pictured)</i>	1	\$43.89
5, 6, 7, 8	501005	Seat Hook and Support Hardware Kit, Catalyst <i>This kit contains quantity of 1 each</i>	1	\$17.06
1	000011	Frame Connector Upper CAT	1	\$4.49
2a	001454	Upper Frame Pivot 14" Wide CAT <i>If wanting to replace the Upper Pivot, order the Upper Pivot Assembly 500024</i>	1	\$22.10
2b	000002	Upper Frame Pivot 15" - 22" Wide CAT <i>If wanting to replace the Upper Pivot, order the Upper Pivot Assembly based on the chair width</i>	1	\$22.33
3	000009	Frame Connector Lower CAT	1	\$4.49
4	000001	Lower Frame Pivot CAT	1	\$27.17
5	000003	Seat Hook CAT	2	\$7.55
6	000004	Seat Hook Support CAT	2	\$6.42
7	100668	M6 x 20 CLS10.9 BTN SHCS ZC w/Patch	2	\$1.55
8	100831	1/2" Washer OD .040" Thick	3	\$1.55
9	000065	M6 x 23.68 Threaded Barrel <i>Additional 2 are needed with chairs 18"-20" deep</i>	2	\$4.05
10	000066	M6 x 27 Threaded Barrel	2	\$4.37
11	100667	M6 x 12 BTN SHCS BLKZC w/Patch	8	\$1.55
12	000336	Latch Block	1	\$36.78
13	101819	M6 x 40 FHCS Blk Zn	1	\$1.03
14	100658	M6 Nylock Nut Blk Zn	2	\$1.03
15	100704	M6 x 90 CLS8.8 DIN931 HHCS ZC	1	\$1.55
19	100746	M6 Flat Washer Blk Zn	1	\$1.55