


# Catalyst Flip Back Armrest

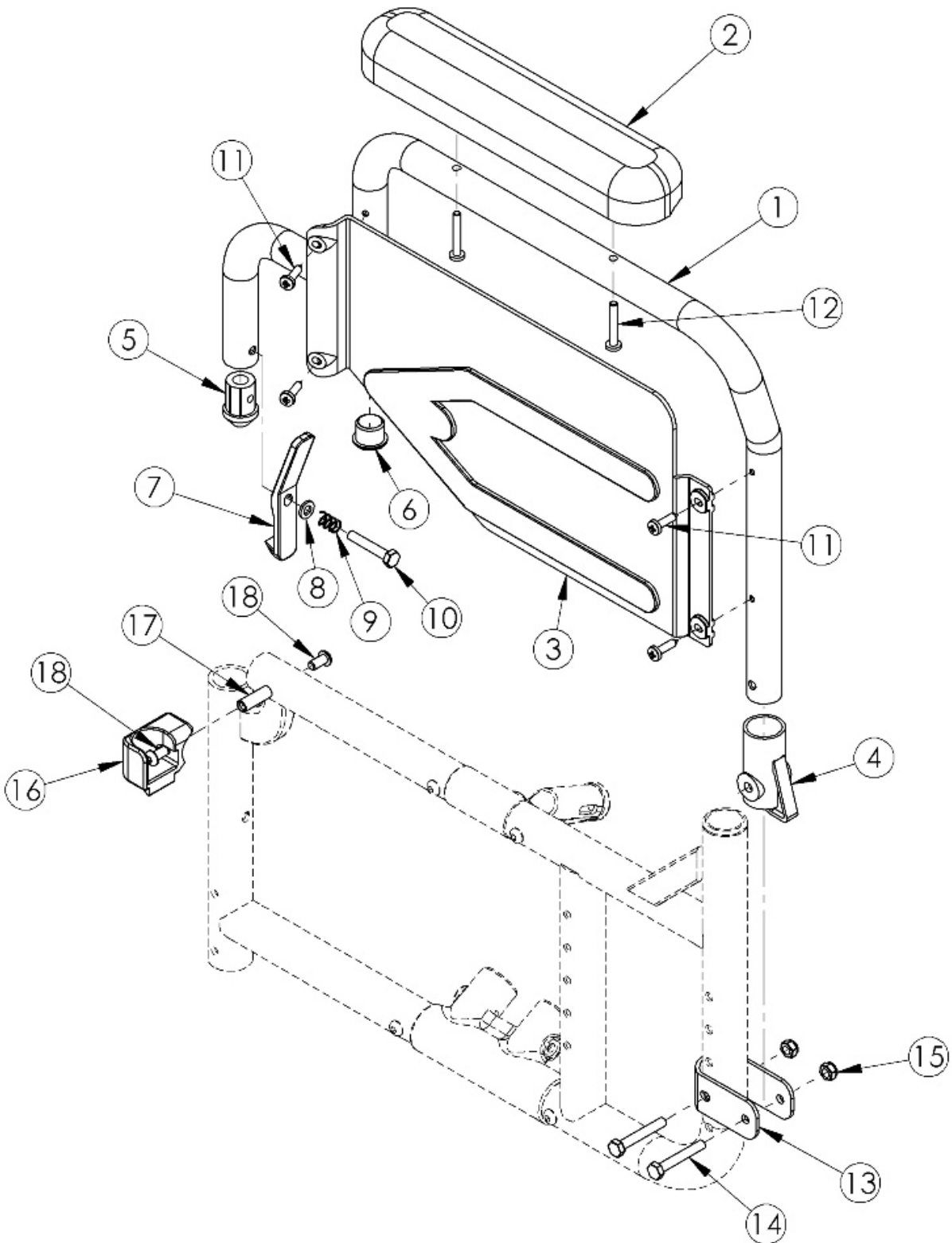

If retrofitting to Flip Back Armrests, new front frames are required.

Item Number	Part Number	Description	Quantity	Retail
1a, 2a, 3b, 4 - 12, 13a, 14, 15, 16a, 17, 18	502255	Armrest Flip Back w/ Receiver and Bracket Kit Left Desk CAT4	1	\$152.46
1c, 2a, 3a, 4 - 12, 13a, 14, 15, 16b, 17, 18	502256	Armrest Flip Back w/ Receiver and Bracket Kit Right Desk CAT4	1	\$152.46
1b, 2b, 3b, 4 - 12, 13a, 14, 15, 16a, 17, 18	502257	Armrest Flip Back w/ Receiver and Bracket Kit Left Full CAT4	1	\$152.46
1d, 2b, 3a, 4 - 12, 13a, 14, 15, 16b, 17, 18	502258	Armrest Flip Back w/ Receiver and Bracket Kit Right Full CAT4	1	\$152.46
1a, 2a, 3b, 4 - 12, 13b, 14, 15, 16a, 17, 18	502259	Armrest Flip Back w/ Receiver and Bracket Kit Left Desk CAT5	1	\$152.46
1c, 2a, 3a, 4 - 12, 13b, 14, 15, 16b, 17, 18	502260	Armrest Flip Back w/ Receiver and Bracket Kit Right Desk CAT5	1	\$152.46
1b, 2b, 3b, 4 - 12, 13b, 14, 15, 16a, 17, 18	502261	Armrest Flip Back w/ Receiver and Bracket Kit Left Full CAT5	1	\$152.46
1d, 2b, 3a, 4 - 12, 13b, 14, 15, 16b, 17, 18	502262	Armrest Flip Back w/ Receiver and Bracket Kit Right Full CAT5	1	\$152.46
1a, 1c, 2a, 3a, 3b, 4 - 12, 13a, 14 - 18	502279	Armrest Flip Back w/ Receiver and Bracket Kit PAIR Desk CAT4	1	\$304.92
1a, 1c, 2b, 3a, 3b, 4 - 12, 13a, 14 - 18	502280	Armrest Flip Back w/ Receiver and Bracket Kit PAIR Full CAT4	1	\$304.92
1b, 1d, 2a, 3a, 3b, 4 - 12, 13b, 14 - 18	502281	Armrest Flip Back w/ Receiver and Bracket Kit PAIR Desk CAT5	1	\$304.92
1b, 1d, 2b, 3a, 3b, 4 - 12, 13b, 14 - 18	502282	Armrest Flip Back w/ Receiver and Bracket Kit PAIR Full CAT5	1	\$304.92
1a, 2a, 3b, 4 - 12	500734	Armrest Assembly Std Flip Back Desk Left CAT	1	\$136.68
1c, 2a, 3a, 4 - 12	500735	Armrest Assembly Std Flip Back Desk Right CAT	1	\$136.68
1b, 2b, 3b, 4 - 12	500736	Armrest Assembly Std Flip Back Full Left CAT	1	\$136.68
1d, 2b, 3a, 4 - 12	500737	Armrest Assembly Std Flip Back Full Right CAT	1	\$136.68
13a, 14, 15	502251	Armrest Flip Back Bracket Kit CAT4	1	\$12.43
13b, 14, 15	502252	Armrest Flip Back Bracket Kit CAT5	1	\$12.43
16a, 17, 18	502253	Armrest Flip Back Receiver Kit Left CAT	1	\$15.80
16b, 17, 18	502254	Armrest Flip Back Receiver Kit Right CAT	1	\$15.80
2a	100501	Arm Pad Desk	1	\$23.18

Item Number	Part Number	Description	Quantity	Retail
2b	100500	Arm Pad Full	1	\$23.18
3a	100687	Armrest Flip Back Panel Fixed Right	1	\$33.49
3b	100688	Armrest Flip Back Panel Fixed Left	1	\$33.49
4	100691	Armrest Flip Back Rear Pivot Bracket	1	\$3.29
5	100690	Armrest Flip Back Front Tube Plug	1	\$1.75
6	100803	Tube End Plug 7/8"	1	\$1.55
7	100689	Armrest Flip Back Release Lever	1	\$3.83
8	102072	Washer Nylon .234 ID x .406 OD x .062 Thick	1	\$0.52
9	100693	Compression Spring 0.34" OD x 0.70 LG	1	\$1.58
10	100860	M6 x 35 HHCS P/THD ZC	1	\$1.55
11	100723	10 x 3/4 PH Pan HD T-A T/S ZC	4	\$1.03
12	100696	M5 x 30mm PPH Capscrew	2	\$1.55
13a	000080	Armrest Flip Back Bracket CAT 4	1	\$8.06
13b	000280	Armrest Flip Back Bracket CAT 5	1	\$8.06
14	100681	M6 x 40 CLS8.8 DIN931 HHCS ZC	2	\$1.55
15	100658	M6 Nylock Nut Blk Zn	2	\$1.03
16a	000081	Armrest Flip Back Receiver Left CAT	1	\$9.39
16b	000082	Armrest Flip Back Receiver Right CAT	1	\$9.39
17	000065	M6 x 23.68 Threaded Barrel	1	\$4.05
18	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2	\$1.55