


Focus CR Hand Tilt Mechanism Stroller Handle Back

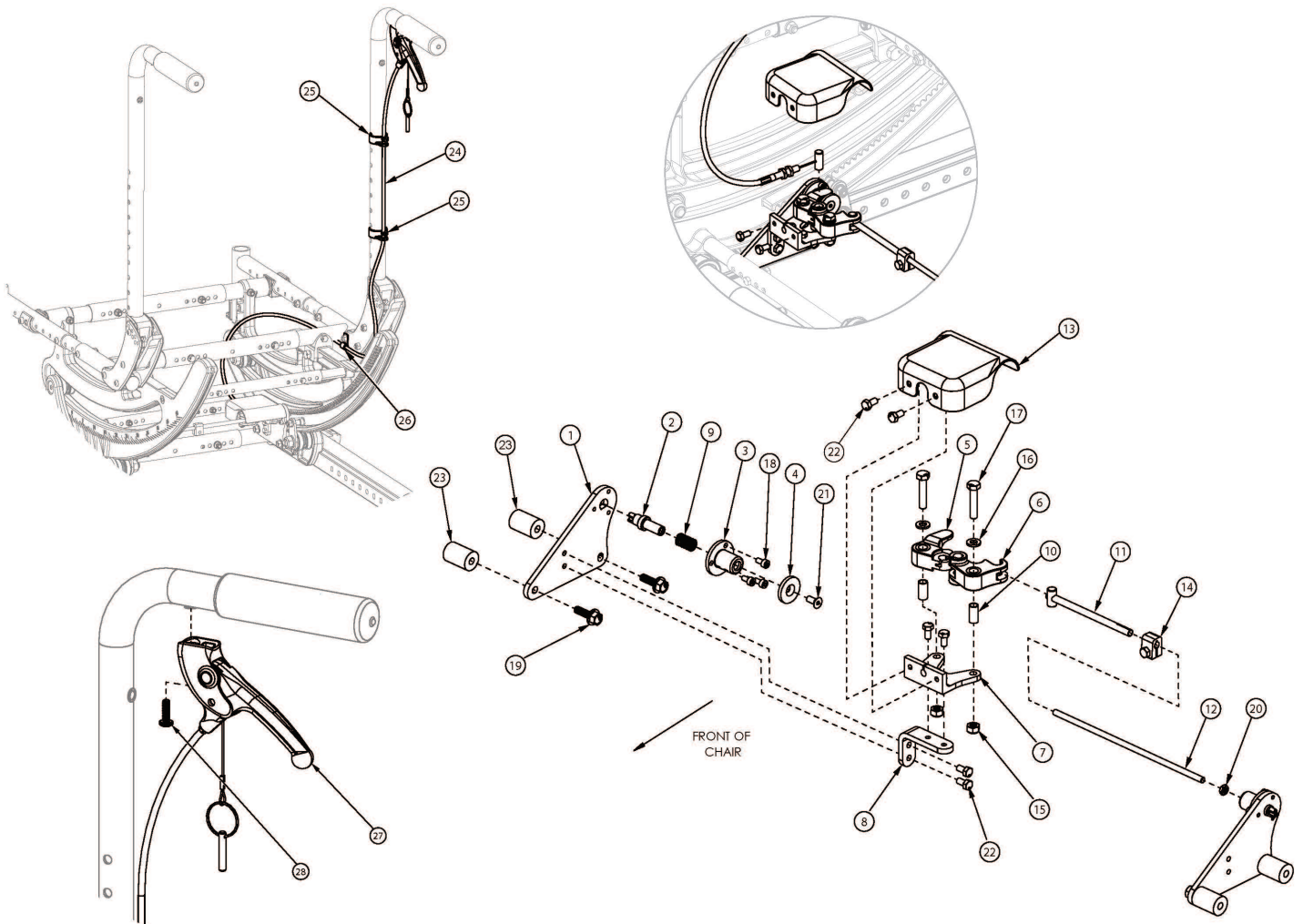

This design effective starting with serial numbers FCR0026599 and FCHD002333. <p>

If serial number is prior to FCR0026599 and FCHD002333 new back cane assemblies are required in addition to the hand tilt mechanism.

Item Number	Part Number	Description	Quantity	Retail
1-11, 12a, 13-18, 19a, 20-22	506246	Single Tilt Hand/Foot Kit, Right, Small Width Range, Focus <i>For complete tilt mechanism, select appropriate Hand Tilt Cable / Trigger Kit in addition to this kit</i>	1	\$200.52
1-11, 12b, 13-18, 19a, 20-22	506247	Single Tilt Hand/Foot Kit, Right, Medium Width Range, Focus <i>For complete tilt mechanism, select appropriate Hand Tilt Cable / Trigger Kit in addition to this kit</i>	1	\$221.81
1-11, 12c, 13-18, 19a, 20-22	506248	Single Tilt Hand/Foot Kit, Right, Large Width Range, Focus <i>For complete tilt mechanism, select appropriate Hand Tilt Cable / Trigger Kit in addition to this kit</i>	1	\$204.97
1-11, 12a, 13-18, 19a, 20-22	506266	Single Tilt Hand/Foot Kit, Left, Small Width Range, Focus	1	\$200.52

Item Number	Part Number	Description	Quantity	Retail
		<i>For complete tilt mechanism, select appropriate Hand Tilt Cable / Trigger Kit in addition to this kit</i>		
1-11, 12b, 13-18, 19a, 20-22	506267	Single Tilt Hand/Foot Kit, Left, Medium Width Range, Focus <i>For complete tilt mechanism, select appropriate Hand Tilt Cable / Trigger Kit in addition to this kit</i>	1	\$221.81
1-11, 12c, 13-18, 19a, 20-22	506268	Single Tilt Hand/Foot Kit, Left, Large Width Range, Focus <i>For complete tilt mechanism, select appropriate Hand Tilt Cable / Trigger Kit in addition to this kit</i>	1	\$204.97
1-11, 12a, 13-18, 19b, 20-23	506249	Single Tilt Hand/Foot Kit, Right, Small Width Range, Heavy Duty, Focus <i>For complete tilt mechanism, select appropriate Hand Tilt Cable / Trigger Kit in addition to this kit</i>	1	\$310.64
1-11, 12b, 13-18, 19b, 20-23	506250	Single Tilt Hand/Foot Kit, Right, Medium Width Range, Heavy Duty, Focus <i>For complete tilt mechanism, select appropriate Hand Tilt Cable / Trigger Kit in addition to this kit</i>	1	\$310.64
1-11, 12c, 13-18, 19b, 20-23	506251	Single Tilt Hand/Foot Kit, Right, Large Width Range, Heavy Duty, Focus <i>For complete tilt mechanism, select appropriate Hand Tilt Cable / Trigger Kit in addition to this kit</i>	1	\$310.64
1-11, 12a, 13-18, 19b, 20-23	506269	Single Tilt Hand/Foot Kit, Left, Small Width Range, Heavy Duty, Focus <i>For complete tilt mechanism, select appropriate Hand Tilt Cable / Trigger Kit in addition to this kit</i>	1	\$310.64
1-11, 12b, 13-18, 19b, 20-23	506270	Single Tilt Hand/Foot Kit, Left, Medium Width Range, Heavy Duty, Focus <i>For complete tilt mechanism, select appropriate Hand Tilt Cable / Trigger Kit in addition to this kit</i>	1	\$310.64
1-11, 12c, 13-18, 19b, 20-23	506271	Single Tilt Hand/Foot Kit, Left, Large Width Range, Heavy Duty, Focus <i>For complete tilt mechanism, select appropriate Hand Tilt Cable / Trigger Kit in addition to this kit</i>	1	\$310.64
11, 12a, 14, 20	506263	Linkage Rod Assembly,Tilt, Small <i>Used with Small Width Range</i>	1	\$37.49
11, 12b, 14, 20	506264	Linkage Rod Assembly,Tilt, Medium <i>Used with Medium Width Range</i>	1	\$37.70
11, 12c, 14, 20	506265	Linkage Rod Assembly,Tilt, Large <i>Used with Large Width Range</i>	1	\$38.26
24a, 25-28	506784	Hand Tilt Cable/Trigger Kit with Short Cable, Stroller Handle, Focus <i>Used with Short Stroller Handle Back Post</i>	1	\$130.00
24b, 25-28	506785	Hand Tilt Cable/Trigger Kit with Medium Cable, Stroller Handle, Focus <i>Used with Tall Stroller Handle Back Post</i>	1	\$130.86
1	004280	Mount Plate, Tilt Lock	2	\$14.21
2	004281	Pin, Plunger, Tilt Lock	2	\$60.46
3	004282	Housing, Plunger, Tilt Lock	2	\$10.30
4	004283	Thrust Washer, Tilt Lock	1	\$7.11

Item Number	Part Number	Description	Quantity	Retail
1-4, 9, 18, 19a	506259	Tilt Lock Mechanism Single Hand/Foot, Slave Right	1	\$96.72
1-4, 9, 18, 19a	506260	Tilt Lock Mechanism Single Hand/Foot, Slave Left	1	\$96.72
1-4, 9, 18, 19b, 23	506261	Tilt Lock Mechanism Single Hand/Foot, Heavy Duty, Slave Right	1	\$151.93
1-4, 9, 18, 19b, 23	506262	Tilt Lock Mechanism Single Hand/Foot, Heavy Duty, Slave Left	1	\$151.93
5	004284	Cam Lever, Drive, Tilt Lock	1	\$31.72
6	004285	Cam Lever, Slave, Tilt Lock	1	\$31.72
7	004286	Mount, Linkage, Dual	1	\$45.32
8	004289	Bracket, Mount, Tilt Lock	1	\$15.24
9	004290	Spring, Comp, .42OD x 1.0L x .038W	2	\$1.03
10	004291	Bushing, Pivot, Tilt Lock	2	\$6.70
11	004294	Tube Assembly, Slave, Tilt Lock	1	\$12.67
12a	004295	Rod, Slave, Small <i>Used with Small Width Range</i>	1	\$7.83
12b	004296	Rod, Slave, Medium <i>Used with Medium Width Range</i>	1	\$8.03
12c	004297	Rod, Slave, Large <i>Used with Large Width Range</i>	1	\$8.60
13	004298	Cover, Tilt Lock	1	\$12.77
14	004308	Shaft Clamp, 6mm x M5	1	\$16.48
15	100658	M6 Nylock Nut Blk Zn	2	\$1.03
16	100746	M6 Flat Washer Blk Zn	2	\$1.55
17	101824	M6 x 30 HHCS Blk Zn	2	\$2.06
18	102800	M4 x 8mm SHCS Blk Zn	6	\$0.52
19a	103510	M6 x 1 x 20 HHCS SFLG CLS8.8 Blk Zn DIN6921 <i>Used with Standard Weight Capacity</i>	4	\$1.03
19b	103511	M6 x 1 x 45 HHCS SFLG CLS8.8 Blk Zn DIN6921 <i>Used with Heavy Duty Option</i>	4	\$3.09
20	103514	M5 x 0.8 Jam Nut BLK ZN DIN439B	1	\$0.52
21	103522	M5 x 0.8 x 12 FHCS CLS 10.9 Nyl Blk Zn DIN7991	1	\$1.03
22	200427	M5 x 10 CLS8.8 HHCS ZC w/Patch DIN933	6	\$0.52

Item Number	Part Number	Description	Quantity	Retail
23	001405	Spacer, .69OD x .24ID x .1.01Lg <i>Used with Heavy Duty Option</i>	4	\$25.54
24a	004301	Cable, Tilt, Medium <i>Used with Short Stroller Handle Back Post</i>	1	\$104.03
24b	004302	Cable, Tilt, Long <i>Used with Tall Stroller Handle Back Post</i>	1	\$106.09
25	001117	Cable Clip 1"	2	\$3.75
26	001239	P-Clip 3/16"	1	\$6.18
27	003285	Lever Assembly, Angle 135°	1	\$24.46
28	100734	M5 x 16 PHMS ZC	1	\$1.55