


Focus CR Seat Pan

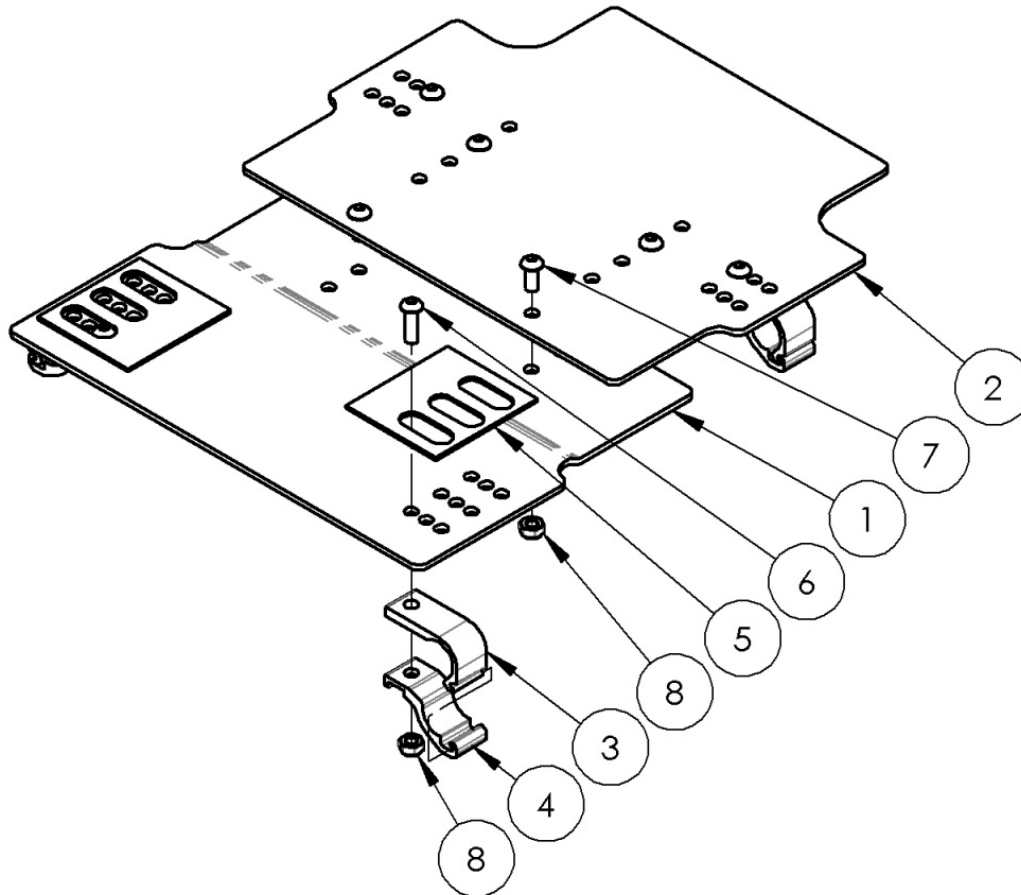

Grows two inches in width from even widths and one inch from odd widths. Depth growth is matched to frame depth range and can be adjusted to any depth in the range.

Item Number	Part Number	Description	Quantity	Retail
1a, 2a, 3 - 8	501144	Seat Pan Assm Short 14-16" Wide Focus	1	\$402.02
1b, 2b, 3 - 8	501145	Seat Pan Assm Short 16-18" Wide Focus	1	\$402.02
1c, 2c, 3 - 8	501146	Seat Pan Assm Short 18-20" Wide Focus	1	\$402.02
1d, 2d, 3 - 8	501147	Seat Pan Assm Short 20-22" Wide Focus	1	\$402.02
1e, 2e, 3 - 8	501218	Seat Pan Assm Short 22" Wide Focus	1	\$679.08
1f, 2a, 3 - 8	501148	Seat Pan Assm Medium 14-16" Wide Focus	1	\$402.02

Item Number	Part Number	Description	Quantity	Retail
1g, 2b, 3 - 8	501149	Seat Pan Assm Medium 16-18" Wide Focus	1	\$402.02
1h, 2c, 3 - 8	501150	Seat Pan Assm Medium 18-20" Wide Focus	1	\$402.02
1i, 2d, 3 - 8	501151	Seat Pan Assm Medium 20-22" Wide Focus	1	\$402.02
1j, 2e, 3 - 8	501219	Seat Pan Assm Medium 22" Wide Focus	1	\$679.08
1k, 2a, 3 - 8	501152	Seat Pan Assm Long 14-16" Wide Focus	1	\$402.02
1l, 2b, 3 - 8	501153	Seat Pan Assm Long 16-18" Wide Focus	1	\$402.02
1m, 2c, 3 - 8	501154	Seat Pan Assm Long 18-20" Wide Focus	1	\$402.02
1n, 2d, 3 - 8	501155	Seat Pan Assm Long 20-22" Wide Focus	1	\$402.02
1o, 2e, 3 - 8	501220	Seat Pan Assm Long 22" Wide Focus	1	\$679.08
1a	001184	Seat Pan Front Short 14-16" Wide TILT	1	\$191.36
1b	001185	Seat Pan Front Short 16-18" Wide TILT	1	\$191.36
1c	001186	Seat Pan Front Short 18-20" Wide TILT	1	\$191.36
1d	001187	Seat Pan Front Short 20-22" Wide TILT	1	\$191.36
1e	001188	Seat Pan Front Short 22" Wide TILT	1	\$191.36
1f	001189	Seat Pan Front Medium 14-16" Wide TILT	1	\$191.36
1g	001190	Seat Pan Front Medium 16-18" Wide TILT	1	\$191.36
1h	001191	Seat Pan Front Medium 18-20" Wide TILT	1	\$191.36
1i	001192	Seat Pan Front Medium 20-22" Wide TILT	1	\$191.36
1j	001193	Seat Pan Front Medium 22" Wide TILT	1	\$191.36
1k	001194	Seat Pan Front Long 14-16" Wide TILT	1	\$191.36
1l	001195	Seat Pan Front Long 16-18" Wide TILT	1	\$191.36
1m	001196	Seat Pan Front Long 18-20" Wide TILT	1	\$191.36
1n	001197	Seat Pan Front Long 20-22" Wide TILT	1	\$191.36
1o	001198	Seat Pan Front Long 22" Wide TILT	1	\$191.36
2a	001199	Seat Pan Rear 14-16" Wide TILT	1	\$191.36
2b	001200	Seat Pan Rear 16-18" Wide TILT	1	\$191.36
2c	001201	Seat Pan Rear 18-20" Wide TILT	1	\$191.36
2d	001202	Seat Pan Rear 20-22" Wide TILT	1	\$191.36

Item Number	Part Number	Description	Quantity	Retail
2e	001203	Seat Pan Rear 22" Wide TILT	1	\$191.36
3	001204	Seat Pan/Base Bracket TILT	4	\$34.18
4	001205	Seat Pan/Base Clamp TILT	4	\$35.29
5	001206	AB Tape 3x3 Die Cut Tilt	2	\$2.47
6	101814	M8 x 25 BHCS Blk Zn	4	\$1.03
7	101813	M8 x 18 BHCS Blk Zn	4	\$2.06
8	100659	M8 Nylock Nut Blk Zn	8	\$1.03