


Catalyst 4 Height Adjustable Flip Back T-Arm

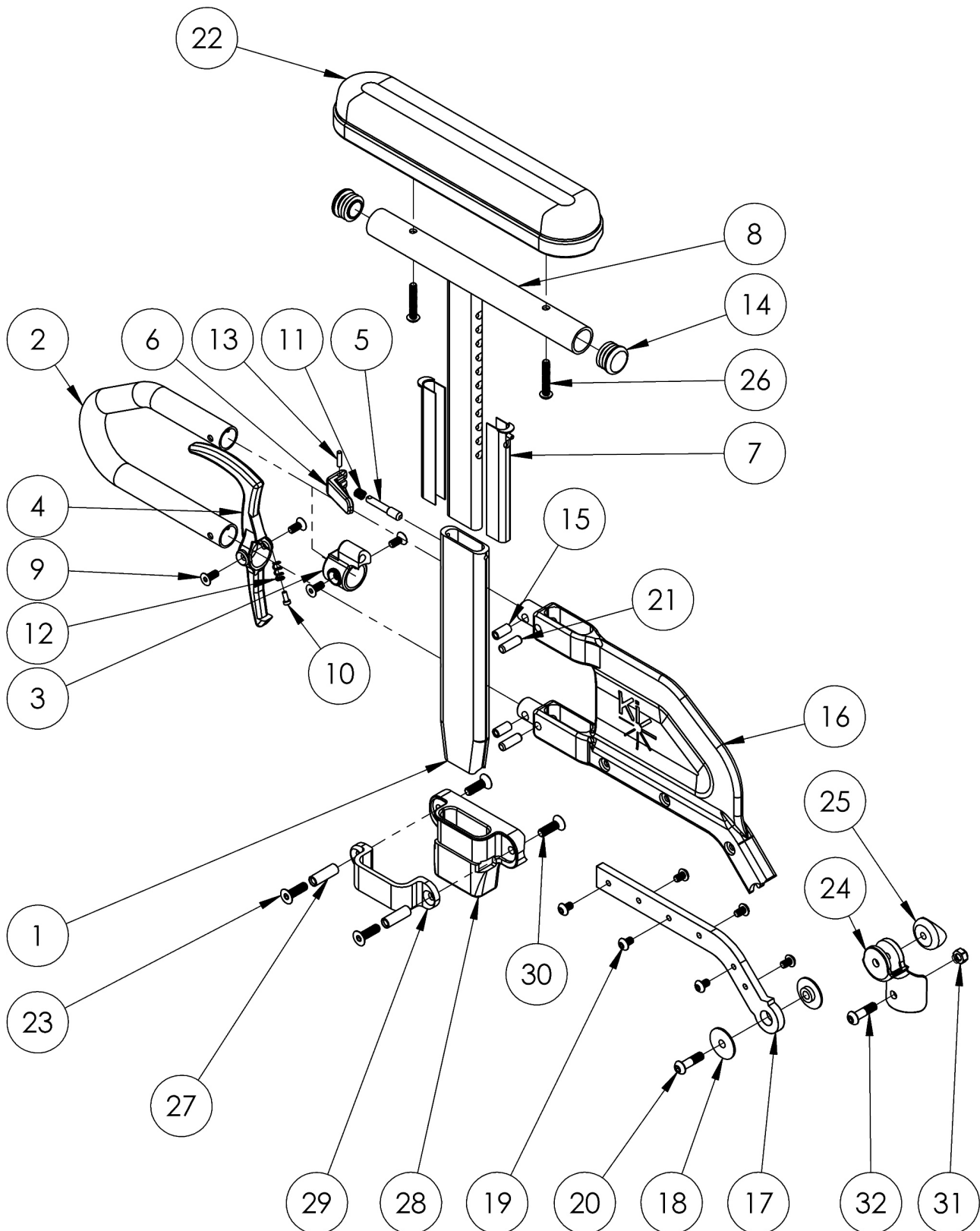

Short Side Shield used with 14"-17" Seat Depths. Long Side Shield used with 18"-20" Seat Depths.

Item Number	Part Number	Description	Quantity	Retail
1	003420	T-Arm Post	1	\$22.00
2	000095	Height Adj T Armrest Transfer Tube	1	\$26.72
7	000090	Height Adj T Armrest Shim	2	\$2.43
8a, 14	000091	Height Adj T Armrest Upper Desk	1	\$71.43
8b, 14	000101	Height Adj T Armrest Upper Full	1	\$71.43
14	100803	Tube End Plug 7/8"	2	\$1.50
18, 20	504566	Height Adjustable Flip Back T-Arm Hinge Bolt / Bushing Kit	1	\$3.00
22a	100500	Arm Pad Full	1	\$22.50
22b	100501	Arm Pad Desk	1	\$22.50
23	100866	M6 x 20 CLS12.9 DIN7991 FSHCS BLK ZC w/Patch	2	\$2.50
24	003425	Height Adjustable Flip Back T-Arm Hinge, Catalyst 4	1	\$15.00
25	003426	Height Adjustable Flip Back T-Arm Hinge Spacer, Catalyst 4	1	\$2.00
18, 20, 24, 25	003730	Armrest Flip Back Height Adj T-Arm Hinge Assembly Catalyst 4	1	\$20.00
26	100696	M5 x 30mm PPH Capscrew	2	\$1.50
27	000065	M6 x 23.68 Threaded Barrel	2	\$3.93
28	000087	Height Adjustable T-Arm Receiver 1" CAT	1	\$14.51
29	000088	Height Adj T Armrest Bracket	1	\$26.10
30	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2	\$1.50
31, 32	504577	Height Adjustable Flip Back T-Arm Mounting Hardware Assembly <i>This kit contains various mounting hardware for multiple configurations</i>	1	\$13.00
3, 5, 6, 9, 11, 13	504560	Height Adjustable Flip Back T-Arm Latch Assembly	1	\$43.26
4, 9, 10, 12	504561	Height Adjustable Flip Back T-Arm Release Lever Assembly	1	\$11.08
15, 16a, 21	504562	Height Adjustable Flip Back T-Arm Side Shield Short Assembly	1	\$28.00
15, 16b, 21	504563	Height Adjustable Flip Back T-Arm Side Shield Long Assembly	1	\$28.00
17a, 19	504564	Height Adjustable Flip Back T-Arm Metal Insert Short Assembly	1	\$32.00
17b, 19	504565	Height Adjustable Flip Back T-Arm Metal Insert Long Assembly	1	\$34.00

Item Number	Part Number	Description	Quantity	Retail
23, 27-30	501730	Height Adj T-Arm Receiver Assm 1" CAT	1	\$52.98
1-7, 8a, 9-15, 16a, 17a, 18-21	003459	Height Adjustable Flip Back T-Arm Short Desk	1	\$125.00
1-7, 8b, 9-15, 16a, 17a, 18-21	003460	Height Adjustable Flip Back T-Arm Short Full	1	\$125.00
1-7, 8a, 9-15, 16b, 17b, 18-21	003461	Height Adjustable Flip Back T-Arm Long Desk	1	\$125.00
1-7, 8b, 9-15, 16b, 17b, 18-21	003462	Height Adjustable Flip Back T-Arm Long Full	1	\$125.00
1-7, 9-13, 15, 16a, 17a, 18-21, 24, 25	505617	Height Adjustable Flip Back T-Arm Short Retrofit Assembly, Catalyst 4	1	\$183.92
1-7, 9-13, 15, 16b, 17b, 18-21, 24, 25	505618	Height Adjustable Flip Back T-Arm Long Retrofit Assembly, Catalyst 4	1	\$185.92
1-7, 8a, 9-15, 16a, 17a, 18-21, 22b, 24-26, 31, 32	505266	Height Adjustable Flip Back T-Arm Short Desk with Standard Pad Assembly, Catalyst 4	1	\$167.50
1-7, 8b, 9-15, 16a, 17a, 18-21, 22a, 24-26, 31, 32	505267	Height Adjustable Flip Back T-Arm Short Full with Standard Pad Assembly, Catalyst 4	1	\$167.50
1-7, 8a, 9-15, 16b, 17b, 18-21, 22b, 24-26, 31, 32	505268	Height Adjustable Flip Back T-Arm Long Desk with Standard Pad Assembly, Catalyst 4	1	\$167.50
1-7, 8b, 9-15, 16b, 17b, 18-21, 22a, 24-26, 31, 32	505269	Height Adjustable Flip Back T-Arm Long Full with Standard Pad Assembly, Catalyst 4	1	\$167.50
1-7, 8a, 9-15, 16a, 17a, 18-21, 22b, 23-32	505270	Height Adjustable Flip Back T-Arm Short Desk with Standard Pad and Receiver Assembly, Catalyst 4	1	\$223.97
1-7, 8b, 9-15, 16a, 17a, 18-21, 22a, 23-32	505271	Height Adjustable Flip Back T-Arm Short Full with Standard Pad and Receiver Assembly, Catalyst 4	1	\$223.97
1-7, 8a, 9-15, 16b, 17b, 18-21, 22b, 23-32	505272	Height Adjustable Flip Back T-Arm Long Desk with Standard Pad and Receiver Assembly, Catalyst 4	1	\$223.97
1-7, 8b, 9-15, 16b, 17b, 18-21, 22a, 23-32	505273	Height Adjustable Flip Back T-Arm Long Full with Standard Pad and Receiver Assembly, Catalyst 4	1	\$223.97