


(Discontinued) Catalyst 5Vx Rear Frame

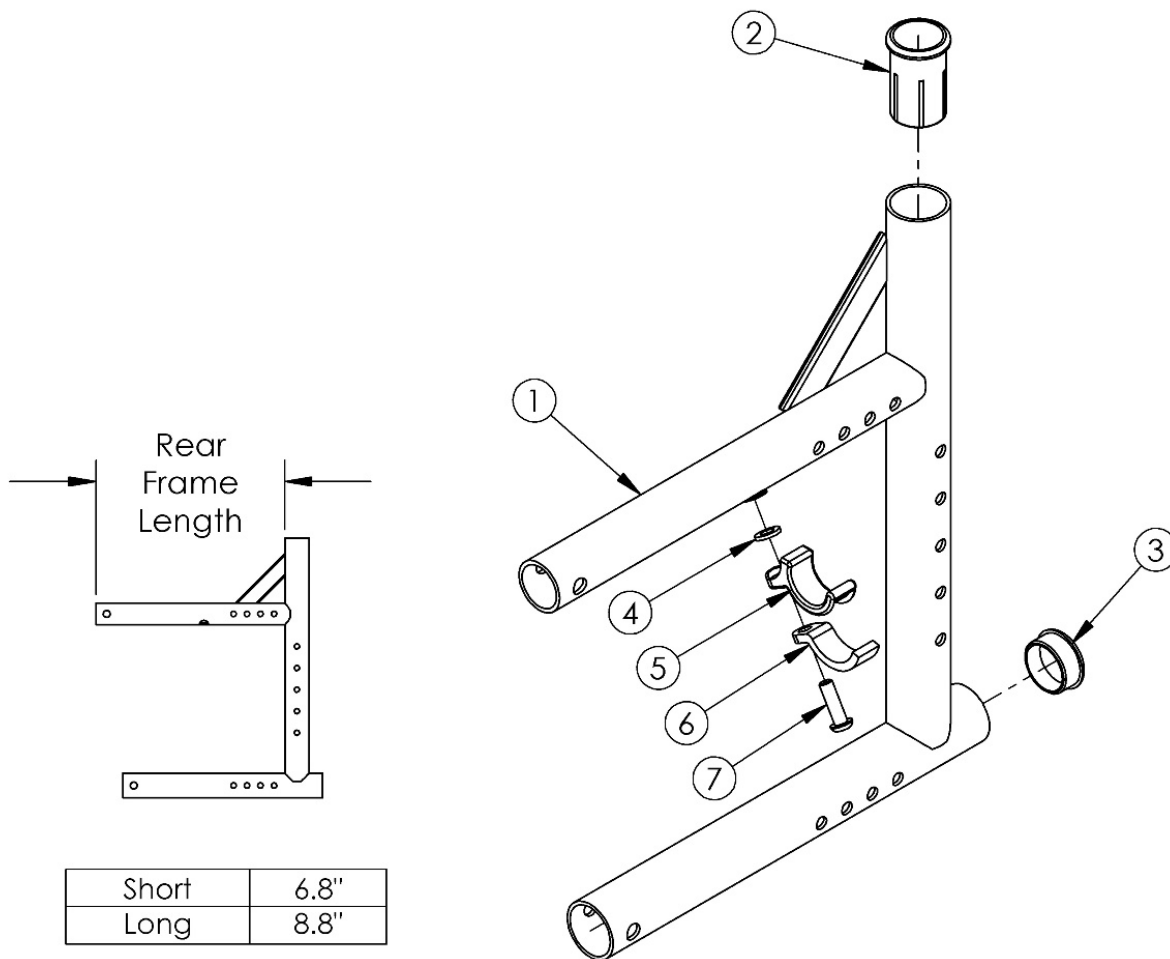

Use the short frame for 14-17" Seat Depths. Use the long frame for 18-20" Seat Depths.<p>

This rear frame was used on serial numbers prior to C5053765 and C5HD003546.<p>

If switching to a new Seat Depth, new Height Adjustable Flip Back T-Arms may be needed.

Item Number	Part Number	Description	Quantity	Retail
4, 5, 6, 7	501005	Seat Hook and Support Hardware Kit, Catalyst <i>This kit contains quantity of 1 each</i>	1	\$17.06
1a, 2, 3	100267_ _ _	Frame Assm Cat 5VX Rear Left Short	1	\$217.06 - \$332.11
1b, 2, 3	100269_ _ _	Frame Assm Cat 5VX Rear Left Long	1	\$217.06 - \$332.11
1c, 2, 3	100266_ _ _	Frame Assm Cat 5VX Rear Right Short	1	\$217.06 - \$332.11
1d, 2, 3	100268_ _ _	Frame Assm Cat 5VX Rear Right Long	1	\$217.06 - \$332.11

Item Number	Part Number	Description	Quantity	Retail
2	000030	Backrest Sleeve Cat	1	\$3.45
3	100532	Tube End Plug 1 1/8"	1	\$2.58
4	100831	1/2" Washer OD .040" Thick	1	\$1.55
5	000003	Seat Hook CAT	1	\$7.55
6	000004	Seat Hook Support CAT	1	\$6.42
7	100668	M6 x 20 CLS10.9 BTN SHCS ZC w/Patch	1	\$1.55

Matte Finish	
Code	Color Name
CGM	Charcoal Gray
EGM	Emerald Green
GGM	Graphite Gray*
IBM	Iridescent Burgundy
INM	Iridescent Navy
AGM	Military Green**
RRM	Retro Red
RBM	Royal Blue
SBM	Shadow Black
SWM	Smooth White

* Discontinued Color

** Formerly called Army Green

Gloss Finish	
Code	Color Name
BLK	Black
BLU	Blue
SKY	Blue Skies
BRO	Bright Orange*
PNK	Bubblegum Pink
BUR	Burgundy*
CBL	Candy Blue
PUR	Candy Purple
RED	Candy Red
COP	Copper
DPR	Deep Purple*
EBL	Electric Blue*
GRN	Forest Green*
FUS	Fuchsia*
GSM	Granny Smith
MTF	Metallic Fuchsia
MOR	Metallic Orange
ORG	Orange*
SAP	Sapphire*
SIL	Silver
TEA	Teal
YLW	Yellow
FLM	Graphic Image Transfer