


# Discontinued Catalyst Height Adjustable T-Arm

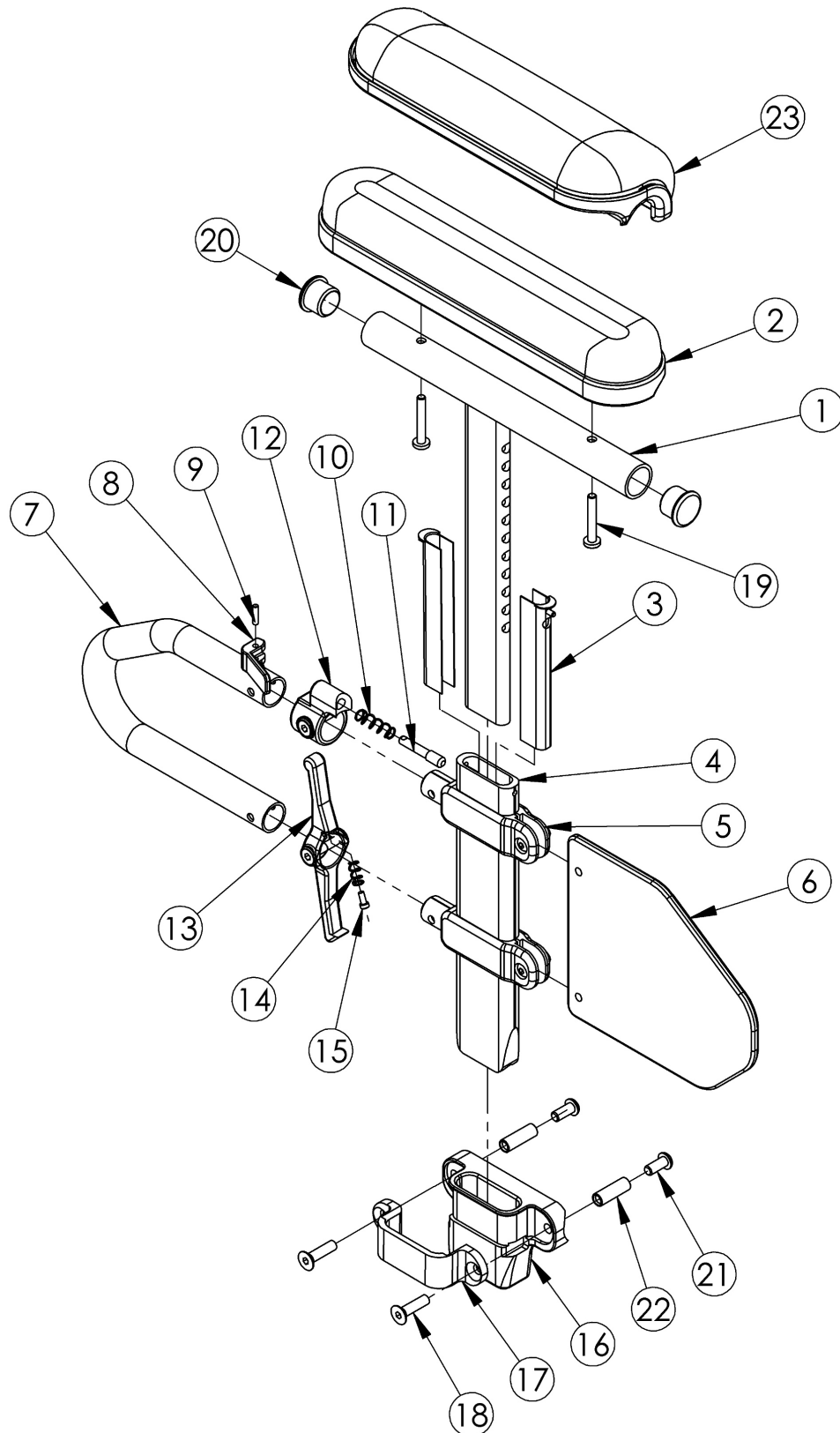

Short assemblies are used with 14-17 Seat Depths. Long assemblies are used with 18"-20" Seat Depths.

Item Number	Part Number	Description	Quantity	Retail
1a, 2b, 3-5, 6a, 7-15, 19a, 20	501682	Height Adj T-Arm Assm Short Desk w/ Std Pad Left <i>Replaced with 505646</i>	1	\$210.52
1a, 2b, 3-5, 6a, 7-15, 19a, 20	501683	Height Adj T-Arm Assm Short Desk w/ Std Pad Right <i>Replaced with 505646</i>	1	\$210.52
1a, 2b, 3-5, 6a, 7-15, 19a, 20	503051	Height Adj T Armrest, 14"-17" Depths, Desk w/ Std Pad PAIR CAT <i>Replaced with 505647</i>	1	\$457.93
1b, 2a, 3-5, 6a, 7-15, 19a, 20	501684	Height Adj T-Arm Assm Short Full w/ Std Pad Left <i>Replaced with 505644</i>	1	\$210.52
1b, 2a, 3-5, 6a, 7-15, 19a, 20	501685	Height Adj T-Arm Assm Short Full w/ Std Pad Right <i>Replaced with 505644</i>	1	\$210.52
1b, 2a, 3-5, 6a, 7-15, 19a, 20	503053	Height Adj T Armrest, 14"-17" Depths, Full w/ Std Pad PAIR CAT <i>Replaced with 505645</i>	1	\$457.93
1a, 2b, 3-5, 6b, 7-15, 19a, 20	501686	Height Adj T-Arm Assm Long Desk w/ Std Pad Left <i>Replaced with 505493</i>	1	\$210.52
1a, 2b, 3-5, 6b, 7-15, 19a, 20	501687	Height Adj T-Arm Assm Long Desk w/ Std Pad Right <i>Replaced with 505493</i>	1	\$210.52
1a, 2b, 3-5, 6b, 7-15, 19a, 20	503052	Height Adj T Armrest, 18"-20" Depths, Desk w/ Std Pad PAIR CAT <i>Replaced with 505494</i>	1	\$457.93
1b, 2a, 3-5, 6b, 7-15, 19a, 20	501688	Height Adj T-Arm Assm Long Full w/ Std Pad Left <i>Replaced with 505491</i>	1	\$210.52
1b, 2a, 3-5, 6b, 7-15, 19a, 20	501689	Height Adj T-Arm Assm Long Full w/ Std Pad Right <i>Replaced with 505491</i>	1	\$210.52
1b, 2a, 3-5, 6b, 7-15, 19a, 20	503054	Height Adj T Armrest, 18"-20" Depths, Full w/ Std Pad PAIR CAT <i>Replaced with 505492</i>	1	\$457.93
1a, 2d, 3-5, 6a, 7-15, 19b, 20	501690	Height Adj T-Arm Assm Short Desk w/ Waterfall Pad Left <i>Replaced with 505650</i>	1	\$219.96
1a, 2d, 3-5, 6a, 7-15, 19b, 20	501691	Height Adj T-Arm Assm Short Desk w/ Waterfall Pad Right <i>Replaced with 505650</i>	1	\$219.96
1a, 2d, 3-5, 6a, 7-15, 19b, 20	503059	Height Adj T Armrest, 14"-17" Depths, Desk w/ Waterfall Pad PAIR CAT <i>Replaced with 505651</i>	1	\$476.45
1b, 2c, 3-5, 6a, 7-15, 19b, 20	501692	Height Adj T-Arm Assm Short Full w/ Waterfall Pad Left <i>Replaced with 505648</i>	1	\$219.96
1b, 2c, 3-5, 6a, 7-15, 19b, 20	501693	Height Adj T-Arm Assm Short Full w/ Waterfall Pad Right <i>Replaced with 505648</i>	1	\$219.96

Item Number	Part Number	Description	Quantity	Retail
1b, 2c, 3-5, 6a, 7-15, 19b, 20	503061	Height Adj T Armrest, 14"-17" Depths, Full w/ Waterfall Pad PAIR CAT <i>Replaced with 505649</i>	1	\$476.45
1a, 2d, 3-5, 6b, 7-15, 19b, 20	501694	Height Adj T-Arm Assm Long Desk w/ Waterfall Pad Left <i>Replaced with 505497</i>	1	\$219.96
1a, 2d, 3-5, 6b, 7-15, 19b, 20	501695	Height Adj T-Arm Assm Long Desk w/ Waterfall Pad Right <i>Replaced with 505497</i>	1	\$219.96
1a, 2d, 3-5, 6b, 7-15, 19b, 20	503060	Height Adj T Armrest, 18"-20" Depths, Desk w/ Waterfall Pad PAIR CAT <i>Replaced with 505498</i>	1	\$476.45
1b, 2c, 3-5, 6b, 7-15, 19b, 20	501696	Height Adj T-Arm Assm Long Full w/ Waterfall Pad Left <i>Replaced with 505495</i>	1	\$219.96
1b, 2c, 3-5, 6b, 7-15, 19b, 20	501697	Height Adj T-Arm Assm Long Full w/ Waterfall Pad Right <i>Replaced with 505495</i>	1	\$219.96
1b, 2c, 3-5, 6b, 7-15, 19b, 20	503062	Height Adj T Armrest, 18"-20" Depths, Full w/ Waterfall Pad PAIR CAT <i>Replaced with 505496</i>	1	\$476.45
1a, 2b, 3-5, 6a, 7-18, 19a, 20-22	501698	Height Adj T-Arm Assm Short Desk w/ Std Pad and 1" Receiver Left CAT <i>Replaced with 505654</i>	1	\$265.09
1a, 2b, 3-5, 6a, 7-18, 19a, 20-22	501699	Height Adj T-Arm Assm Short Desk w/ Std Pad and 1" Receiver Right CAT <i>Replaced with 505654</i>	1	\$265.09
1a, 2b, 3-5, 6a, 7-18, 19a, 20-22	503018	Height Adj T Armrest w/ Brackets and Hardware, 14"-17" Depths, Desk w/ Std Pad PAIR CAT <i>Replaced with 505655</i>	1	\$583.69
1b, 2a, 3-5, 6a, 7-18, 19a, 20-22	501700	Height Adj T-Arm Assm Short Full w/ Std Pad and 1" Receiver Left CAT <i>Replaced with 505652</i>	1	\$265.09
1b, 2a, 3-5, 6a, 7-18, 19a, 20-22	501701	Height Adj T-Arm Assm Short Full w/ Std Pad and 1" Receiver Right CAT <i>Replaced with 505652</i>	1	\$265.09
1b, 2a, 3-5, 6a, 7-18, 19a, 20-22	503020	Height Adj T Armrest w/ Brackets and Hardware, 14"-17" Depths, Full w/ Std Pad PAIR CAT <i>Replaced with 505653</i>	1	\$583.69
1a, 2b, 3-5, 6b, 7-18, 19a, 20-22	501702	Height Adj T-Arm Assm Long Desk w/ Std Pad and 1" Receiver Left CAT <i>Replaced with 505501</i>	1	\$265.09
1a, 2b, 3-5, 6b, 7-18, 19a, 20-22	501703	Height Adj T-Arm Assm Long Desk w/ Std Pad and 1" Receiver Right CAT <i>Replaced with 505501</i>	1	\$265.09
1a, 2b, 3-5, 6b, 7-18, 19a, 20-22	503019	Height Adj T Armrest w/ Brackets and Hardware, 18"-20" Depths, Desk w/ Std Pad PAIR CAT <i>Replaced with 505502</i>	1	\$583.69

Item Number	Part Number	Description	Quantity	Retail
1b, 2a, 3-5, 6b, 7-18, 19a, 20- 22	501704	Height Adj T-Arm Assm Long Full w/ Std Pad and 1" Receiver Left CAT <i>Replaced with 505499</i>	1	\$265.09
1b, 2a, 3-5, 6b, 7-18, 19a, 20- 22	501705	Height Adj T-Arm Assm Long Full w/ Std Pad and 1" Receiver Right CAT <i>Replaced with 505499</i>	1	\$265.09
1b, 2a, 3-5, 6b, 7-18, 19a, 20- 22	503021	Height Adj T Armrest w/ Brackets and Hardware, 18"-20" Depths, Full w/ Std Pad PAIR CAT <i>Replaced with 505500</i>	1	\$583.69
1a, 2d, 3-5, 6a, 7-18, 19b, 20- 22	501706	Height Adj T-Arm Assm Short Desk w/ Waterfall Pad and 1" Receiver Left CAT <i>Replaced with 505658</i>	1	\$274.53
1a, 2d, 3-5, 6a, 7-18, 19b, 20- 22	501707	Height Adj T-Arm Assm Short Desk w/ Waterfall Pad and 1" Receiver Right CAT <i>Replaced with 505658</i>	1	\$274.53
1a, 2d, 3-5, 6a, 7-18, 19b, 20- 22	503055	Height Adj T Armrest w/ Brackets and Hardware, 14"-17" Depths, Desk w/ Waterfall Pad PAIR CAT <i>Replaced with 505659</i>	1	\$602.21
1b, 2c, 3-5, 6a, 7-18, 19b, 20- 22	501708	Height Adj T-Arm Assm Short Full w/ Waterfall Pad and 1" Receiver Left CAT <i>Replaced with 505656</i>	1	\$274.53
1b, 2c, 3-5, 6a, 7-18, 19b, 20- 22	501709	Height Adj T-Arm Assm Short Full w/ Waterfall Pad and 1" Receiver Right CAT <i>Replaced with 505656</i>	1	\$274.53
1b, 2c, 3-5, 6a, 7-18, 19b, 20- 22	503057	Height Adj T Armrest w/ Brackets and Hardware, 14"-17" Depths, Full w/ Waterfall Pad PAIR CAT <i>Replaced with 505657</i>	1	\$602.21
1a, 2d, 3-5, 6b, 7-18, 19b, 20- 22	501710	Height Adj T-Arm Assm Long Desk w/ Waterfall Pad and 1" Receiver Left CAT <i>Replaced with 505505</i>	1	\$274.53
1a, 2d, 3-5, 6b, 7-18, 19b, 20- 22	501711	Height Adj T-Arm Assm Long Desk w/ Waterfall Pad and 1" Receiver Right CAT <i>Replaced with 505505</i>	1	\$274.53
1a, 2d, 3-5, 6b, 7-18, 19b, 20- 22	503056	Height Adj T Armrest w/ Brackets and Hardware, 18"-20" Depths, Desk w/ Waterfall Pad PAIR CAT <i>Replaced with 505506</i>	1	\$602.21
1b, 2c, 3-5, 6b, 7-18, 19b, 20- 22	501712	Height Adj T-Arm Assm Long Full w/ Waterfall Pad and 1" Receiver Left CAT <i>Replaced with 505503</i>	1	\$274.53
1b, 2c, 3-5, 6b, 7-18, 19b, 20- 22	501713	Height Adj T-Arm Assm Long Full w/ Waterfall Pad and 1" Receiver Right CAT <i>Replaced with 505503</i>	1	\$274.53
1b, 2c, 3-5, 6b, 7-18, 19b, 20- 22	503058	Height Adj T Armrest w/ Brackets and Hardware, 18"-20" Depths, Full w/ Waterfall Pad PAIR CAT	1	\$602.21

Item Number	Part Number	Description	Quantity	Retail
		<i>Replaced with 505504</i>		
1a, 3-5, 6a, 7-15, 20	100238	Height Adj T Armrest Short Desk Right <i>Replaced with 003463</i>	1	\$203.90
1a, 3-5, 6a, 7-15, 20	100242	Height Adj T Armrest Short Desk Left <i>Replaced with 003463</i>	1	\$203.90
1b, 3-5, 6a, 7-15, 20	100239	Height Adj T Armrest Short Full Right <i>Replaced with 003464</i>	1	\$203.90
1b, 3-5, 6a, 7-15, 20	100243	Height Adj T Armrest Short Full Left <i>Replaced with 003464</i>	1	\$203.90
1a, 3-5, 6b, 7-15, 20	100240	Height Adj T Armrest Long Desk Right <i>Replaced with 003465</i>	1	\$203.90
1a, 3-5, 6b, 7-15, 20	100244	Height Adj T Armrest Long Desk Left <i>Replaced with 003465</i>	1	\$203.90
1b, 3-5, 6b, 7-15, 20	100241	Height Adj T Armrest Long Full Right <i>Replaced with 003466</i>	1	\$203.90
1b, 3-5, 6b, 7-15, 20	100245	Height Adj T Armrest Long Full Left <i>Replaced with 003466</i>	1	\$203.90
16, 17, 18, 21, 22	501730	Height Adj T-Arm Receiver Assm 1" CAT	1	\$54.57
1a, 20	000091	Height Adj T Armrest Upper Desk	1	\$73.57
1b, 20	000101	Height Adj T Armrest Upper Full	1	\$73.57
2a	100500	Arm Pad Full	1	\$23.18
2b	100501	Arm Pad Desk	1	\$23.18
3	000090	Height Adj T Armrest Shim	2	\$2.50
4	000089	Height Adj T Armrest Post	1	\$24.61
5	000092	Height Adj T Armrest Clamp	2	\$8.79
6a	000097	Height Adj T Armrest Side Guard Short <i>Used with 14-17 Seat Depths</i>	1	\$14.06
6b	000120	Height Adj T Armrest Side Guard Long <i>Used with 18-20 Seat Depths</i>	1	\$36.35
7	000095	Height Adj T Armrest Transfer Tube	1	\$27.52
8	000098	Height Adj Armrest Lever	1	\$3.51
9	200257	Dowel Pin 1/8 x 1/2	1	\$2.58
10	000134	Height Adj Armrest Lever Spring	1	\$1.09

Item Number	Part Number	Description	Quantity	Retail
11	000099	Height Adj Armrest Lever Pin	1	\$18.75
12	000096	Height Adj T Armrest Pin Lock Housing	1	\$14.42
8-12	500541	Arm Height Adj T Upper Lever Assembly	1	\$39.76
13	000100	Height Adj T Armrest Release Lever	1	\$9.14
14	000135	Height Adj T Armrest Release Lever Spring	1	\$1.07
15	000152	Height Adj T Armrest Release Spring Button	1	\$1.07
16	000087	Height Adjustable T-Arm Receiver 1" CAT	1	\$14.95
17	000088	Height Adj T Armrest Bracket	1	\$26.88
18	100866	M6 x 20 CLS12.9 DIN7991 FSHCS Blk Zc w/Patch	2	\$2.58
19a	100696	M5 x 30mm PPH Capscrew <i>Used with Standard Arm Pads</i>	2	\$1.55
19b	100654	M5 x 40 DIN7985 PH Pan HD M/S ZC <i>Used with Waterfall Arm Pads</i>	2	\$1.55
20	100803	Tube End Plug 7/8"	2	\$1.55
21	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2	\$1.55
22	000065	M6 x 23.68 Threaded Barrel	2	\$4.05
23a	102452	Waterfall Arm Pad Full	1	\$32.16
23b	102521	Waterfall Arm Pad Desk	1	\$32.16