



Ethos Luggage Carrier

Frame Mount

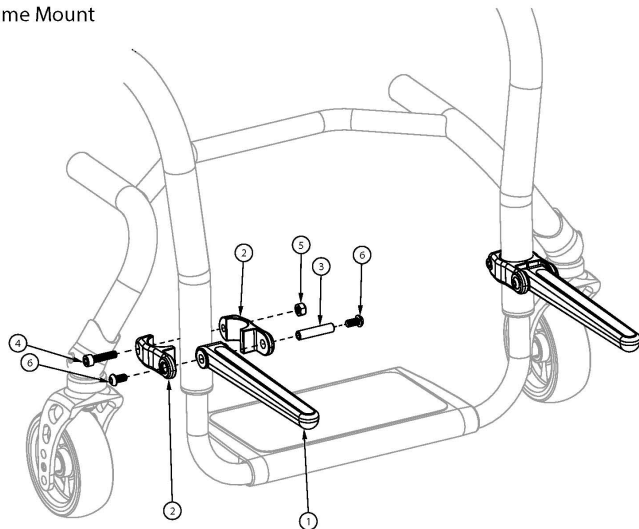

Extension Mount

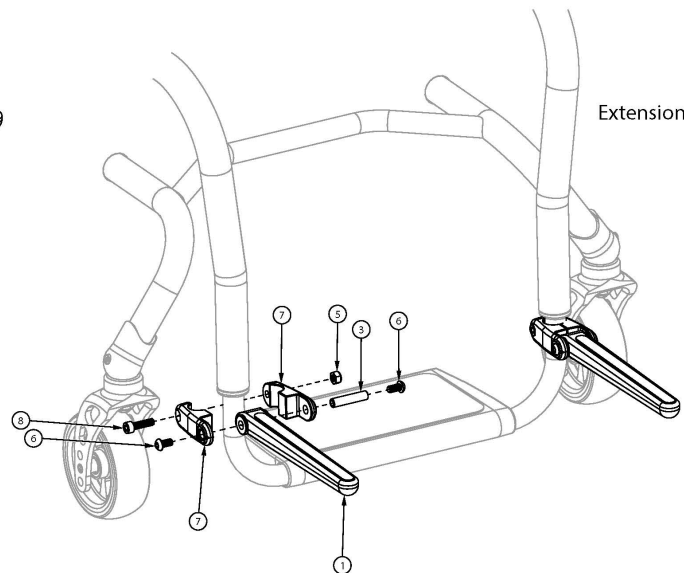

Item Number	Part Number	Description	Quantity	Retail
1 - 8	504466	Luggage Carrier Kit, Klik	1	\$85.00
1	003430	Luggage Carrier Fork Assembly	2	\$28.84
2	003411	Luggage Carrier Clamp 1.125 <i>Used to mount Luggage Carrier to the frame</i>	4	\$8.24
3	003413	Luggage Carrier Barrel	2	\$8.24
4	100780	M6 x 25 SHCS ZC <i>Only used when Luggage Carrier is mounted to the frame</i>	2	\$2.06
5	100658	M6 Nylock Nut Blk Zn	2	\$1.03
6	100667	M6 x 12 BTN SHCS BLKZC w/Patch	4	\$1.55
7	003409	Luggage Carrier Clamp .75 <i>Used to mount Luggage Carrier to the footrest tubes/extensions</i>	4	\$8.24
8	101845	M6 x 22 SHCS Blk Zn <i>Only used when Luggage Carrier is mounted to the footrest tubes/extensions</i>	2	\$1.55