


Discontinued Catalyst 5 Height Adjustable Flip Back Armrest

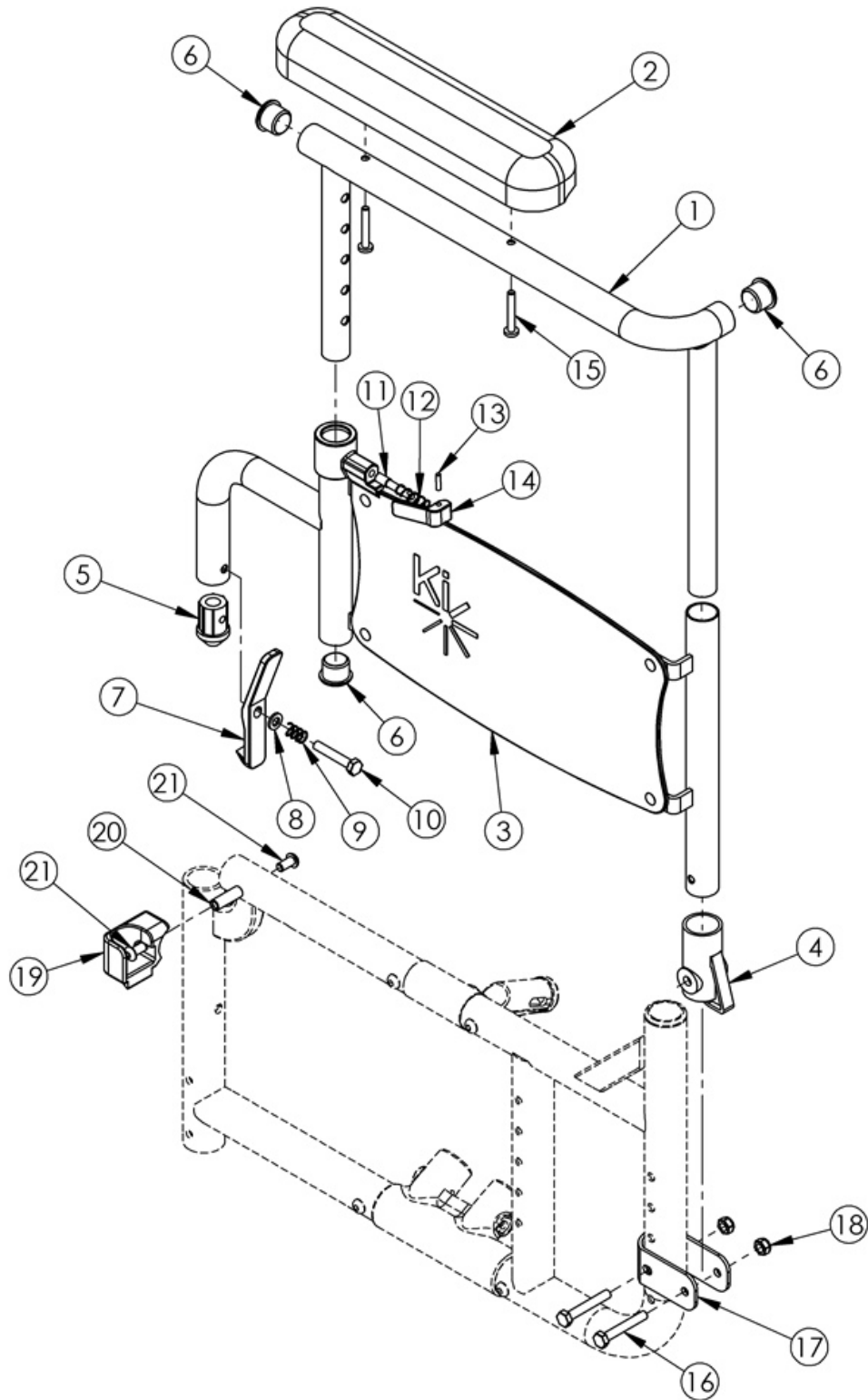

Item Number	Part Number	Description	Quantity	Retail
1a	000382L	Armrest Flip Back Height Adj Frame Upper Desk Left	1	\$93.88
1b	000382R	Armrest Flip Back Height Adj Frame Upper Desk Right	1	\$93.88
1c	000383L	Armrest Flip Back Height Adj Frame Upper Full Left	1	\$110.30
1d	000383R	Armrest Flip Back Height Adj Frame Upper Full Right	1	\$110.30
2a	100500	Arm Pad Full	1	\$22.50
2b	100501	Arm Pad Desk	1	\$22.50
3a, 11 - 14	000431-L	Armrest Flip Back Height Adj Frame Lower Left	1	\$81.14
3b, 11 - 14	000431-R	Armrest Flip Back Height Adj Frame Lower Right	1	\$81.14
4	100691	Armrest Flip Back Rear Pivot Bracket	1	\$3.19
5	100690	Armrest Flip Back Front Tube Plug	1	\$1.70
6	100803	Tube End Plug 7/8"	3	\$1.50
7	100689	Armrest Flip Back Release Lever	1	\$3.72
8	102072	Washer Nylon .234 ID x .406 OD x .062 Thick	1	\$0.50
9	100693	Compression Spring 0.34" OD x 0.70 LG	1	\$1.53
10	100860	M6 x 35 CLS8.8 DIN931 HHCS P/THD ZC	1	\$1.50
11	000099	Height Adj Armrest Lever Pin	1	\$18.20
12	000134	Height Adj Armrest Lever Spring	1	\$1.06
13	200257	Dowel Pin 1/8 x 1/2	1	\$2.50
14	000098	Height Adj Armrest Lever	1	\$3.41
15	100696	M5 x 30mm PPH Capscrew	2	\$1.50
16	100681	M6 x 40 CLS8.8 DIN931 HHCS ZC	2	\$1.50
17	000280	Armrest Flip Back Bracket CAT 5	1	\$7.83
18	100658	M6 Nylock Nut Blk Zn	2	\$1.00
19a	000081	Armrest Flip Back Receiver Left CAT	1	\$9.12
19b	000082	Armrest Flip Back Receiver Right CAT	1	\$9.12
20	000065	M6 x 23.68 Threaded Barrel	1	\$3.93
21	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2	\$1.50

Item Number	Part Number	Description	Quantity	Retail
11-14	500711	Arm Height Adj Flip Back Lever Assembly	1	\$24.60
1b, 2b, 6, 15	503152	Armrest Assembly Height Adj Flip Back Upper Desk Right	1	\$120.54
1a, 2b, 6, 15	503153	Armrest Assembly Height Adj Flip Back Upper Desk Left	1	\$120.54
1d, 2a, 6, 15	503154	Armrest Assembly Height Adj Flip Back Upper Full Right	1	\$136.96
1c, 2a, 6, 15	503155	Armrest Assembly Height Adj Flip Back Upper Full Left	1	\$136.96
1a, 2b, 3a, 4-15	100193	Armrest Assembly Height Adj Flip Back Complete Desk Left	1	\$204.34
1b, 2b, 3b, 4-15	100192	Armrest Assembly Height Adj Flip Back Complete Desk Right	1	\$204.34
1c, 2a, 3a, 4-15	100191	Armrest Assembly Height Adj Flip Back Complete Full Left	1	\$209.86
1d, 2a, 3b, 4-15	100190	Armrest Assembly Height Adj Flip Back Complete Full Right	1	\$209.86
3a, 4 -14	503150	Armrest Assembly Height Adj Flip Back Lower Left	1	\$93.76
3b, 4 -14	503151	Armrest Assembly Height Adj Flip Back Lower Right	1	\$93.76
16 -18	502264	Armrest Height Adjustable Flip Back Bracket Kit CAT5	1	\$12.07
19a, 20, 21	502265	Armrest Height Adjustable Flip Back Receiver Kit Left CAT	1	\$15.34
19b, 20, 21	502266	Armrest Height Adjustable Flip Back Receiver Kit Right CAT	1	\$15.34
1a, 2b, 3a, 4-18, 19a, 20, 21	502271	Armrest Height Adjustable Flip Back w/ Receiver and Bracket Kit Left Desk CAT5	1	\$241.43
1b, 2b, 3b, 4-18, 19b, 20, 21	502272	Armrest Height Adjustable Flip Back w/ Receiver and Bracket Kit Right Desk CAT5	1	\$241.43
1c, 2a, 3a, 4-18, 19a, 20, 21	502273	Armrest Height Adjustable Flip Back w/ Receiver and Bracket Kit Left Full CAT5	1	\$257.85
1d, 2a, 3b, 4-18, 19b, 20, 21	502274	Armrest Height Adjustable Flip Back w/ Receiver and Bracket Kit Right Full CAT5	1	\$257.85
1a, 1b, 2b, 3a, 3b, 4-21	502277	Armrest Height Adjustable Flip Back w/ Receiver and Bracket Kit PAIR Desk CAT5	1	\$482.86
1c, 1d, 2a, 3a, 3b, 4-21	502278	Armrest Height Adjustable Flip Back w/ Receiver and Bracket Kit PAIR Full CAT5	1	\$515.69