


Focus CR Height Adjustable T-Arm

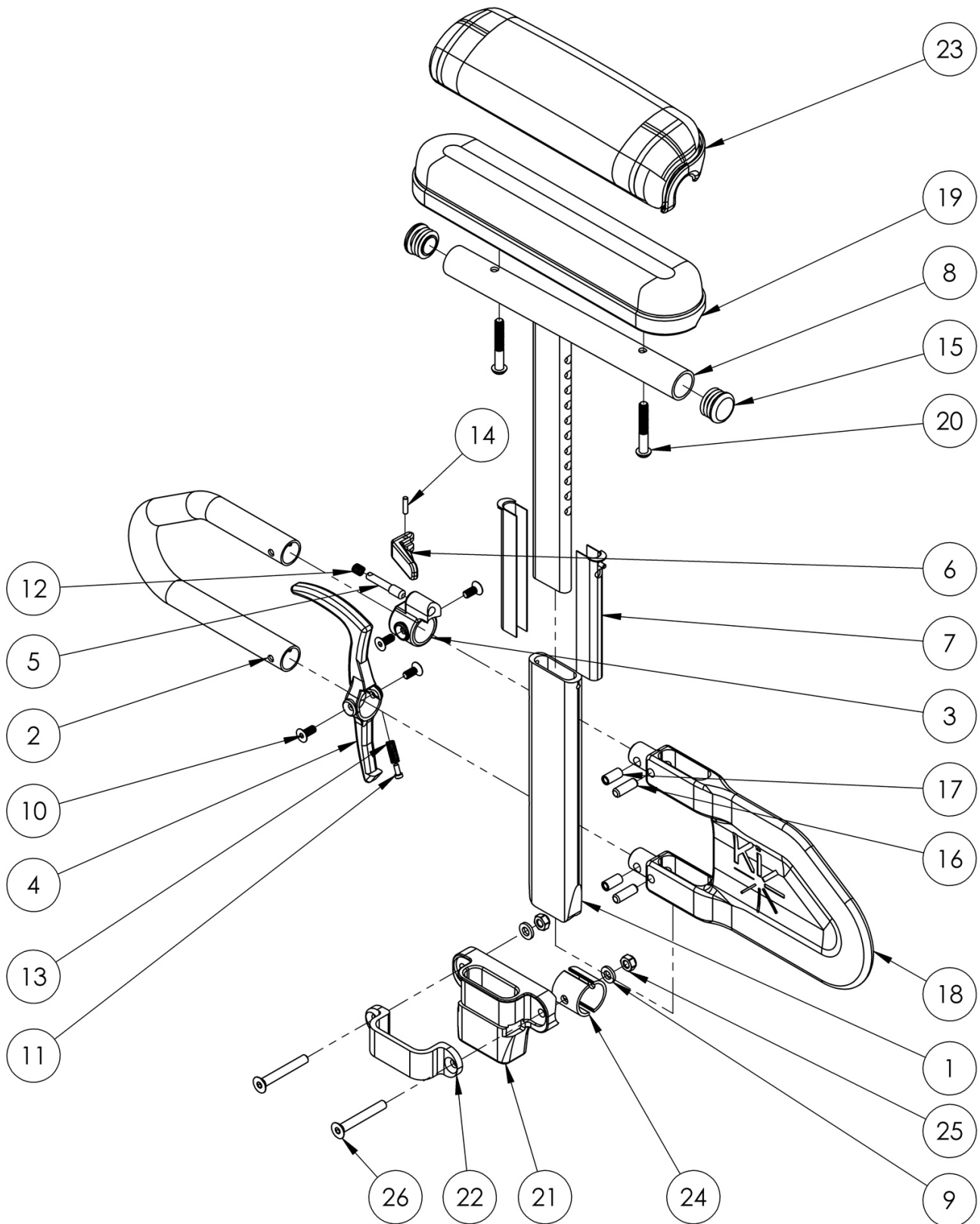

This design effective December 6, 2017 <p>

Not available with Heavy Duty Option

Item Number	Part Number	Description	Quantity	Retail
1-7, 10-14, 16-18	004160	Height Adjustable T-Arm Lower Assembly Short <i>Does not include t-arm upper, arm pad or arm pad screws</i>	1	\$130.33
1-7, 8a, 10-18	003463	Height Adjustable T-Arm Short Desk <i>Does not include arm pad or arm pad screws</i>	1	\$203.90
1-7, 8b, 10-18	003464	Height Adjustable T-Arm Short Full <i>Does not include arm pad or arm pad screws</i>	1	\$203.90
1-7, 8a, 10-18, 19b, 20a	505646	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Desk	1	\$210.52
1-7, 8b, 10-18, 19a, 20a	505644	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Full	1	\$210.52
1-7, 8a, 10-18, 20b, 23b	505650	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Desk	1	\$219.96
1-7, 8b, 10-18, 20b, 23a	505648	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Full	1	\$219.96
1-7, 8a, 10-18, 19b, 20a	505647	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Desk, PAIR	1	\$421.04
1-7, 8b, 10-18, 19a, 20a	505645	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Full, PAIR	1	\$421.04
1-7, 8a, 10-18, 20b, 23b	505651	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Desk, PAIR	1	\$476.45
1-7, 8b, 10-18, 20b, 23a	505649	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Full, PAIR	1	\$476.45
1-7, 8a, 9-18, 19b, 20a, 21, 22, 24-26	506083	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Desk and Receiver, FOCUS	1	\$272.84
1-7, 8b, 9-18, 19a, 20a, 21, 22, 24-26	506084	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Full and Receiver, FOCUS	1	\$272.84
1-7, 8a, 9-18, 20b, 21, 22, 23b, 24-26	506085	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Desk and Receiver, FOCUS	1	\$293.25
1-7, 8b, 9-18, 20b, 21, 22, 23a, 24-26	506086	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Full and Receiver, FOCUS	1	\$293.25
1-7, 8a, 9-18, 19b, 20a, 21, 22, 24-26	506087	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Desk and Receiver PAIR, FOCUS	1	\$545.67
1-7, 8b, 9-18, 19a, 20a, 21, 22, 24-26	506088	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Full and Receiver PAIR, FOCUS	1	\$545.67
1-7, 8a, 9-18, 20b, 21, 22, 23b, 24-26	506089	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Desk and Receiver PAIR, FOCUS	1	\$586.51
1-7, 8b, 9-18, 20b, 21, 22, 23a, 24-26	506090	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Full and Receiver PAIR, FOCUS	1	\$586.51
3, 5, 6, 10, 12, 14	505490	Height Adj T-Arm Upper Latch Assembly	1	\$39.76

Item Number	Part Number	Description	Quantity	Retail
4, 10, 11, 13	505507	Height Adj T-Arm Release Lever Assembly	1	\$8.32
9, 21, 22, 24-26	502298	Armrest T-Arm Receiver Assembly Focus	1	\$54.84
16-18	506082	Height Adj T-Arm Short Shield Assembly	1	\$16.12
1	000089	Height Adj T Armrest Post	1	\$24.61
2	000095	Height Adj T Armrest Transfer Tube	1	\$27.52
7	000090	Height Adj T Armrest Shim	2	\$2.50
8a, 15	000091	Height Adj T Armrest Upper Desk	1	\$73.57
8b, 15	000101	Height Adj T Armrest Upper Full	1	\$73.57
9	100746	M6 Flat Washer Blk Zn	2	\$1.55
10	003458	M5 X 12mm FHCS BLK ZN W/ Patch	4	\$1.03
11	000152	Height Adj T Armrest Release Spring Button	1	\$1.07
13	000135	Height Adj T Armrest Release Lever Spring	1	\$1.07
15	100803	Tube End Plug 7/8"	2	\$1.55
16	003641	.25" x .75" Dowel Pin	2	\$1.03
17	003457	M5 x 14mm Barrel Nut	2	\$1.03
19a	100500	Arm Pad Full	1	\$23.18
19b	100501	Arm Pad Desk	1	\$23.18
20a	100696	M5 x 30mm PPH Capscrew <i>Used with Standard Arm Pads</i>	2	\$1.55
20b	100654	M5 x 40 DIN7985 PH Pan HD M/S ZC <i>Used with Waterfall Arm Pads</i>	2	\$1.55
21	000087	Height Adjustable T-Arm Receiver 1" CAT	1	\$14.95
22	000088	Height Adj T Armrest Bracket	1	\$26.88
23a	102452	Waterfall Arm Pad Full	1	\$32.16
23b	102521	Waterfall Arm Pad Desk	1	\$32.16
24	001230	Tube Spacer 1 x .083	2	\$3.30
25	100658	M6 Nylock Nut Blk Zn	2	\$1.03
26	101592	M6 x 45 CLS10.9 DIN7991 FHCS BLKZC	2	\$2.58