RABBIT

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02.2005

GB RABBIT

Congratulations on choosing the Rabbit from R82. We are convinced that it will fully live up to your expectations.

The Rabbit is a mobile standing frame for disabled children. With its many adjustment possibilities, the Rabbit is able to provide proper support in the standing position, and also give the child the possibility of movement and getting around on its own.

This manual describes and illustrates all functions, and will enable you to take maximum advantage of your Rabbit.

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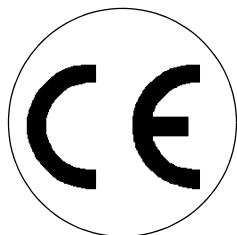
GB SAFETY

The Rabbit has earned the CE-mark. This certifies that it meets all relevant European safety requirements.

NB! Remove the CE-mark, when rebuilding the product or when using other than original R82 spare parts and fittings.

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GB GUARANTEE

The Rabbit is guaranteed for 12 months. The guarantee is only valid when using original R82 spare parts and fittings. All special adjustments must be made by R82.

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GB TOOLS

The enclosed tool-bag contain: 4 & 6 mm Allen key and 10 mm wrench. These tools are used to perform some of the adjustments described in this manual.

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GB MAINTENANCE

The Rabbit functions without any maintenance, but it is recommended to clean the frame frequently.

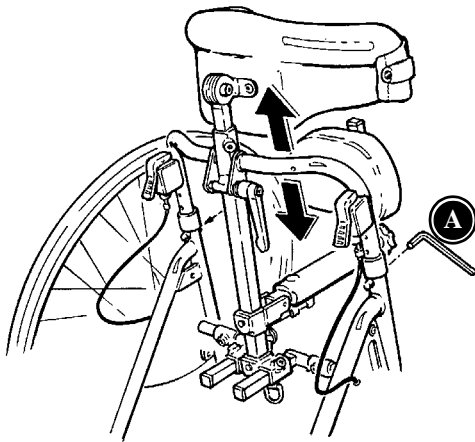
Warning! Do not use any cleaning materials containing chlorine or methylated spirit.

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GB ADJUSTMENT/ANG- LING

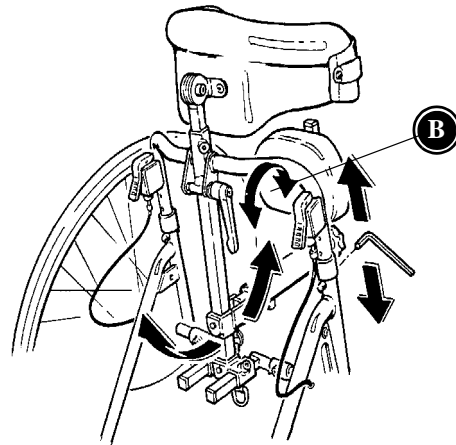
Height adjustment of the frame:
Use the enclosed 4 mm Allen key and the 10 mm wrench to remove the screws in both sides (A). The frame can be adjusted up or down and the screws are put in place in the holes in the frame.

Angling the bar of the body support:
Loosen the handle (B) to angle the bar of the body support together with the back- and knee supports.

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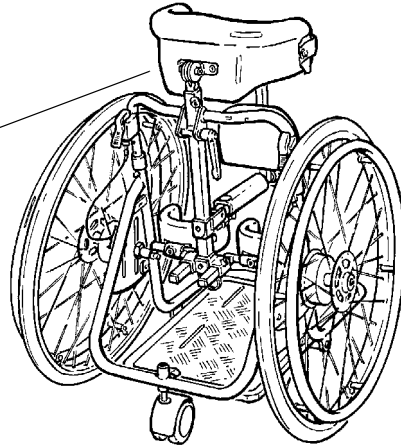
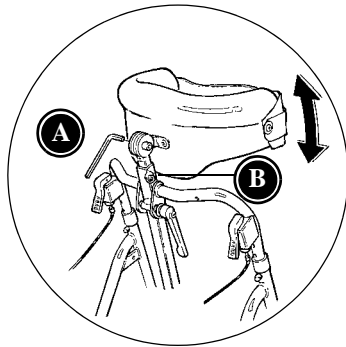
GB ANGLING THE BODY SUPPORT

Use the enclosed 6 mm Allen key (A) to angle the body support.

Use the enclosed 4 mm Allen key (B) to adjust the height of the body support.

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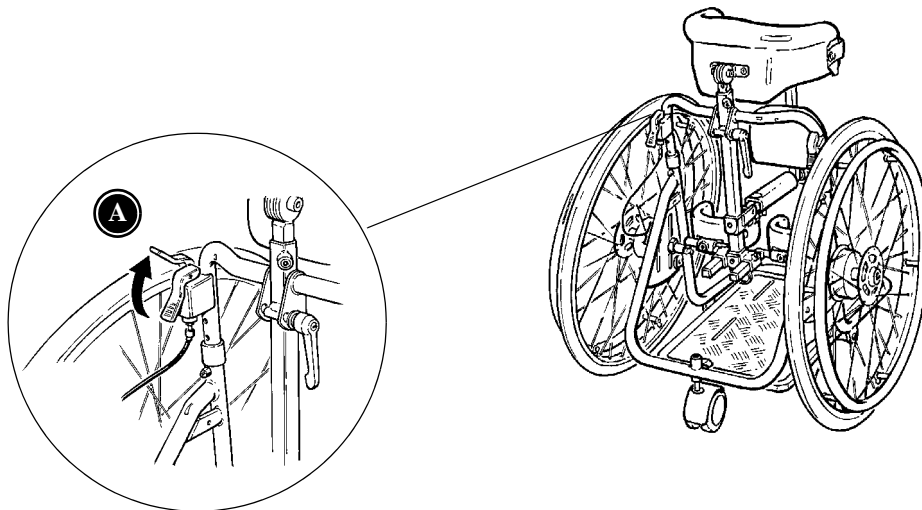
GB BRAKES

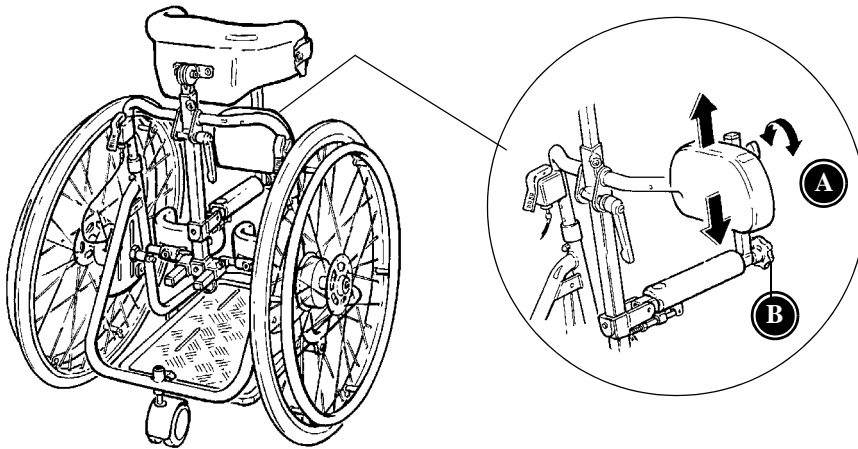
Brakes are mounted on both sides of the Rabbit. Pull up the handle (A) to activate the brakes. The brakes are used during transportation or when the child is standing still - playing, maby with the tray/play box mounted. (See page 12).

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GB BACK SUPPORT

The complete back support unit is mounted on the bar for the body support and adjust in height with the enclosed 4 mm Allen key.

Loosen the handle (A) on the back to adjust the height of the back support.

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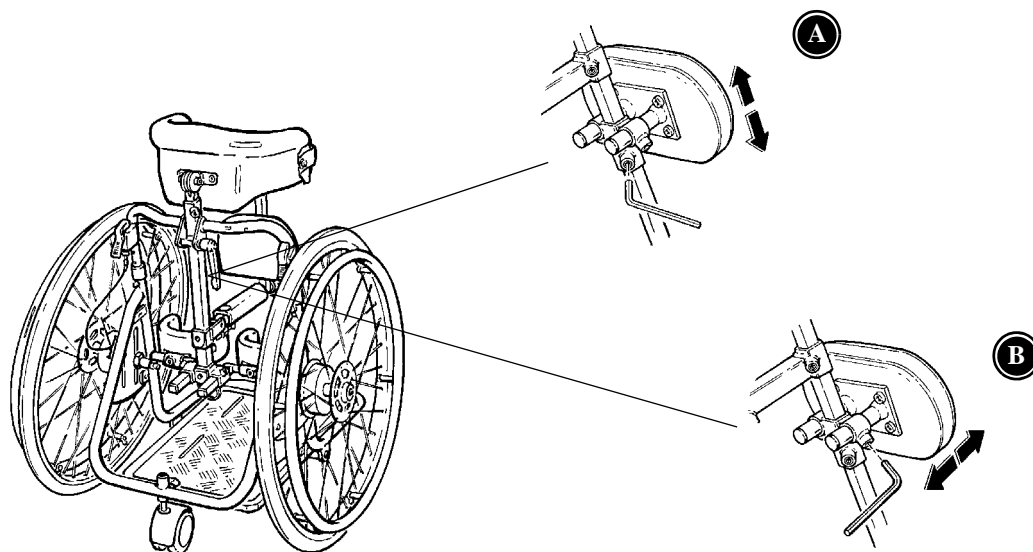
GB ABDOMINAL SUP- PORT

Mount the abdominal support on the body support.

Use the Allen Key when you adjust the height (A)

Use the Allen Key when you adjust the depth (B).

Note! You can use the abdominal support on the Rabbit size 1 and 2.

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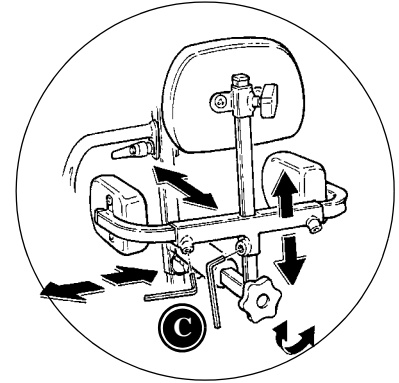
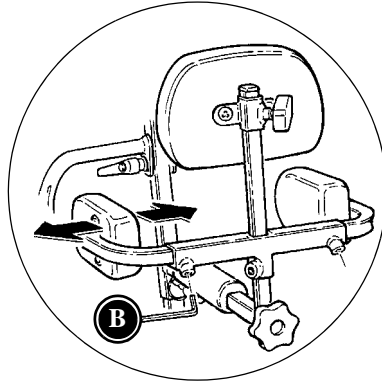
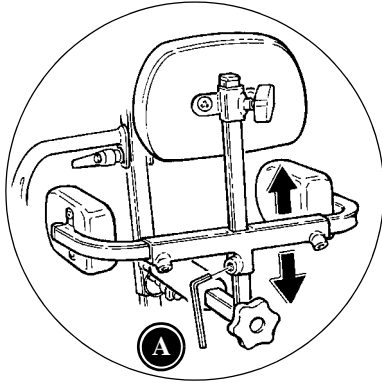
GB SIDE SUPPORTS

- (A) The side supports are mounted on the back support bar. Fasten at the wanted height with the enclosed 4 mm Allen key.
- (B) The side supports adjust in width individually with the enclosed 4 mm Allen key.
- (C) Turn the knob at the end, to adjust the complete side support unit in depth.

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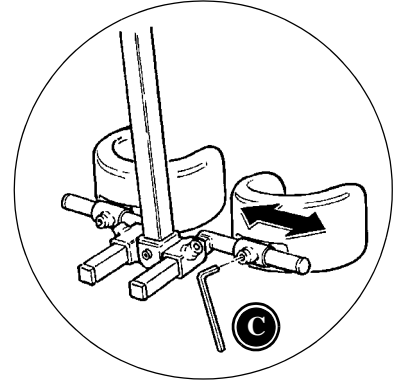
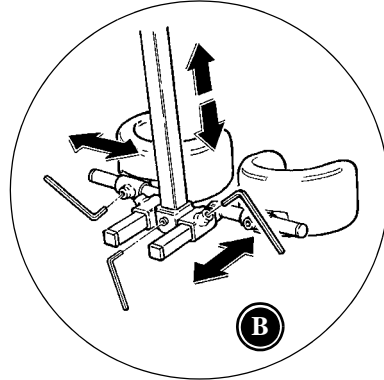
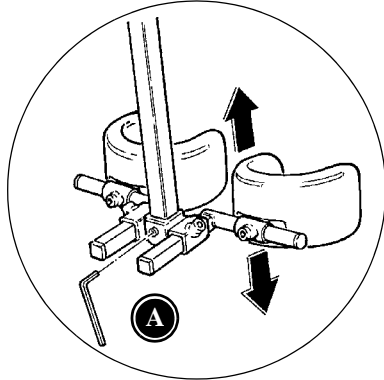
GB KNEE SUPPORTS

- (A) The knee supports are mounted on the body support bar, and fastened with the enclosed 4 mm Allen key.
- (B) The knee supports can be adjusted in depth individually. Use the enclosed 4 mm Allen key.
- (C) Use the enclosed 4 mm Allen key to adjust the width and angle of the knee supports individually.

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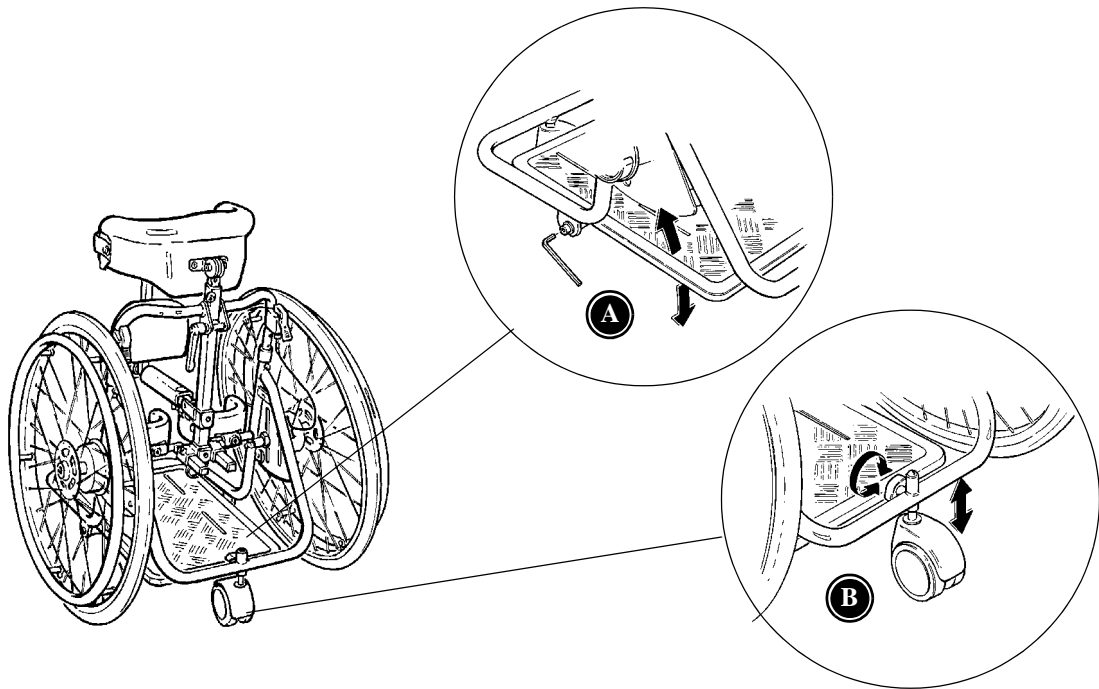
GB FOOTPLATE AND SUPPORT WHEEL

Footplate:

Use the enclosed 6 mm Allen key (A) to angle the footplate. Fasten at the wanted angle.

Support wheel:

A height adjustable support wheel is mounted on the Rabbit. Fasten with the handle (B) in the wanted height.

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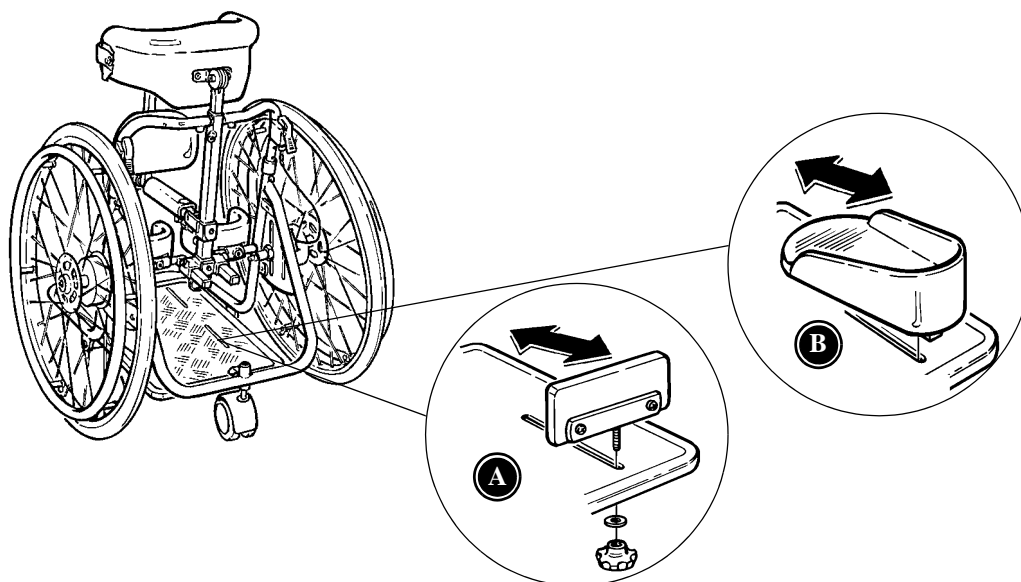
GB HEEL SUPPORT OR "DUCKS FOOT"

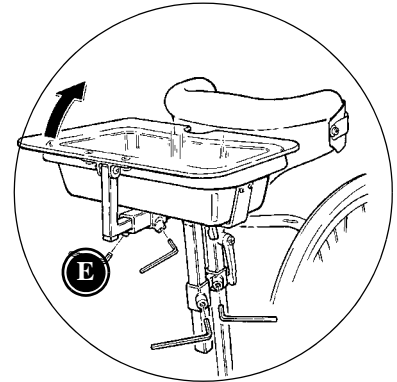
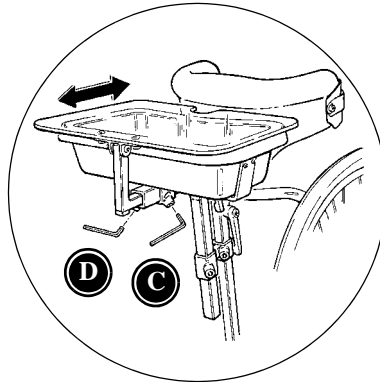
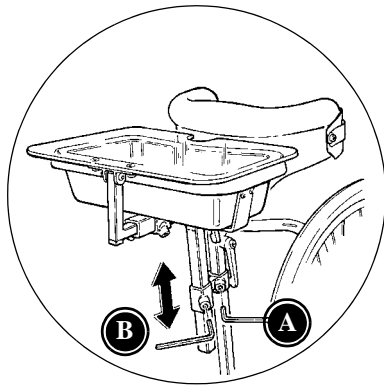
Heel support (A) or "ducks foot" (B) is mounted into the slots in the foot plate, as shown on the figures.

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GB PLAY BOX AND TRAY

Mount the play box on the body support bar with the enclosed fitting. Fasten the fitting with the enclosed 4 mm Allen key (A). Mount the bar in the fitting and fasten at the wanted height (B).

The play box can be adjusted in depth with the enclosed 4 mm Allen key (C). A tray can be mounted on the play box with the enclosed Allen key (D).

The tray is now also a lid for the play box and is easily lifted (E).

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GB ARTICLE NUMBERS

	Size 1	Size 2
Rabbit with 22" wheels	86701	-
Rabbit with 24" wheels	86711	86701
Rabbit with 26" wheels	86721	86712
Rabbit with 28" wheels	-	86722
Wheels w/spoke protectors 22"	86792-22	-
Wheels w/spoke protectors 24"	86792-24	86792-24
Wheels w/spoke protectors 26"	86792-26	86792-26
Wheels w/spoke protectors 28"	-	86792-28
Wheels size 22"	86791-22	-
Wheels size 24"	86791-24	86791-24
Wheels size 26"	86791-26	86791-26
Wheels size 28"	-	86791-28
Spoke protectors size 22"	86790-22	-
Spoke protectors size 24"	86790-24	86790-24
Spoke protectors size 26"	86790-26	86790-26
Spoke protectors size 28"	-	86790-28
Back support, complete	86731	86731
Side supports, set	86760	86760
Tray with play box	86740	86740
Tray *	86730	86730
Play box	86750	86750
Heel supports, set	86104	86104
Abdominal support	86720	86720

* Tray can only be mounted together with the play box

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Producent: R82 A/S