


Rogue XP 1" Offset Backrest Mount

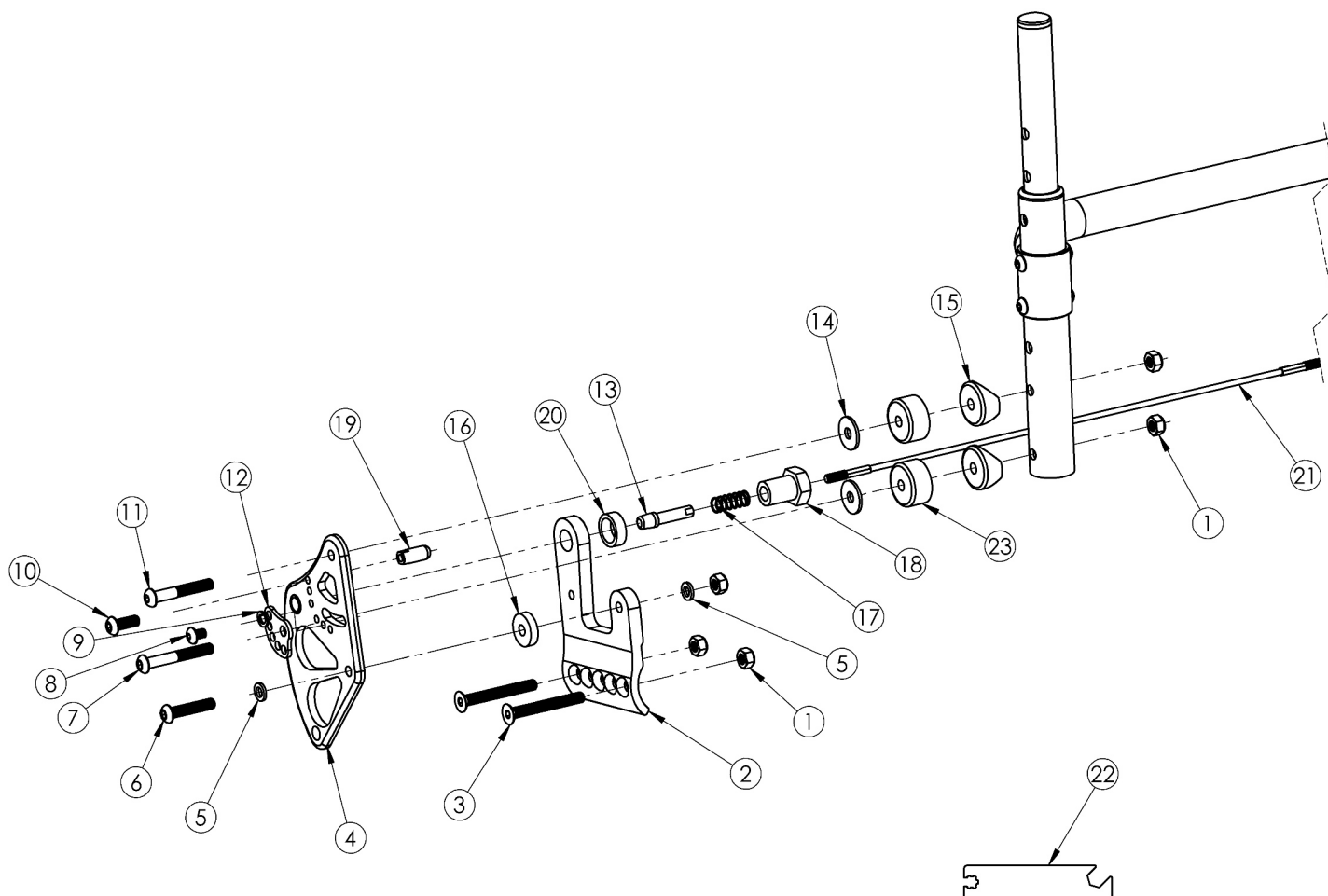

Back Release Cable based on Seat Width. If retrofitting or changing backrest mount style, a new rigidizer bar may be needed

Item Number	Part Number	Description	Quantity
1, 2a, 3a - 3c, 4a, 5-20, 23	503698	1" Inward Offset Backrest Mount Assembly, Right, Black, Rogue XP	1
1, 2a, 3a - 3c, 4b, 5-20, 23	503698_ _ _	1" Inward Offset Backrest Mount Assembly, Right, Color Anodized, Rogue XP	1
1, 2b, 3a - 3c, 4c, 5-20, 23	503699	1" Inward Offset Backrest Mount Assembly, Left, Black, Rogue XP	1
1, 2b, 3a - 3c, 4d, 5-20, 23	503699_ _ _	1" Inward Offset Backrest Mount Assembly, Left, Color Anodized, Rogue XP	1
1, 2a, 2b, 3a - 3c, 4a, 4c, 5-20, 23	503700	1" Inward Offset Backrest Mount Assembly, PAIR, Black, Rogue XP	1
1, 2a, 2b, 3a - 3c, 4b, 4d, 5-20, 23	503700_ _ _	1" Inward Offset Backrest Mount Assembly, PAIR, Color Anodized, Rogue XP	1
21a, 13, 17	121181	Back Release Cable 10" Wide Black with Lock Out Pins	1
21b, 13, 17	121182	Back Release Cable, 11" Wide, Black with Lock Out Pins	1

Item Number	Part Number	Description	Quantity
21c, 13, 17	121183	Back Release Cable, 12" Wide, Black with Lock Out Pins	1
21d, 13, 17	121184	Back Release Cable, 13" Wide, Black with Lock Out Pins	1
21e, 13, 17	121185	Back Release Cable, 14" Wide, Black with Lock Out Pins	1
21f, 13, 17	121186	Back Release Cable, 15" Wide, Black with Lock Out Pins	1
21g, 13, 17	121187	Back Release Cable, 16" Wide, Black with Lock Out Pins	1
21h, 13, 17	121188	BACK RELEASE CABLE, 17" WIDE, BLACK with Lock Out Pins	1
21i, 13, 17	121189	BACK RELEASE CABLE, 18" WIDE, BLACK with Lock Out Pins	1
1	100658	M6 Nylock Nut Blk Zn	5
2a	003098	XP Back Mount Right Rogue	1
2b	003099	XP Back Mount Left Rogue	1
3a	101129	M6 x 50 FSHCS P/THD BLKZC <i>Used when transit is not present</i>	2
3b	103068	M6 x 60mm FHCS Blk Zn <i>Use quantity 2 per side when transit is present</i>	2
3c	101130	M6 x 55 FSHCS P/THD BLKZC <i>Use quantity 1 per side when transit is present (not shown in image)</i>	1
4a	003229	Back Bracket Relaxed Assembly Right, Black Anodized	1
4b	002770_ _ _	Back Bracket Relaxed Right Color Anodized/Color Accent	1
4c	003231	Back Bracket Relaxed Assembly Left, Black Anodized	1
4d	002771_ _ _	Back Bracket Relaxed Left Color Anodized/Color Accent	1
5	102072	Washer Nylon .234 ID x .406 OD x .062 Thick	2
6	102625	M6 X 30 BHCS BLK ZC	1
7	003241	M6 x 60 BHCS P/THD Custom Blk Zc	1
8	102621	M5 x 8 BHCS BLK ZC	1
9	102614	M5 Split Lock Washer BLK ZC	1
10	101716	M5 x 16 SHCS Blk Zn	1
11	003232	M6 x 55 BHCS Blk Zc	1
12	001762	Arc Adjustment Cam	1

Item Number	Part Number	Description	Quantity
13	119660	Lockout Pin	1
14	102629	Seastrom 5610-69-062 Washer Nylon Black	2
15	000055	Saddle 1"	2
16	102630	Washer Nylon 25 X 75 X .187 Black	1
17	000965	Backrest Cable Release Spring RG	1
18	001768	Pin Receiver with Hex	1
19	000922	Backrest Stop RG	1
20	001770	Spacer For Pin Receiver <i>This spacer is not used if Swing Away Armrests are present</i>	1
21a	119651	Back Release Cable, Black, 10" Wide	1
21b	119652	Back Release Cable, Black, 11" Wide	1
21c	119653	Back Release Cable, Black, 12" Wide	1
21d	119654	Back Release Cable, Black, 13" Wide	1
21e	119655	Back Release Cable, Black, 14" Wide	1
21f	119656	Back Release Cable, Black, 15" Wide	1
21g	119657	Back Release Cable, Black, 16" Wide	1
21h	119658	Back Release Cable, Black, 17" Wide	1
21i	119659	Back Release Cable, Black, 18" Wide	1
23	002475	Offset Backrest Spacer	2

Color Anodizing Samples

