


User Activated Anti-Tips Rogue ALX

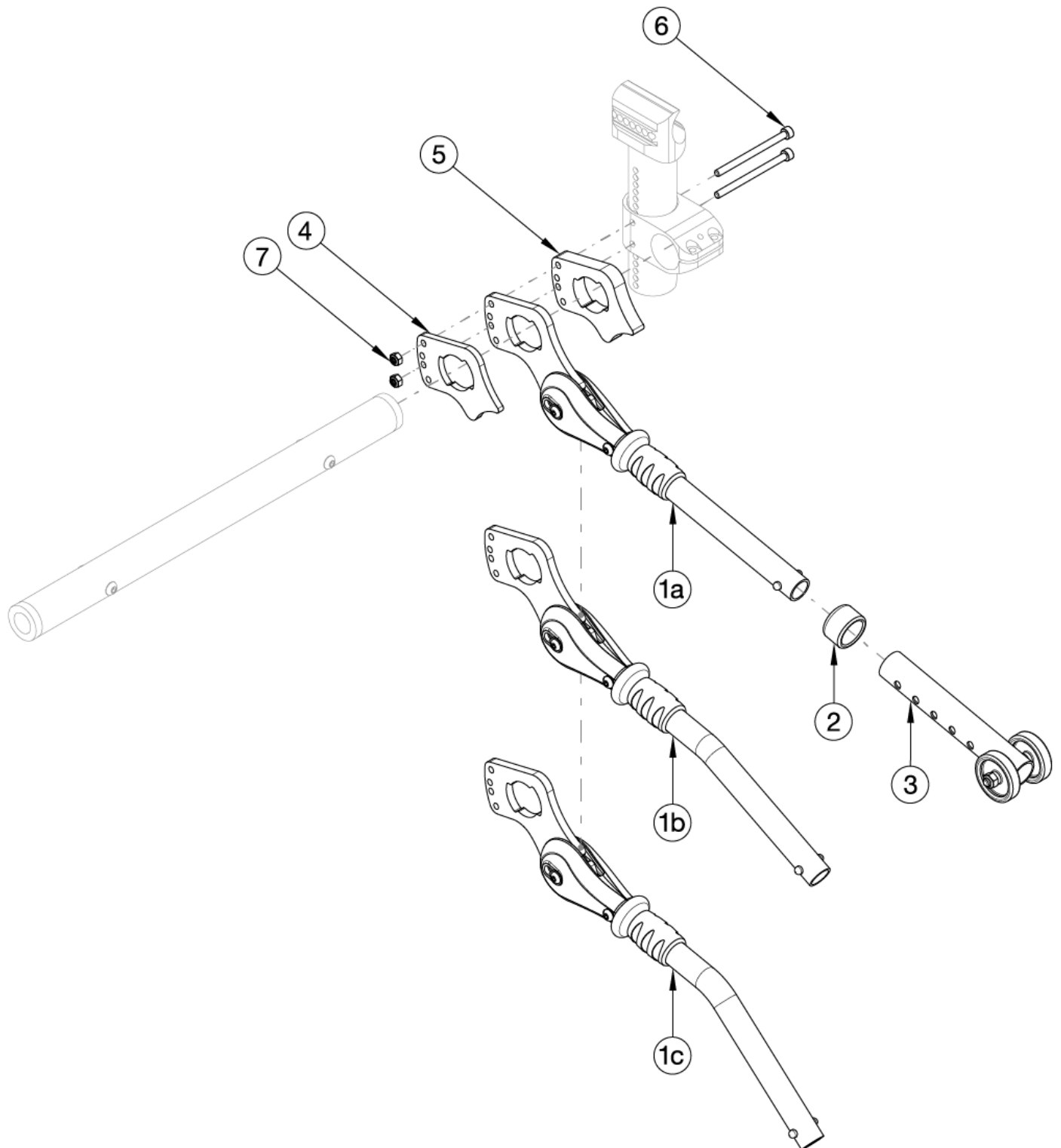

Contact Customer Service for chairs with more than 2" of variance between Front and Rear Seat Height. Complete assemblies contain "one" complete user activated anti-tip. Quantity two is required for a pair. A pair of User Activated Anti-Tips is not available if a tipping lever is present. Not available with One Arm Drive.

Item Number	Part Number	Description	Quantity
1a, 2, 3a, 4-7	122614	UserActivated Anti-Tip Complete Assembly Straight Rogue ALX	1
1b, 2, 3a, 4-7	122615	UserActivated Anti-Tip Complete Assembly Bent Rogue ALX	1
1c, 2, 3a, 4-7	122616	UserActivated Anti-Tip Complete Assembly 15 Degrees Rogue ALX	1
1a, 2, 3b, 4-7	122617	UserActivated Anti-Tip Complete Assembly X-Large Straight Rogue ALX <i>Used for 26" Wheels</i>	1
1b, 2, 3b, 4-7	122618	UserActivated Anti-Tip Complete Assembly X-Large Bent Rogue ALX <i>Used for 26" Wheels</i>	1
1c, 2, 3b, 4-7	122619	UserActivated Anti-Tip Complete Assembly X-Large 15 Degrees Rogue ALX <i>Used for 26" Wheels</i>	1
1a	119620	User Activated Anti-Tip Assembly, Straight (MX Tower), Rogue ALX	1
1b	119618	User Activated Anti-Tip Assembly, Bent (MX Tower), Rogue ALX	1
1c	119619	User Activated Anti-Tip Assembly, Bent 15 Deg (MX Tower), Rogue ALX	1
2	004306	Anti-Tip Anti Rattle Bushing	1
3a	119668	Anti-Tip Lower Assembly	1
3b	116544	Anti-Tip Lower Assembly X-Large Rogue2 / Rogue ALX <i>Used for 26" Wheels</i>	1
4	121005	User Tipper Mount Inner Spacer (MX Tower), Rogue ALX	1
5	119612	User Tipper Mount Outer Spacer (MX Tower), Rogue ALX	1
6	119614	SHCS M5 X 0.8 X 70, Partially Threaded, Black Zinc, CLS 10.9	2
7	100657	M5 Nylock Nut Blk Zn	2