


Ethos Rear Base Frame

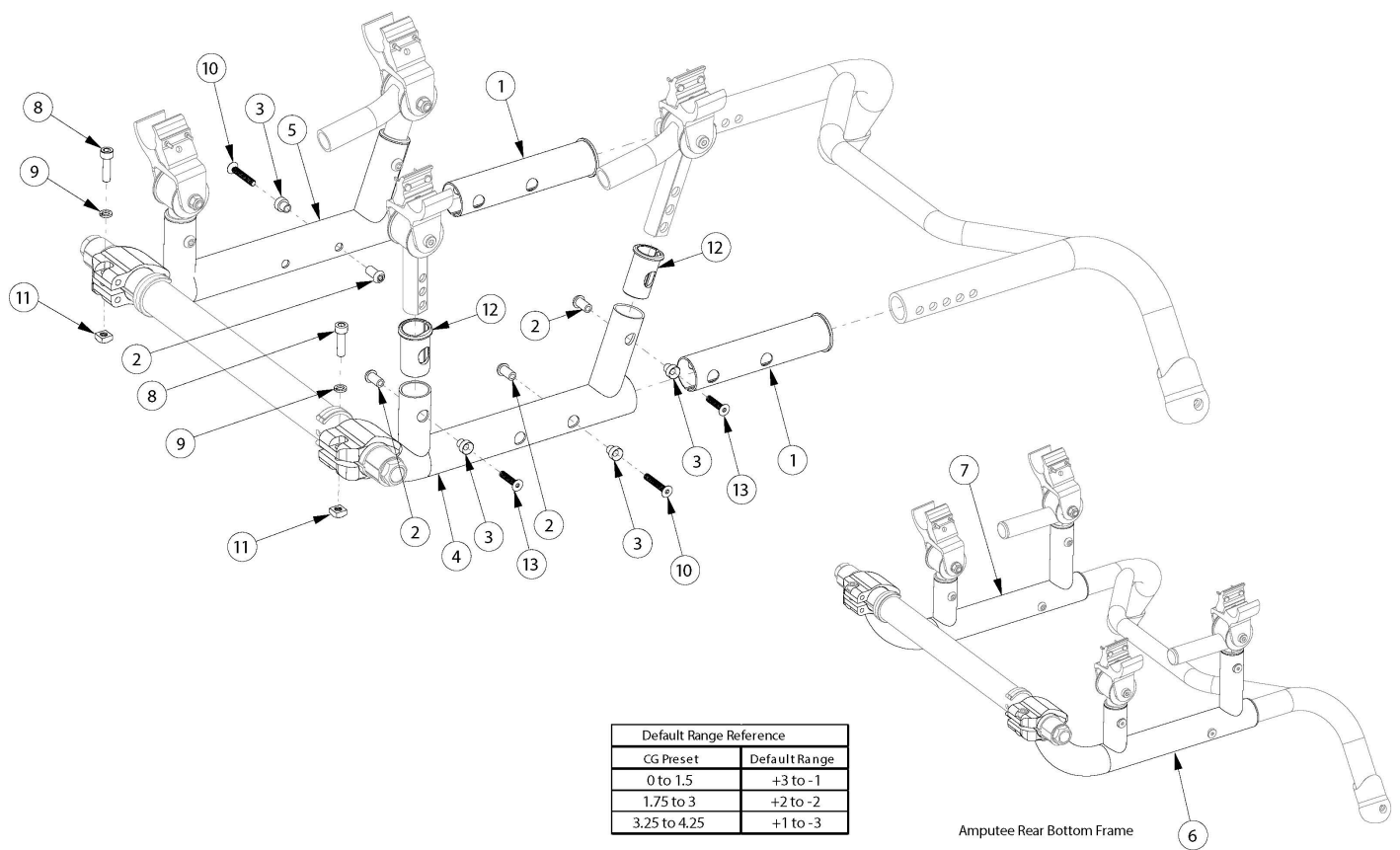

Replacement Rear Base Frames can be quoted but must be ordered through Customer Service

Item Number	Part Number	Description	Quantity	Retail
1 - 4, 10, 12, 13	RFETR	Replacement Ethos Rear Base Frame, Right	1	\$76.50
1 - 3, 5, 10, 12, 13	RFETL	Replacement Ethos Rear Base Frame, Left	1	\$76.50
1 - 5, 10, 12, 13	RFETPR	Replacement Ethos Rear Base Frame, Left and Right	1	\$153.00
1 - 3, 6, 10, 12, 13	RFETRAMP	Replacement Ethos Amputee Rear Base Frame, Right	1	\$76.50
1 - 3, 7, 10, 12, 13	RFETLRAMP	Replacement Ethos Amputee Rear Base Frame, Left	1	\$76.50
1 - 3, 6, 7, 10, 12, 13	RFETPRAMP	Replacement Ethos Amputee Rear Base Frame, Left and Right	1	\$153.00
1a	003868	Base Frame Bushing, Black	2	\$2.04
1b	107711_ _ _	Base Frame Bushing, Highlight Color	2	\$2.04
2	003869	M5 Barrel Nut	6	\$18.21

Item Number	Part Number	Description	Quantity	Retail
3	003870	Barrel, M5 FHCS	6	\$8.67
8	101845	M6 x 22 SHCS Blk Zn	2	\$1.58
9	102810	M6 Lockwasher, Hi-Collar, ZC	2	\$0.53
10	103321	M5 x 30 FHCS Blk Zc	2	\$1.02
11	103505	M6 x 1.0 Square Nut Blk Zc	2	\$1.02
12a	003867	Vertical Upright Bushing With Flat, Black	4	\$2.04
12b	108060_ _ _	Vertical Upright Bushing, Highlight Color	4	\$2.04
13	103322	M5 x 20 FHCS Blk Zc	4	\$1.02