


## **(Discontinued) Focus CR One Arm Drive Hex Style**

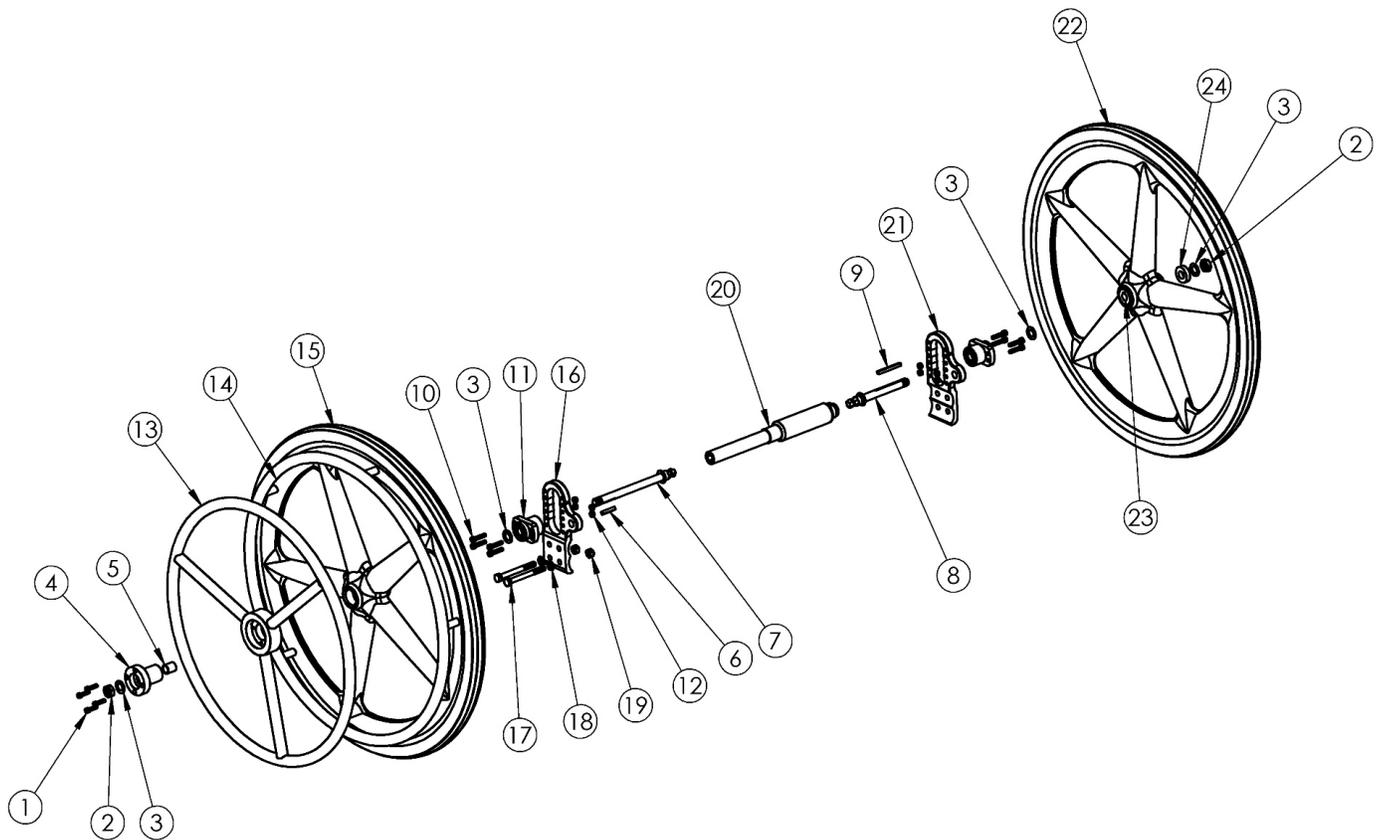

*One Arm Drive Kits do not include rear tires*

Item Number	Part Number	Description	Quantity	Retail
11, 16, 21	003212	One Arm Drive Focus Axle Plate Assembly PAIR	1	\$579.36
10	101843	M5 x 20 SHCS Blk Zn	8	\$0.53
12	100657	M5 Nylock Nut Blk Zn	8	\$0.53
13a	003198	One Arm Drive Handrim Aluminum 24"	1	\$262.65
13b	003201	One Arm Drive Handrim Poly Coated 24"	1	\$414.99
14	100975	Handrim Aluminum Anodized 5 Rivet Nut 24"	1	\$57.78
15	100960	Wheel X Core Mag 5-Spoke 24" Black	1	\$136.58
17a	101831	M8 x 80 HHCS Blk Zn	4	\$3.07
17b	101832	M8 x 90 HHCS Blk Zn	4	\$3.68
17c	102384	M8 x 100 HHCS Blk Zc Part THD	2	\$3.68

Item Number	Part Number	Description	Quantity	Retail
18	101858	Washer Flat M8 Blk Zn	8	\$0.53
19	100659	M8 Nylock Nut Blk Zn	4	\$1.05
20f	504348	One Arm Drive - Drive Shaft 17.69", Focus <i>Used on 19" wide chairs. This kit contains 102646 Drive Shaft and Installation Instructions.</i>	1	\$551.78
20i	504351	One Arm Drive - Drive Shaft 20.69", Focus <i>Used on 22" wide chairs. This kit contains 102752 Drive Shaft and Installation Instructions.</i>	1	\$551.78
22-24	003238	24" X Core Mag with One Arm Drive Hub	1	\$254.68