


Rogue2 Wheel Lock Clamps

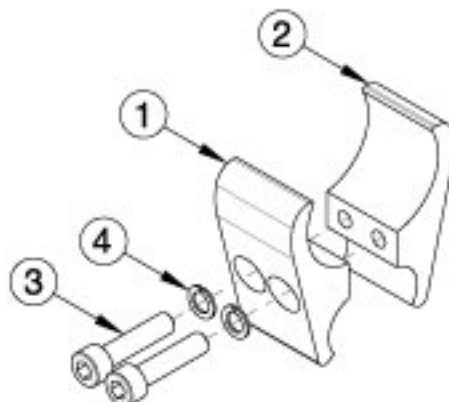

Item Number	Part Number	Description	Quantity
1a, 2a, 3, 4	501001	Wheel Lock Clamp Assembly, Rigid, Standard <i>Used with Under Seat Scissor Lock, Short Thro Scissor Lock and Low Profile Scissor Lock,</i>	1
1b, 2b, 3, 4	501002	Wheel Lock Clamp Assembly, Rigid, Modified <i>Used with Push to Lock, Push to Lock (Flush Mount), Pull to Lock, Push to Lock w/ Extension Handle, Pull to Lock w/ Extension Handle and Grade Aids</i>	1
1a	000376	Clamp Top 1.375" Rigid <i>Used with Under Seat Scissor Lock, Short Thro Scissor Lock and Low Profile Scissor Lock</i>	1
1b	000461	Clamp Top 1.375" Modified Rigid <i>Used with Push to Lock, Push to Lock (Flush Mount), Pull to Lock, Push to Lock w/ Extension Handle, Pull to Lock w/ Extension Handle and Grade Aids</i>	1
2a	000377	Clamp Bottom 1.375" Rigid <i>Used with Under Seat Scissor Lock, Short Thro Scissor Lock and Low Profile Scissor Lock</i>	1
2b	000967	Clamp Bottom 1.375" Extended Rigid <i>Used with Push to Lock, Push to Lock (Flush Mount), Pull to Lock, Push to Lock w/ Extension Handle, Pull to Lock w/ Extension Handle and Grade Aids</i>	1
3	100780	M6 x 25 SHCS ZC	2
4	101835	M6 Lock Washer Blk Zn	2