




Please read these instructions carefully before beginning the assembly. Failure to understand and follow assembly instructions may result in injury to technician and/or end user and may void the warranty. If you have any questions call Sunrise Medical Technical Support at +1 (800) 333-4000.

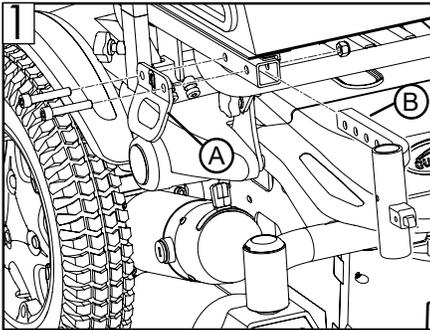

Parts:

- 1. 1 ea Receiver assembly RH
- 2. 1 ea Receiver assembly LH
- 3. 2 ea Wire harness (power ELR hanger only)






Required tools:

- 1. 6mm hex key
- 2. 13mm Socket wrench or Torque wrench

A. Hanger Receiver

Hanger receiver parts can be mounted in several configurations. To mount legrest to inside or outside of seat tube:

1. Remove swing-away legrest.
2. Remove receiver assembly:
 - a. Remove transit hook (A) and fasteners. (Fig. 1)
 - b. Remove hanger receiver (B) from the seat. (Fig. 1)
3. Mount receiver assembly:
 - a. Place hanger receiver in the desired position (for mounting inside the seat tube refer to B-Fig. 1, for offset mounting on the outside of the seat tube refer to D-Fig. 3, for offset mounting on the inside of the seat tube refer to D-Fig. 4). For offset mounting, you will need mounting block (C) (Fig. 2) to be inserted in the seat tube with longer hardware (F-Fig. 3/F-Fig. 4). Refer to parts manual for details.
 - b. Attach the transit hook (E) to the seat tube with screws (F). Torque to 13-17 Nm [115-150 in-lbs]. Attach nylon-insert nuts (G). (Fig. 3 and Fig. 4)
Note: it is recommended that all nylon-insert nuts be replaced.
 - c. Insert end caps (H) in front side of seat tube and rear side of hanger receiver. (Fig 5)
 - d. For power ELR hangers, insert the small end of the wire harness (I) into the top of the hanger receiver. Securely tighten the set screw (J) with a 3mm hex key to secure the harness to the receiver. Torque to 1.5-2.0 Nm [6-8 in-lbs]. (Fig. 6)
4. Repeat steps 1-3 on opposite side of chair.

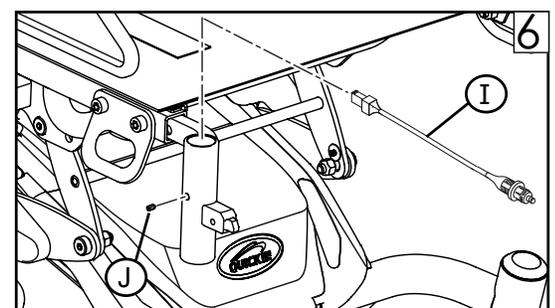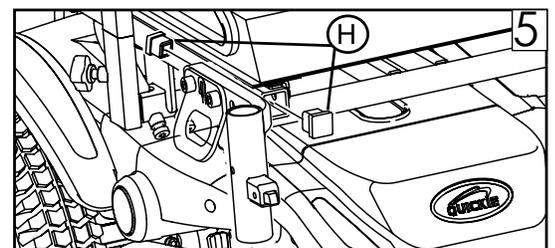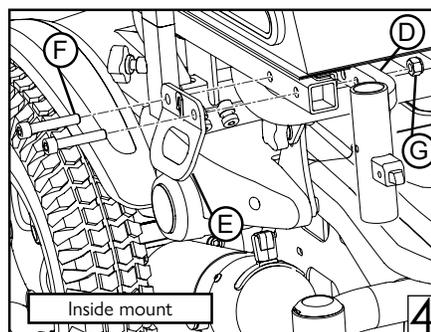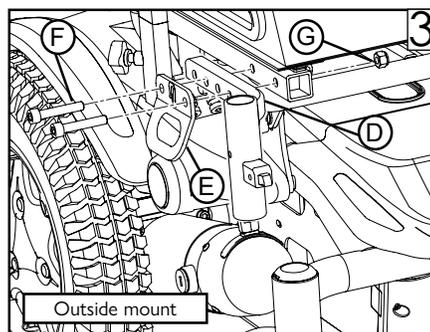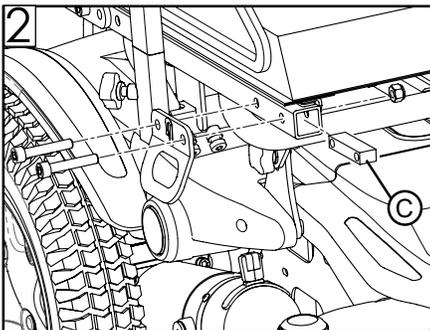

⚠ DEALER/TECHNICIAN WARNING

Attention dealers and qualified technicians, do not operate or service this device without first reading the owners manual. If you do not understand the instructions and warnings in the owners manual please contact the Quickie Technical Service Department before operating and/or servicing the Quickie device. Failure to do so may result in damage and/or injury. Find more information and important warnings see the wheelchair owner's manual or at: www.SunriseMedical.com.