


# Rogue ALX Adjustable Height Backrest Frame with Non-Height Adjustable Rigidizer Bar (Formerly Tsunami)

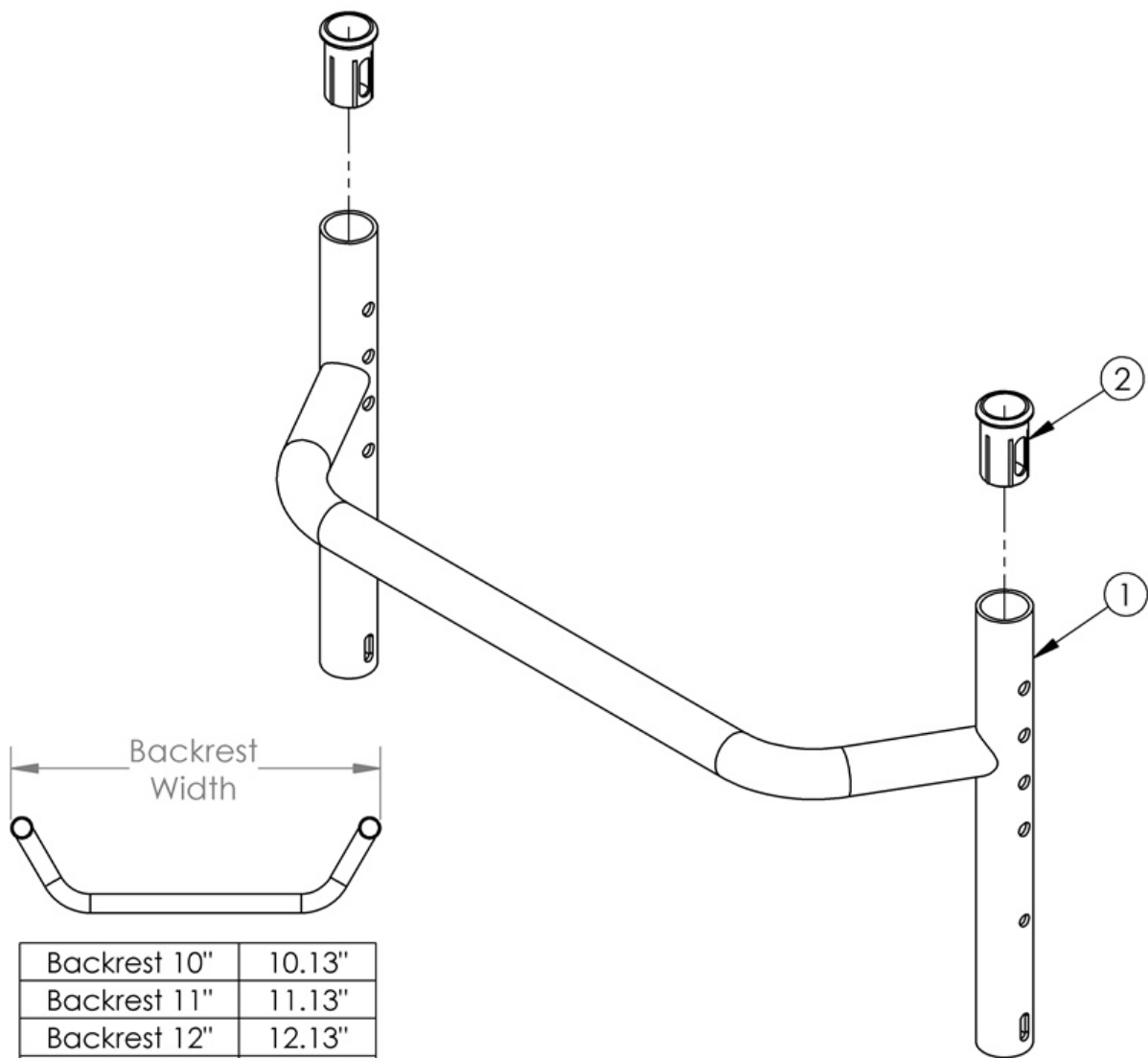

Backrest 10"	10.13"
Backrest 11"	11.13"
Backrest 12"	12.13"
Backrest 13"	13.13"
Backrest 14"	14.13"
Backrest 15"	15.13"
Backrest 16"	16.13"
Backrest 17"	17.13"
Backrest 18"	18.13"
Backrest 19"	19.13"
Backrest 20"	20.13"
Backrest 21"	21.13"
Backrest 22"	22.13"

Item Number	Part Number	Description	Quantity
1a, 2	110637	10" Rigid Backrest with Sleeves <i>Used with standard push handles, low profile push handles or omit push handles in any back height range. Also used with fold down push handles with a tall back height range.</i>	1
1b, 2	110638	11" Rigid Backrest with Sleeves <i>Used with standard push handles, low profile push handles or omit push handles in any back height range. Also used with fold down push handles with a tall back height range.</i>	1
1c, 2	110639	12" Rigid Backrest with Sleeves <i>Used with standard push handles, low profile push handles or omit push handles in any back height range. Also used with fold down push handles with a tall back height range.</i>	1
1d, 2	110640	13" Rigid Backrest with Sleeves <i>Used with standard push handles, low profile push handles or omit push handles in any back height range. Also used with fold down push handles with a tall back height range.</i>	1
1e, 2	110641	14" Rigid Backrest with Sleeves <i>Used with standard push handles, low profile push handles or omit push handles in any back height range. Also used with fold down push handles with a tall back height range.</i>	1
1f, 2	110642	15" Rigid Backrest with Sleeves <i>Used with standard push handles, low profile push handles or omit push handles in any back height range. Also used with fold down push handles with a tall back height range.</i>	1
1g, 2	110643	16" Rigid Backrest with Sleeves <i>Used with standard push handles, low profile push handles or omit push handles in any back height range. Also used with fold down push handles with a tall back height range.</i>	1
1h, 2	110644	17" Rigid Backrest with Sleeves <i>Used with standard push handles, low profile push handles or omit push handles in any back height range. Also used with fold down push handles with a tall back height range.</i>	1
1i, 2	110645	18" Rigid Backrest with Sleeves <i>Used with standard push handles, low profile push handles or omit push handles in any back height range. Also used with fold down push handles with a tall back height range.</i>	1
1j, 2	110646	19" Rigid Backrest with Sleeves <i>Used with standard push handles, low profile push handles or omit push handles in any back height range. Also used with fold down push handles with a tall back height range.</i>	1
1k, 2	110647	20" Rigid Backrest with Sleeves <i>Used with standard push handles, low profile push handles or omit push handles in any back height range. Also used with fold down push handles with a tall back height range.</i>	1
1l, 2	110648	21" Rigid Backrest with Sleeves <i>Used with standard push handles, low profile push handles or omit push handles in any back height range. Also used with fold down push handles with a tall back height range.</i>	1

Item Number	Part Number	Description	Quantity
1m, 2	110649	22" Rigid Backrest with Sleeves <i>Used with standard push handles, low profile push handles or omit push handles in any back height range. Also used with fold down push handles with a tall back height range.</i>	1
1n, 2	110650	10" Rigid Backrest with Sleeves for Fold Down Push Handles, Short/Medium <i>Used with fold down push handles with a short or medium back height range</i>	1
1o, 2	110651	11" Rigid Backrest with Sleeves for Fold Down Push Handles, Short/Medium <i>Used with fold down push handles with a short or medium back height range</i>	1
1p, 2	110652	12" Rigid Backrest with Sleeves for Fold Down Push Handles, Short/Medium <i>Used with fold down push handles with a short or medium back height range</i>	1
1q, 2	110653	13" Rigid Backrest with Sleeves for Fold Down Push Handles, Short/Medium <i>Used with fold down push handles with a short or medium back height range</i>	1
1r, 2	110654	14" Rigid Backrest with Sleeves for Fold Down Push Handles, Short/Medium <i>Used with fold down push handles with a short or medium back height range</i>	1
1s, 2	110655	15" Rigid Backrest with Sleeves for Fold Down Push Handles, Short/Medium <i>Used with fold down push handles with a short or medium back height range</i>	1
1t, 2	110656	16" Rigid Backrest with Sleeves for Fold Down Push Handles, Short/Medium <i>Used with fold down push handles with a short or medium back height range</i>	1
1u, 2	110657	17" Rigid Backrest with Sleeves for Fold Down Push Handles, Short/Medium <i>Used with fold down push handles with a short or medium back height range</i>	1
1v, 2	110658	18" Rigid Backrest with Sleeves for Fold Down Push Handles, Short/Medium <i>Used with fold down push handles with a short or medium back height range</i>	1
1w, 2	110659	19" Rigid Backrest with Sleeves for Fold Down Push Handles, Short/Medium <i>Used with fold down push handles with a short or medium back height range</i>	1
1x, 2	110660	20" Rigid Backrest with Sleeves for Fold Down Push Handles, Short/Medium <i>Used with fold down push handles with a short or medium back height range</i>	1
1y, 2	110661	21" Rigid Backrest with Sleeves for Fold Down Push Handles, Short/Medium <i>Used with fold down push handles with a short or medium back height range</i>	1
1z, 2	110662	22" Rigid Backrest with Sleeves for Fold Down Push Handles, Short/Medium <i>Used with fold down push handles with a short or medium back height range</i>	1
2	000034	Sleeve 3/4" x 1 1/2"	2