


# Arc Seat Frame - Growth

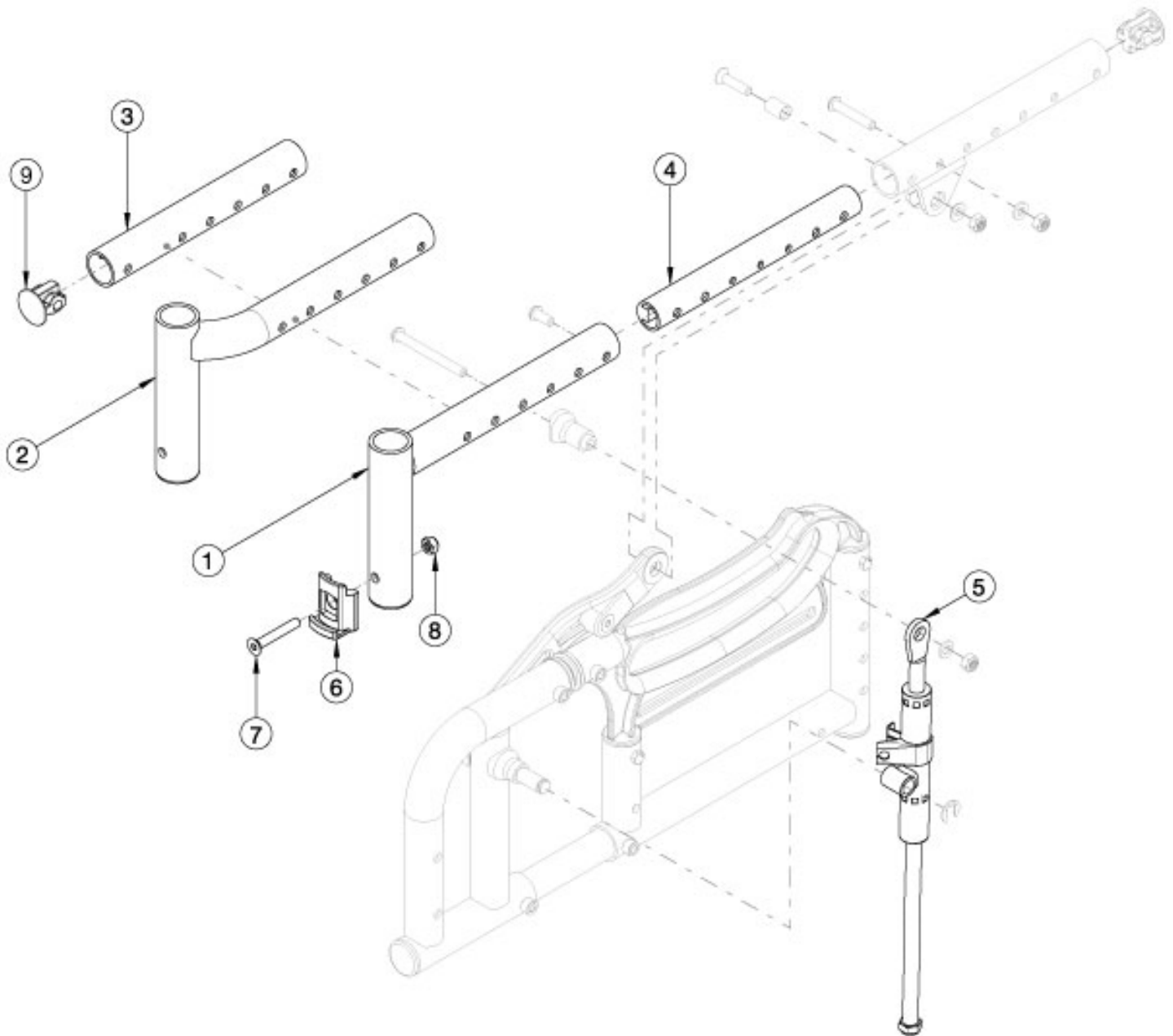

Is your depth changing? Will your existing seat frame achieve your new depth? If no, please select the appropriate seat frames below. Select color of parts based on Seat Frame color of chair. If changing from extra short depth range to short, medium, or long, new mechlocks are needed.

Item Number	Part Number	Description	Quantity
1a, 4b, 6-8	113121_ _ _	Seat Frame Assembly, Standard, X-Short, Arc <i>Accommodates 12-15" Depths</i>	1
1b, 4a, 6-8	113122_ _ _	Seat Frame Assembly, Standard, Short, Arc <i>Accommodates 14-17" Depths</i>	1

Item Number	Part Number	Description	Quantity
1c, 4a, 6-8	113123_ _ _	Seat Frame Assembly, Standard, Medium, Arc <i>Accommodates 16-19" Depths</i>	1
1d, 4a, 6-8	113124_ _ _	Seat Frame Assembly, Standard, Long, Arc <i>Accommodates 18-20" Depths</i>	1
2a, 4b, 6-9	113125_ _ _	Seat Frame Assembly, Offset, X-Short, Right, Arc <i>Accommodates 12-15" Depths</i>	1
2e, 4b, 6-9	113126_ _ _	Seat Frame Assembly, Offset, X-Short, Left, Arc <i>Accommodates 12-15" Depths</i>	1
2b, 4a, 6-9	113127_ _ _	Seat Frame Assembly, Offset, Short, Right, Arc <i>Accommodates 14-17" Depths</i>	1
2f, 4a, 6-9	113128_ _ _	Seat Frame Assembly, Offset, Short, Left, Arc <i>Accommodates 14-17" Depths</i>	1
2c, 4a, 6-9	113129_ _ _	Seat Frame Assembly, Offset, Medium, Right, Arc <i>Accommodates 16-19" Depths</i>	1
2g, 4a, 6-9	113130_ _ _	Seat Frame Assembly, Offset, Medium, Left, Arc <i>Accommodates 16-19" Depths</i>	1
2d, 4a, 6-9	113131_ _ _	Seat Frame Assembly, Offset, Long, Right, Arc <i>Accommodates 18-20" Depths</i>	1
2h, 4a, 6-9	113132_ _ _	Seat Frame Assembly, Offset, Long, Left, Arc <i>Accommodates 18-20" Depths</i>	1
3a, 4b, 9	113133_ _ _	Seat Frame Assembly, Contracture, X-Short, Arc <i>Accommodates 12-15" Depths</i>	1
3b, 4a, 9	113134_ _ _	Seat Frame Assembly, Contracture, Short, Arc <i>Accommodates 14-17" Depths</i>	1
3c, 4a, 9	113135_ _ _	Seat Frame Assembly, Contracture, Medium, Arc <i>Accommodates 16-19" Depths</i>	1
3d, 4a, 9	113136_ _ _	Seat Frame Assembly, Contracture, Long, Arc <i>Accommodates 18-20" Depths</i>	1
4a	110165	Growth Tube, Seat <i>Used on short, medium, and long seat depth ranges</i>	1
4b	110166	Growth Tube, Seat, Extra Short <i>Used on x-short seat depth range</i>	1
5a	110206	Mechlock, 194mm Stroke, Right <i>Used on short, medium, and long seat depth ranges</i>	1
5b	110207	Mechlock, 194mm Stroke, Left <i>Used on short, medium, and long seat depth ranges</i>	1

<b>Item Number</b>	<b>Part Number</b>	<b>Description</b>	<b>Quantity</b>
5c	110208	Mechlock, 190mm Stroke, Right <i>Used on x-short seat depth range</i>	1
5d	110209	Mechlock, 190mm Stroke, Left <i>Used on x-short seat depth range</i>	1