
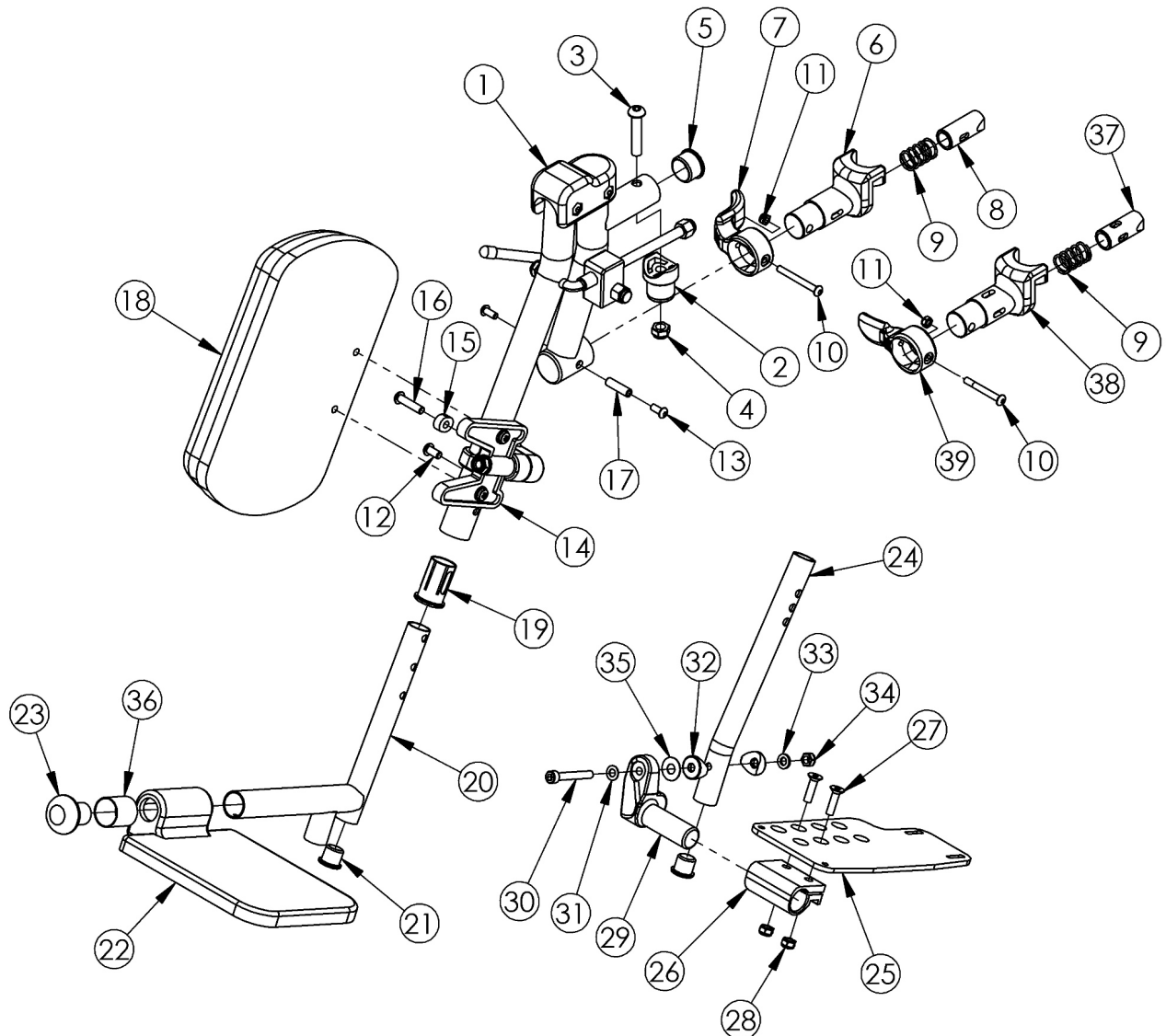


AADL Catalyst Elevating Leg Rest

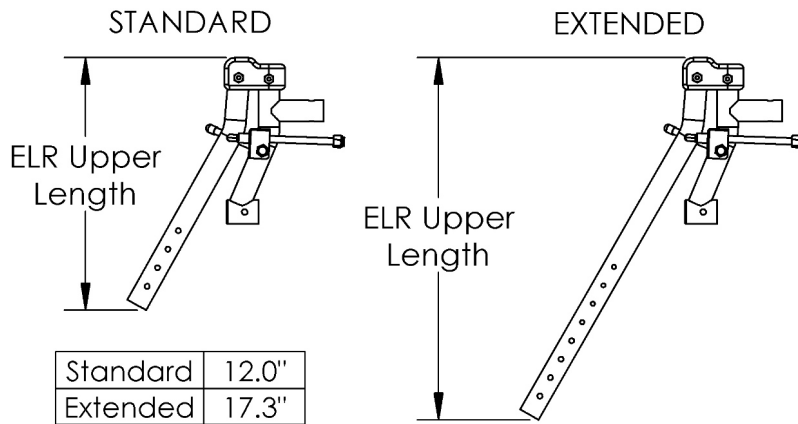

Small composite footplates are used on 14-16" seat widths. Large composite footplates are used on 17-22" seat widths Sizes for assemblies are based on seat width For a complete Elevating Leg Rest, select an upper assembly and the desired extension tube and footplate assembly

| Item Number | Part Number | Description | Quantity |
|--|-------------|---|----------|
| 1a, 1b, 2-17, 18a, 19, 20b, 21, 22b, 23 | 500556 | ELR Assembly 17-22 Large Composite Footplate Long Ext, Classic Release, PAIR | 1 |
| 1a, 2-17, 18a, 19, 20b, 21, 22b, 23 | 500563 | ELR Assembly 17-22 Large Composite Footplate Long Ext, Classic Release, Right | 1 |
| 1b, 2-17, 18a, 19, 20a, 21, 22b, 23 | 500568 | ELR Assembly 17-22 Large Composite Footplate Short Ext, Classic Release, Left | 1 |
| 1b, 2-17, 18a, 19, 20b, 21, 22b, 23 | 500569 | ELR Assembly 17-22 Large Composite Footplate Long Ext, Classic Release, Left | 1 |
| 1a, 1b, 2-17, 18b, 19, 21, 24b, 25a, 25b, 26-35 | 501040 | ELR Assembly 14 Angle Adj Footplate Long Ext, Classic Release, PAIR | 1 |
| 1a, 1b, 2-17, 18b, 19, 21, 24a, 25a, 25b, 26-35 | 501041 | ELR Assembly 14 Angle Adj Footplate Short Ext, Classic Release, PAIR | 1 |
| 1a, 1b, 2-17, 18a, 19, 21, 24b, 25a, 25b, 26-35 | 500570 | ELR Assembly 15 Angle Adj Footplate Long Ext, Classic Release, PAIR | 1 |
| 1a, 1b, 2-17, 18a, 19, 21, 24a, 25a, 25b, 26-35 | 500571 | ELR Assembly 15 Angle Adj Footplate Short Ext, Classic Release, PAIR | 1 |
| 1a, 1b, 2-17, 18a, 19, 21, 24b, 25c, 25d, 26-35 | 500572 | ELR Assembly 16-17 Angle Adj Footplate Long Ext, Classic Release, PAIR | 1 |
| 1a, 1b, 2-17, 18a, 19, 21, 24a, 25c, 25d, 26-35 | 500573 | ELR Assembly 16-17 Angle Adj Footplate Short Ext, Classic Release, PAIR | 1 |
| 1a, 1b, 2-17, 18a, 19, 21, 24b, 25e, 25f, 26-35 | 500574 | ELR Assembly 18-22 Angle Adj Footplate Long Ext, Classic Release, PAIR | 1 |
| 1a, 1b, 2-17, 18a, 19, 21, 24a, 25e, 25f, 26-35 | 500575 | ELR Assembly 18-22 Angle Adj Footplate Short Ext, Classic Release, PAIR | 1 |
| 1a, 2-17, 18b, 19, 21, 24b, 25a, 26-28, 29a, 30-35 | 501042 | ELR Assembly 14 Angle Adj Footplate Long Ext, Classic Release, Right | 1 |
| 1a, 2-17, 18b, 19, 21, 24a, 25a, 26-28, 29a, 30-35 | 501043 | ELR Assembly 14 Angle Adj Footplate Short Ext, Classic Release, Right | 1 |

| Item Number | Part Number | Description | Quantity |
|--|-------------|--|----------|
| 1a, 2-17, 18a, 19, 21, 24b, 25a, 26-28, 29a, 30-35 | 500576 | ELR Assembly 15 Angle Adj Footplate Long Ext, Classic Release, Right | 1 |
| 1a, 2-17, 18a, 19, 21, 24a, 25a, 26-28, 29a, 30-35 | 500577 | ELR Assembly 15 Angle Adj Footplate Short Ext, Classic Release, Right | 1 |
| 1a, 2-17, 18a, 19, 21, 24b, 25c, 26-28, 29a, 30-35 | 500578 | ELR Assembly 16-17 Angle Adj Footplate Long Ext, Classic Release, Right | 1 |
| 1a, 2-17, 18a, 19, 21, 24a, 25c, 26-28, 29a, 30-35 | 500579 | ELR Assembly 16-17 Angle Adj Footplate Short Ext, Classic Release, Right | 1 |
| 1a, 2-17, 18a, 19, 21, 24b, 25e, 26-28, 29a, 30-35 | 500580 | ELR Assembly 18-22 Angle Adj Footplate Long Ext, Classic Release, Right | 1 |
| 1a, 2-17, 18a, 19, 21, 24a, 25e, 26-28, 29a, 30-35 | 500581 | ELR Assembly 18-22 Angle Adj Footplate Short Ext, Classic Release, Right | 1 |
| 1b, 2-17, 18b, 19, 21, 24b, 25b, 26-28, 29b, 30-35 | 501044 | ELR Assembly 14 Angle Adj Footplate Long Ext, Classic Release, Left | 1 |
| 1b, 2-17, 18b, 19, 21, 24a, 25b, 26-28, 29b, 30-35 | 501045 | ELR Assembly 14 Angle Adj Footplate Short Ext, Classic Release, Left | 1 |
| 1b, 2-17, 18a, 19, 21, 24b, 25b, 26-28, 29b, 30-35 | 500582 | ELR Assembly 15 Angle Adj Footplate Long Ext, Classic Release, Left | 1 |
| 1b, 2-17, 18a, 19, 21, 24a, 25b, 26-28, 29b, 30-35 | 500583 | ELR Assembly 15 Angle Adj Footplate Short Ext, Classic Release, Left | 1 |
| 1b, 2-17, 18a, 19, 21, 24b, 25d, 26-28, 29b, 30-35 | 500584 | ELR Assembly 16-17 Angle Adj Footplate Long Ext, Classic Release, Left | 1 |
| 1b, 2-17, 18a, 19, 21, 24a, 25d, 26-28, 29b, 30-35 | 500585 | ELR Assembly 16-17 Angle Adj Footplate Short Ext, Classic Release, Left | 1 |
| 1b, 2-17, 18a, 19, 21, 24b, 25f, 26-28, 29b, 30-35 | 500586 | ELR Assembly 18-22 Angle Adj Footplate Long Ext, Classic Release, Left | 1 |
| 1b, 2-17, 18a, 19, 21, 24a, 25f, 26-28, 29b, 30-35 | 500587 | ELR Assembly 18-22 Angle Adj Footplate Short Ext, Classic Release, Left | 1 |

| Item Number | Part Number | Description | Quantity |
|--|-------------|---|----------|
| 1c, 1d, 2-17, 18a, 19, 20b, 21, 22b, 23 | 501563 | Extended Length ELR Assembly 17-22 Large Composite Footplate Long Ext, Classic Release, PAIR | 1 |
| 1c, 2-17, 18a, 19, 20b, 21, 22b, 23 | 501570 | Extended Length ELR Assembly 17-22 Large Composite Footplate Long Ext, Classic Release, Right | 1 |
| 1d, 2-17, 18a, 19, 20b, 21, 22b, 23 | 501576 | Extended Length ELR Assembly 17-22 Large Composite Footplate Long Ext, Classic Release, Left | 1 |
| 1c, 1d, 2-17, 18b, 19, 21, 24b, 25a, 25b, 26-35 | 501577 | Extended Length ELR Assembly 14 Angle Adj Footplate Long Ext, Classic Release, PAIR | 1 |
| 1c, 1d, 2-17, 18b, 19, 21, 24a, 25a, 25b, 26-35 | 501578 | Extended Length ELR Assembly 14 Angle Adj Footplate Short Ext, Classic Release, PAIR | 1 |
| 1c, 1d, 2-17, 18a, 19, 21, 24b, 25a, 25b, 26-35 | 501579 | Extended Length ELR Assembly 15 Angle Adj Footplate Long Ext, Classic Release, PAIR | 1 |
| 1c, 1d, 2-17, 18a, 19, 21, 24a, 25a, 25b, 26-35 | 501580 | Extended Length ELR Assembly 15 Angle Adj Footplate Short Ext, Classic Release, PAIR | 1 |
| 1c, 1d, 2-17, 18a, 19, 21, 24b, 25c, 25d, 26-35 | 501581 | Extended Length ELR Assembly 16-17 Angle Adj Footplate Long Ext, Classic Release, PAIR | 1 |
| 1c, 1d, 2-17, 18a, 19, 21, 24a, 25c, 25d, 26-35 | 501582 | Extended Length ELR Assembly 16-17 Angle Adj Footplate Short Ext, Classic Release, PAIR | 1 |
| 1c, 1d, 2-17, 18a, 19, 21, 24b, 25e, 25f, 26-35 | 501583 | Extended Length ELR Assembly 18-22 Angle Adj Footplate Long Ext, Classic Release, PAIR | 1 |
| 1c, 1d, 2-17, 18a, 19, 21, 24a, 25e, 25f, 26-35 | 501584 | Extended Length ELR Assembly 18-22 Angle Adj Footplate Short Ext, Classic Release, PAIR | 1 |
| 1c, 2-17, 18b, 19, 21, 24b, 25a, 26-28, 29a, 30-35 | 501585 | Extended Length ELR Assembly 14 Angle Adj Footplate Long Ext, Classic Release, Right | 1 |
| 1c, 2-17, 18b, 19, 21, 24a, 25a, 26-28, 29a, 30-35 | 501586 | Extended Length ELR Assembly 14 Angle Adj Footplate Short Ext, Classic Release, Right | 1 |
| 1c, 2-17, 18a, 19, 21, 24b, 25a, 26-28, 29a, 30-35 | 501587 | Extended Length ELR Assembly 15 Angle Adj Footplate Long Ext, Classic Release, Right | 1 |
| 1c, 2-17, 18a, 19, 21, 24a, 25a, 26-28, 29a, 30-35 | 501588 | Extended Length ELR Assembly 15 Angle Adj Footplate Short Ext, Classic Release, Right | 1 |

| Item Number | Part Number | Description | Quantity |
|--|-------------|--|----------|
| 1c, 2-17, 18a, 19, 21, 24b, 25c, 26-28, 29a, 30-35 | 501589 | Extended Length ELR Assembly 16-17 Angle Adj Footplate Long Ext, Classic Release, Right | 1 |
| 1c, 2-17, 18a, 19, 21, 24a, 25c, 26-28, 29a, 30-35 | 501590 | Extended Length ELR Assembly 16-17 Angle Adj Footplate Short Ext, Classic Release, Right | 1 |
| 1c, 2-17, 18a, 19, 21, 24b, 25e, 26-28, 29a, 30-35 | 501591 | Extended Length ELR Assembly 18-22 Angle Adj Footplate Long Ext, Classic Release, Right | 1 |
| 1c, 2-17, 18a, 19, 21, 24a, 25e, 26-28, 29a, 30-35 | 501592 | Extended Length ELR Assembly 18-22 Angle Adj Footplate Short Ext, Classic Release, Right | 1 |
| 1d, 2-17, 18b, 19, 21, 24b, 25b, 26-28, 29b, 30-35 | 501593 | Extended Length ELR Assembly 14 Angle Adj Footplate Long Ext, Classic Release, Left | 1 |
| 1d, 2-17, 18b, 19, 21, 24a, 25b, 26-28, 29b, 30-35 | 501594 | Extended Length ELR Assembly 14 Angle Adj Footplate Short Ext, Classic Release, Left | 1 |
| 1d, 2-17, 18a, 19, 21, 24b, 25b, 26-28, 29b, 30-35 | 501595 | Extended Length ELR Assembly 15 Angle Adj Footplate Long Ext, Classic Release, Left | 1 |
| 1d, 2-17, 18a, 19, 21, 24a, 25b, 26-28, 29b, 30-35 | 501596 | Extended Length ELR Assembly 15 Angle Adj Footplate Short Ext, Classic Release, Left | 1 |
| 1d, 2-17, 18a, 19, 21, 24b, 25d, 26-28, 29b, 30-35 | 501597 | Extended Length ELR Assembly 16-17 Angle Adj Footplate Long Ext, Classic Release, Left | 1 |
| 1d, 2-17, 18a, 19, 21, 24a, 25d, 26-28, 29b, 30-35 | 501598 | Extended Length ELR Assembly 16-17 Angle Adj Footplate Short Ext, Classic Release, Left | 1 |
| 1d, 2-17, 18a, 19, 21, 24b, 25f, 26-28, 29b, 30-35 | 501599 | Extended Length ELR Assembly 18-22 Angle Adj Footplate Long Ext, Classic Release, Left | 1 |
| 1d, 2-17, 18a, 19, 21, 24a, 25f, 26-28, 29b, 30-35 | 501600 | Extended Length ELR Assembly 18-22 Angle Adj Footplate Short Ext, Classic Release, Left | 1 |
| 1a | 500553-R | ELR Upper Standard Length Right | 1 |
| 1b | 500553L | ELR Upper Standard Length Left | 1 |
| 1c | 500601R | ELR Upper Extended Length Right | 1 |
| 1d | 500601L | ELR Upper Extended Length Left | 1 |
| 2 | 000032 | Footrest Hanger Pivot Saddle | 1 |

| Item Number | Part Number | Description | Quantity |
|------------------|-------------|---|----------|
| 3 | 100673 | M8 x 45 BTN SHCS BLKZC | 1 |
| 4 | 100659 | M8 Nylock Nut Blk Zn | 1 |
| 5 | 100804 | Tube End Plug 1" | 1 |
| 7 | 001158 | Lever Swing Away Short | 1 |
| 9 | 000119 | Hanger Spring | 1 |
| 10 | 001374 | M5 x 40 BHCS P/THD BLKZC | 1 |
| 11 | 100657 | M5 Nylock Nut Blk Zn | 1 |
| 6-11 | 003417 | Classic Hanger Latch Assembly, Left | 1 |
| 6-11 | 003416 | Classic Hanger Latch Assembly, Right | 1 |
| 12 | 100667 | M6 x 12 BTN SHCS BLKZC w/Patch | 1 |
| 13 | 100669 | M5 x 10 BTN SHCS BLKZC w/Patch | 2 |
| 14 | 101193 | Calf Pad Mount Assembly | 1 |
| 15 | 000556 | ELR Index Spacer | 1 |
| 16 | 100668 | M6 x 20 CLS10.9 BTN SHCS ZC w/Patch | 1 |
| 17 | 000073 | M5 x 24 Threaded Barrel | 1 |
| 18a | 101585 | Calf Pad | 1 |
| 18b | 101470 | Pediatric Calf Pad <i>Used on 14" wide chairs</i> | 1 |
| 19 | 000034 | Sleeve 3/4" x 1 1/2" | 1 |
| 20b | 110389 | Footrest Extension Tube ELR Long <i>Replacement for 000539</i> | 1 |
| 21 | 100638 | Tube End Plug 3/4" | 1 |
| 20b, 21, 22b, 23 | 505360 | Long ELR Ext Tube with Large Composite Footplate 17-22 Assembly <i>This kit is required if needing to replace 101584 with long extension tubes</i> | 1 |
| 22c | 003583 | Footplate ELR Composite for Heel Loop Small Right | 1 |
| 22d | 003584 | Footplate ELR Composite for Heel Loop Small Left | 1 |
| 22e | 003585 | Footplate ELR Composite for Heel Loop Large Right | 1 |
| 22f | 003586 | Footplate ELR Composite for Heel Loop Large Left | 1 |

| Item Number | Part Number | Description | Quantity |
|--------------------------|-------------|---|----------|
| 23 | 101586 | ELR Footrest Bumper | 1 |
| 24a | 110386 | ELR Extension Tube Short Angle Adjustable <i>Replacement for 000622</i> | 1 |
| 24b | 110387 | ELR Extension Tube Long Angle Adjustable <i>Replacement for 000623</i> | 1 |
| 25a | 000105 | Footplate Angle Adjustable 14-15" Wide CAT/TILT Left (Std ELR Right) | 1 |
| 25b | 000106 | Footplate Angle Adjustable 14-15" Wide CAT/TILT Right (Std ELR Left) | 1 |
| 25c | 000107 | Footplate Angle Adjustable 16-17" Wide CAT/TILT Left (Std ELR Right) | 1 |
| 25d | 000108 | Footplate Angle Adjustable 16-17" Wide CAT/TILT Right (Std ELR Left) | 1 |
| 25e | 000109 | Footplate Angle Adjustable 18-24" Wide CAT/TILT Left (Std ELR Right) | 1 |
| 25f | 000110 | Footplate Angle Adjustable 18-24" Wide CAT/TILT Right (Std ELR Left) | 1 |
| 26 | 100502 | Angle Adj Footplate Clamp 2" | 1 |
| 27 | 100662 | M6 x 25 FSHCS BLK ZC | 2 |
| 28 | 100658 | M6 Nylock Nut Blk Zn | 2 |
| 29a | 000047 | Footplate Angle Adjustable Casting 3/4 Right | 1 |
| 29b | 000049 | Footplate Angle Adjustable Casting 3/4" Left | 1 |
| 30 | 101472 | M6 x 45 SHCS BLK ZC | 1 |
| 31 | 101455 | M6 Belleville Washer (.252 x .551 x .050) | 1 |
| 32 | 000053 | Saddle 3/4" | 2 |
| 33 | 100746 | M6 Flat Washer Blk Zn | 1 |
| 34 | 101456 | M6 Ovl Top L/N ZC | 1 |
| 35 | 000428 | Wear Washer | 1 |
| 36 | 000621 | ELR Spacer Small Footplate <i>Only used with small composite footplate</i> | 1 |
| 37 | 003227INST | Hanger Latch Pin with 4-Way Release Instruction | 1 |
| 38 | 003226INST | Hanger Latch Bracket with 4-way Release Instruction | 1 |
| 39 | 003228INST | Hanger Latch, 4-Way, Lever with Instruction | 1 |
| 9, 10, 11, 37, 38, 39 | 003643INST | Assembled 4-Way Out Hanger Release, Left with Instruction | 1 |
| 9, 10, 11, 37, 38, 39 | 003642INST | Assembled 4-Way Out Hanger Release, Right with Instruction | 1 |

| Item Number | Part Number | Description | Quantity |
|-------------------------------------|-------------|---|----------|
| 1b, 2-17, 18b, 19 | 501046 | ELR Upper Assembly with Classic Hanger Latch 14 Left | 1 |
| 1a, 2-17, 18b, 19 | 501047 | ELR Upper Assembly with Classic Hanger Latch 14 Right | 1 |
| 1b, 2-17, 18a, 19 | 500615L | ELR Upper Assembly with Classic Hanger Latch Left | 1 |
| 1a, 2-17, 18a, 19 | 500615R | ELR Upper Assembly with Classic Hanger Latch Right | 1 |
| 1d, 2-17, 18b, 19 | 501601 | Extended Length ELR Upper Assembly with Classic Hanger Latch 14 Left | 1 |
| 1c, 2-17, 18b, 19 | 501602 | Extended Length ELR Upper Assembly with Classic Hanger Latch 14 Right | 1 |
| 1d, 2-17, 18a, 19 | 501603 | Extended Length ELR Upper Assembly with Classic Hanger Latch Left | 1 |
| 1c, 2-17, 18a, 19 | 501604 | Extended Length ELR Upper Assembly with Classic Hanger Latch Right | 1 |
| 1b, 2-5, 9-17, 18b, 19, 37-39 | 505307 | ELR Upper Assembly with 4-Way Out Hanger Latch 14 Left | 1 |
| 1a, 2-5, 9-17, 18b, 19, 37-39 | 505308 | ELR Upper Assembly with 4-Way Out Hanger Latch 14 Right | 1 |
| 1b, 2-5, 9-17, 18a, 19, 37-39 | 505309 | ELR Upper Assembly with 4-Way Out Hanger Latch Left | 1 |
| 1a, 2-5, 9-17, 18a, 19, 37-39 | 505310 | ELR Upper Assembly with 4-Way Out Hanger Latch Right | 1 |
| 1d, 2-5, 9-17, 18b, 19, 37-39 | 505311 | Extended Length ELR Upper Assembly with 4-Way Out Hanger Latch 14 Left | 1 |
| 1c, 2-5, 9-17, 18b, 19, 37-39 | 505312 | Extended Length ELR Upper Assembly with 4-Way Out Hanger Latch 14 Right | 1 |
| 1d, 2-5, 9-17, 18a, 19, 37-39 | 505313 | Extended Length ELR Upper Assembly with 4-Way Out Hanger Latch Left | 1 |
| 1c, 2-5, 9-17, 18a, 19, 37-39 | 505314 | Extended Length ELR Upper Assembly with 4-Way Out Hanger Latch Right | 1 |
| 21, 24a, 25b, 26 - 28, 29b, 30 - 35 | 505331 | Short ELR Ext Tube with Angle Adjustable Footplate 14/15 Left Assembly | 1 |
| 21, 24a, 25a, 26 - 28, 29a, 30 - 35 | 505332 | Short ELR Ext Tube with Angle Adjustable Footplate 14/15 Right Assembly | 1 |
| 21, 24a, 25a, 25b, 26 - 35 | 505333 | Short ELR Ext Tube with Angle Adjustable Footplate 14/15 Assembly PAIR | 1 |
| 21, 24a, 25b, 26 - 28, 29b, 30 - 35 | 505334 | Short ELR Ext Tube with Angle Adjustable Footplate 14/15 with Heel Loop Left Assembly | 1 |
| 21, 24a, 25a, 26 - 28, 29a, 30 - 35 | 505335 | Short ELR Ext Tube with Angle Adjustable Footplate 14/15 with Heel Loop Right Assembly | 1 |
| 21, 24a, 25a, 25b, 26 - 35 | 505336 | Short ELR Ext Tube with Angle Adjustable Footplate 14/15 with Heel Loop Assembly PAIR | 1 |
| 21, 24a, 25b, 26 - 28, 29b, 30 - 35 | 505337 | Short ELR Ext Tube with Angle Adjustable Footplate 14/15 with Adjustable Heel Loop Left Assembly | 1 |
| 21, 24a, 25a, 26 - 28, 29a, 30 - 35 | 505338 | Short ELR Ext Tube with Angle Adjustable Footplate 14/15 with Adjustable Heel Loop Right Assembly | 1 |

| Item Number | Part Number | Description | Quantity |
|-------------------------------------|-------------|---|----------|
| 21, 24a, 25a, 25b, 26 - 35 | 505339 | Short ELR Ext Tube with Angle Adjustable Footplate 14/15 with Adjustable Heel Loop Assembly PAIR | 1 |
| 21, 24a, 25d, 26 - 28, 29b, 30 - 35 | 505340 | Short ELR Ext Tube with Angle Adjustable Footplate 16/17 Left Assembly | 1 |
| 21, 24a, 25c, 26 - 28, 29a, 30 - 35 | 505341 | Short ELR Ext Tube with Angle Adjustable Footplate 16/17 Right Assembly | 1 |
| 21, 24a, 25c, 25d, 26 - 35 | 505342 | Short ELR Ext Tube with Angle Adjustable Footplate 16/17 Assembly PAIR | 1 |
| 21, 24a, 25d, 26 - 28, 29b, 30 - 35 | 505343 | Short ELR Ext Tube with Angle Adjustable Footplate 16/17 with Heel Loop Left Assembly | 1 |
| 21, 24a, 25c, 26 - 28, 29a, 30 - 35 | 505344 | Short ELR Ext Tube with Angle Adjustable Footplate 16/17 with Heel Loop Right Assembly | 1 |
| 21, 24a, 25c, 25d, 26 - 35 | 505345 | Short ELR Ext Tube with Angle Adjustable Footplate 16/17 with Heel Loop Assembly PAIR | 1 |
| 21, 24a, 25d, 26 - 28, 29b, 30 - 35 | 505346 | Short ELR Ext Tube with Angle Adjustable Footplate 16/17 with Adjustable Heel Loop Left Assembly | 1 |
| 21, 24a, 25c, 26 - 28, 29a, 30 - 35 | 505347 | Short ELR Ext Tube with Angle Adjustable Footplate 16/17 with Adjustable Heel Loop Right Assembly | 1 |
| 21, 24a, 25c, 25d, 26 - 35 | 505348 | Short ELR Ext Tube with Angle Adjustable Footplate 16/17 with Adjustable Heel Loop Assembly PAIR | 1 |
| 21, 24a, 25f, 26 - 28, 29b, 30 - 35 | 505349 | Short ELR Ext Tube with Angle Adjustable Footplate 18-24 Left Assembly | 1 |
| 21, 24a, 25e, 26 - 28, 29a, 30 - 35 | 505350 | Short ELR Ext Tube with Angle Adjustable Footplate 18-24 Right Assembly | 1 |
| 21, 24a, 25e, 25f, 26 - 35 | 505351 | Short ELR Ext Tube with Angle Adjustable Footplate 18-24 Assembly PAIR | 1 |
| 21, 24a, 25f, 26 - 28, 29b, 30 - 35 | 505352 | Short ELR Ext Tube with Angle Adjustable Footplate 18-24 with Heel Loop Left Assembly | 1 |
| 21, 24a, 25e, 26 - 28, 29a, 30 - 35 | 505353 | Short ELR Ext Tube with Angle Adjustable Footplate 18-24 with Heel Loop Right Assembly | 1 |
| 21, 24a, 25e, 25f, 26 - 35 | 505354 | Short ELR Ext Tube with Angle Adjustable Footplate 18-24 with Heel Loop Assembly PAIR | 1 |
| 21, 24a, 25f, 26 - 28, 29b, 30 - 35 | 505355 | Short ELR Ext Tube with Angle Adjustable Footplate 18-24 with Adjustable Heel Loop Left Assembly | 1 |
| 21, 24a, 25e, 26 - 28, 29a, 30 - 35 | 505356 | Short ELR Ext Tube with Angle Adjustable Footplate 18-24 with Adjustable Heel Loop Right Assembly | 1 |
| 21, 24a, 25e, 25f, 26 - 35 | 505357 | Short ELR Ext Tube with Angle Adjustable Footplate 18-24 with Adjustable Heel Loop Assembly PAIR | 1 |
| 20b, 21, 22b, 23 | 505360 | Long ELR Ext Tube with Large Composite Footplate 17-22 Assembly | 1 |

| Item Number | Part Number | Description | Quantity |
|-------------------------------------|-------------|--|----------|
| 20b, 21, 22b, 23 | 505361 | Long ELR Ext Tube with Large Composite Footplate 17-22 Assembly PAIR | 1 |
| 20b, 21, 22d, 23, 36 | 505362 | Long ELR Ext Tube with Small Composite Footplate 14-16 with Heel Loop Left Assembly | 1 |
| 20b, 21, 22c, 23, 36 | 505363 | Long ELR Ext Tube with Small Composite Footplate 14-16 with Heel Loop Right Assembly | 1 |
| 20b, 21, 22c, 22d, 23, 36 | 505364 | Long ELR Ext Tube with Small Composite Footplate 14-16 with Heel Loop Assembly PAIR | 1 |
| 20b, 21, 22d, 23, 36 | 505365 | Long ELR Ext Tube with Small Composite Footplate 14-16 with Adjustable Heel Loop Left Assembly | 1 |
| 20b, 21, 22c, 23, 36 | 505366 | Long ELR Ext Tube with Small Composite Footplate 14-16 with Adjustable Heel Loop Right Assembly | 1 |
| 20b, 21, 22c, 22d, 23, 36 | 505367 | Long ELR Ext Tube with Small Composite Footplate 14-16 with Adjustable Heel Loop Assembly PAIR | 1 |
| 20b, 21, 22f, 23 | 505368 | Long ELR Ext Tube with Large Composite Footplate 17-22 with Heel Loop Left Assembly | 1 |
| 20b, 21, 22e, 23 | 505369 | Long ELR Ext Tube with Large Composite Footplate 17-22 with Heel Loop Right Assembly | 1 |
| 20b, 21, 22e, 22f, 23 | 505370 | Long ELR Ext Tube with Large Composite Footplate 17-22 with Heel Loop Assembly PAIR | 1 |
| 20b, 21, 22f, 23 | 505371 | Long ELR Ext Tube with Large Composite Footplate 17-22 with Adjustable Heel Loop Left Assembly | 1 |
| 20b, 21, 22e, 23 | 505372 | Long ELR Ext Tube with Large Composite Footplate 17-22 with Adjustable Heel Loop Right Assembly | 1 |
| 20b, 21, 22e, 22f, 23 | 505373 | Long ELR Ext Tube with Large Composite Footplate 17-22 with Adjustable Heel Loop Assembly PAIR | 1 |
| 21, 24b, 25b, 26 - 28, 29b, 30 - 35 | 505374 | Long ELR Ext Tube with Angle Adjustable Footplate 14/15 Left Assembly | 1 |
| 21, 24b, 25a, 26 - 28, 29a, 30 - 35 | 505375 | Long ELR Ext Tube with Angle Adjustable Footplate 14/15 Right Assembly | 1 |
| 21, 24b, 25a, 25b, 26 - 35 | 505376 | Long ELR Ext Tube with Angle Adjustable Footplate 14/15 Assembly PAIR | 1 |
| 21, 24b, 25b, 26 - 28, 29b, 30 - 35 | 505377 | Long ELR Ext Tube with Angle Adjustable Footplate 14/15 with Heel Loop Left Assembly | 1 |
| 21, 24b, 25a, 26 - 28, 29a, 30 - 35 | 505378 | Long ELR Ext Tube with Angle Adjustable Footplate 14/15 with Heel Loop Right Assembly | 1 |
| 21, 24b, 25a, 25b, 26 - 35 | 505379 | Long ELR Ext Tube with Angle Adjustable Footplate 14/15 with Heel Loop Assembly PAIR | 1 |
| 21, 24b, 25b, 26 - 28, 29b, 30 - 35 | 505380 | Long ELR Ext Tube with Angle Adjustable Footplate 14/15 with Adjustable Heel Loop Left Assembly | 1 |
| 21, 24b, 25a, 26 - 28, 29a, 30 - 35 | 505381 | Long ELR Ext Tube with Angle Adjustable Footplate 14/15 with Adjustable Heel Loop Right Assembly | 1 |

| Item Number | Part Number | Description | Quantity |
|-------------------------------------|-------------|--|----------|
| 21, 24b, 25a, 25b, 26 - 35 | 505382 | Long ELR Ext Tube with Angle Adjustable Footplate 14/15 with Adjustable Heel Loop Assembly PAIR | 1 |
| 21, 24b, 25d, 26 - 28, 29b, 30 - 35 | 505383 | Long ELR Ext Tube with Angle Adjustable Footplate 16/17 Left Assembly | 1 |
| 21, 24b, 25c, 26 - 28, 29a, 30 - 35 | 505384 | Long ELR Ext Tube with Angle Adjustable Footplate 16/17 Right Assembly | 1 |
| 21, 24b, 25c, 25d, 26 - 35 | 505385 | Long ELR Ext Tube with Angle Adjustable Footplate 16/17 Assembly PAIR | 1 |
| 21, 24b, 25d, 26 - 28, 29b, 30 - 35 | 505386 | Long ELR Ext Tube with Angle Adjustable Footplate 16/17 with Heel Loop Left Assembly | 1 |
| 21, 24b, 25c, 26 - 28, 29a, 30 - 35 | 505387 | Long ELR Ext Tube with Angle Adjustable Footplate 16/17 with Heel Loop Right Assembly | 1 |
| 21, 24b, 25c, 25d, 26 - 35 | 505388 | Long ELR Ext Tube with Angle Adjustable Footplate 16/17 with Heel Loop Assembly PAIR | 1 |
| 21, 24b, 25d, 26 - 28, 29b, 30 - 35 | 505389 | Long ELR Ext Tube with Angle Adjustable Footplate 16/17 with Adjustable Heel Loop Left Assembly | 1 |
| 21, 24b, 25c, 26 - 28, 29a, 30 - 35 | 505390 | Long ELR Ext Tube with Angle Adjustable Footplate 16/17 with Adjustable Heel Loop Right Assembly | 1 |
| 21, 24b, 25c, 25d, 26 - 35 | 505391 | Long ELR Ext Tube with Angle Adjustable Footplate 16/17 with Adjustable Heel Loop Assembly PAIR | 1 |
| 21, 24b, 25f, 26 - 28, 29b, 30 - 35 | 505392 | Long ELR Ext Tube with Angle Adjustable Footplate 18-24 Left Assembly | 1 |
| 21, 24b, 25e, 26 - 28, 29a, 30 - 35 | 505393 | Long ELR Ext Tube with Angle Adjustable Footplate 18-24 Right Assembly | 1 |
| 21, 24b, 25e, 25f, 26 - 35 | 505394 | Long ELR Ext Tube with Angle Adjustable Footplate 18-24 Assembly PAIR | 1 |
| 21, 24b, 25f, 26 - 28, 29b, 30 - 35 | 505395 | Long ELR Ext Tube with Angle Adjustable Footplate 18-24 with Heel Loop Left Assembly | 1 |
| 21, 24b, 25e, 26 - 28, 29a, 30 - 35 | 505396 | Long ELR Ext Tube with Angle Adjustable Footplate 18-24 with Heel Loop Right Assembly | 1 |
| 21, 24b, 25e, 25f, 26 - 35 | 505397 | Long ELR Ext Tube with Angle Adjustable Footplate 18-24 with Heel Loop Assembly PAIR | 1 |
| 21, 24b, 25f, 26 - 28, 29b, 30 - 35 | 505398 | Long ELR Ext Tube with Angle Adjustable Footplate 18-24 with Adjustable Heel Loop Left Assembly | 1 |
| 21, 24b, 25e, 26 - 28, 29a, 30 - 35 | 505399 | Long ELR Ext Tube with Angle Adjustable Footplate 18-24 with Adjustable Heel Loop Right Assembly | 1 |
| 21, 24b, 25e, 25f, 26 - 35 | 505400 | Long ELR Ext Tube with Angle Adjustable Footplate 18-24 with Adjustable Heel Loop Assembly PAIR | 1 |