


Catalyst 4 Armrests - Tall Height Adjustable Flip Back T-Arm

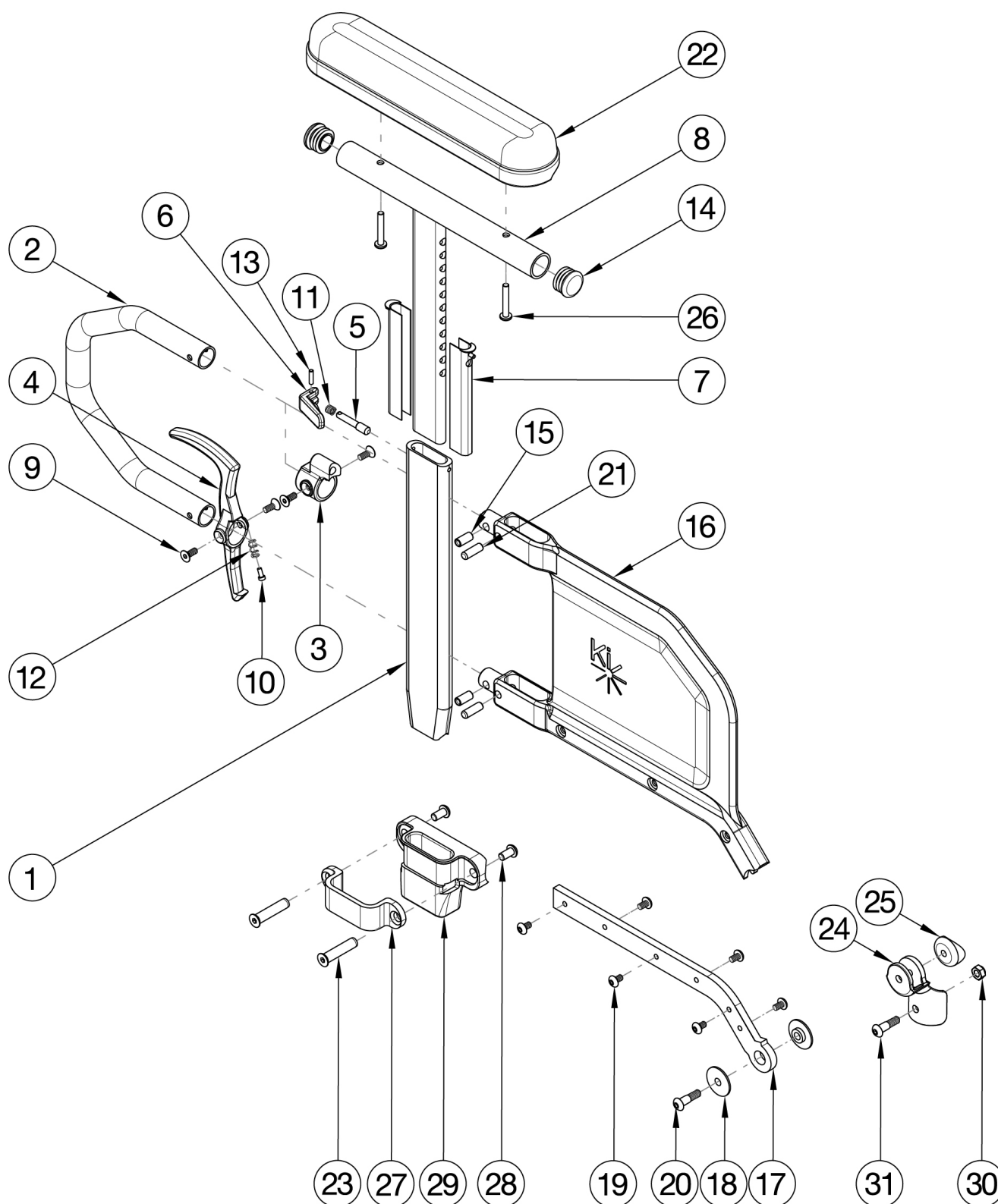

Receiver and Hardware update effective 11/25/24 Short Side Shield used with 14" - 17" Seat Depths. Long Side Shield used with 18" - 20" Seat Depths. If retrofiting, Receivers are needed.

Item Number	Part Number	Description	Quantity
1-7, 8a, 9-15, 16a, 17a, 18-21, 22b, 23-31	116230	Height Adjustable Tall Flip Back T-Arm Short Desk with Standard Pad and Receiver Assembly, Catalyst 4	1
1-7, 8b, 9-15, 16a, 17a, 18-21, 22a, 23-31	116231	Height Adjustable Tall Flip Back T-Arm Short Full with Standard Pad and Receiver Assembly, Catalyst 4	1
1-7, 8a, 9-15, 16b, 17b, 18-21, 22b, 23-31	116232	Height Adjustable Tall Flip Back T-Arm Long Desk with Standard Pad and Receiver Assembly, Catalyst 4	1
1-7, 8b, 9-15, 16b, 17b, 18-21, 22a, 23-31	116233	Height Adjustable Tall Flip Back T-Arm Long Full with Standard Pad and Receiver Assembly, Catalyst 4	1
1-7, 8a, 9-15, 16a, 17a, 18-21, 22b, 24-26, 30, 31	116226	Height Adjustable Tall Flip Back T-Arm Short Desk with Standard Pad Assembly, Catalyst 4	1
1-7, 8b, 9-15, 16a, 17a, 18-21, 22a, 24-26, 30, 31	116227	Height Adjustable Tall Flip Back T-Arm Short Full with Standard Pad Assembly, Catalyst 4	1
1-7, 8a, 9-15, 16b, 17b, 18-21, 22b, 24-26, 30, 31	116228	Height Adjustable Tall Flip Back T-Arm Long Desk with Standard Pad Assembly, Catalyst 4	1
1-7, 8b, 9-15, 16b, 17b, 18-21, 22a, 24-26, 30, 31	116229	Height Adjustable Tall Flip Back T-Arm Long Full with Standard Pad Assembly, Catalyst 4	1
1-7, 9-13, 15, 16a, 17a, 19, 21	114152	Height Adjustable Flip Back T-Arm Short Lower Assy, Tall <i>Does not include t-arm upper, arm pad or arm pad screws</i>	1
1-7, 9-13, 15, 16b, 17b, 19, 21	114153	Height Adjustable Flip Back T-Arm Long Lower Assy, Tall <i>Does not include t-arm upper, arm pad or arm pad screws</i>	1
1-7, 8a, 9-15, 16a, 17a, 19, 21	114436	Height Adjustable Flip Back T-Arm Desk, Short, Tall <i>Does not include arm pad or arm pad screws</i>	1
1-7, 8b, 9-15, 16a, 17a, 19, 21	114437	Height Adjustable Flip Back T-Arm Short, Full, Tall <i>Does not include arm pad or arm pad screws</i>	1
1-7, 8a, 9-15, 16b, 17b, 19, 21	114438	Height Adjustable Flip Back T-Arm Long, Desk, Tall <i>Does not include arm pad or arm pad screws</i>	1
1-7, 8b, 9-15, 16b, 17b, 19, 21	114439	Height Adjustable Flip Back T-Arm Long, Full, Tall <i>Does not include arm pad or arm pad screws</i>	1
1-7, 9-13, 15, 16a, 17a, 18-21, 24, 25	505617	Height Adjustable Flip Back T-Arm Short Retrofit Assembly, Catalyst 4	1

Item Number	Part Number	Description	Quantity
		<i>This kit is used to retrofit from a Height Adjustable T-Arm to a Height Adjustable Flip Back Tall T-Arm. This kit is to retrofit one side.</i>	
1-7, 9-13, 15, 16b, 17b, 18-21, 24, 25	505618	Height Adjustable Flip Back T-Arm Long Retrofit Assembly, Catalyst 4 <i>This kit is used to retrofit from a Height Adjustable T-Arm to a Height Adjustable Flip Back Tall T-Arm. This kit is to retrofit one side.</i>	1
18, 20, 24, 25	112957	Armrest Flip Back Height Adj T-Arm Hinge Assembly, PAIR, Catalyst 4 <i>This kit is for a pair of hinge assemblies. A pair is required as a replacement as of May 19th, 2021</i>	1
3, 5, 6, 9, 11, 13	504560	Height Adjustable Flip Back T-Arm Latch Assembly	1
4, 9, 10, 12	504561	Height Adjustable Flip Back T-Arm Release Lever Assembly	1
23, 27, 28, 29	501730	Height Adj T-Arm Receiver Assm 1" CAT	1
30, 31	504577	Height Adjustable Flip Back T-Arm Mounting Hardware Assembly <i>This kit contains various mounting hardware for multiple configurations</i>	1
1	115983	Height Adjustable Flip Back T-Arm Post Tall	1
2	113138	Height Adjustable T-Arm Transfer Tube, Tall	1
3	000096	Height Adjustable T Armrest Pin Lock Housing	1
4	003419	Armrest Release Lever	1
5	000099	Height Adjustable Armrest Lever Pin	1
6	003418	Armrest Height Adjustment Lever	1
7	000090	Height Adjustable T Armrest Shim	2
8a	002648	Post Weldment, T-Arm, Low, Desk	1
8b	002649	Post Weldment, T-Arm, Low, Full	1
8a, 14	112906	Height Adjustable T Armrest Upper Desk w/ Tube End Plug	1
8b, 14	112907	Height Adjustable T Armrest Upper Full w/ Tube End Plug	1
9	103522	M5 x 0.8 x 12 FHCS CLS 10.9 Nyl Blk Zn DIN7991	2
10	000152	Height Adjustable T Armrest Release Spring Button	1
11	000134	Height Adjustable Armrest Lever Spring	1
12	000135	Height Adjustable T Armrest Release Lever Spring	1
13	200257	Dowel Pin 1/8 x 1/2	1
14	100803	Tube End Plug 7/8"	2

Item Number	Part Number	Description	Quantity
15, 16a, 21	113143	Height Adjustable Flip Back T-Arm Short Shield, Tall	1
15, 16b, 21	113142	Height Adjustable Flip Back T-Arm Long Side Shield, Tall	1
17a, 19	504564	Height Adjustable Flip Back T-Arm Metal Insert Short	1
17b, 19	504565	Height Adjustable Flip Back T-Arm Metal Insert Long	1
22a	100500	Arm Pad Full	1
22b	100501	Arm Pad Desk	1
23	122055	Binding Post, Flat Head, M6, .38"	2
26	100696	M5 x 30mm PPH Capscrew	2
27	122222	Height Adjustable T Armrest Bracket, CAT/SPARK	1
28	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2
29	122221	T-Arm Receiver 1", Cat	1
30	100658	M6 Nylock Nut Blk Zn	2
31	100544	M6 x 75 CLS 8.8 DIN931 HHCS ZC	1