


Catalyst 5 Cross Braces - Open Seating (Seating System)

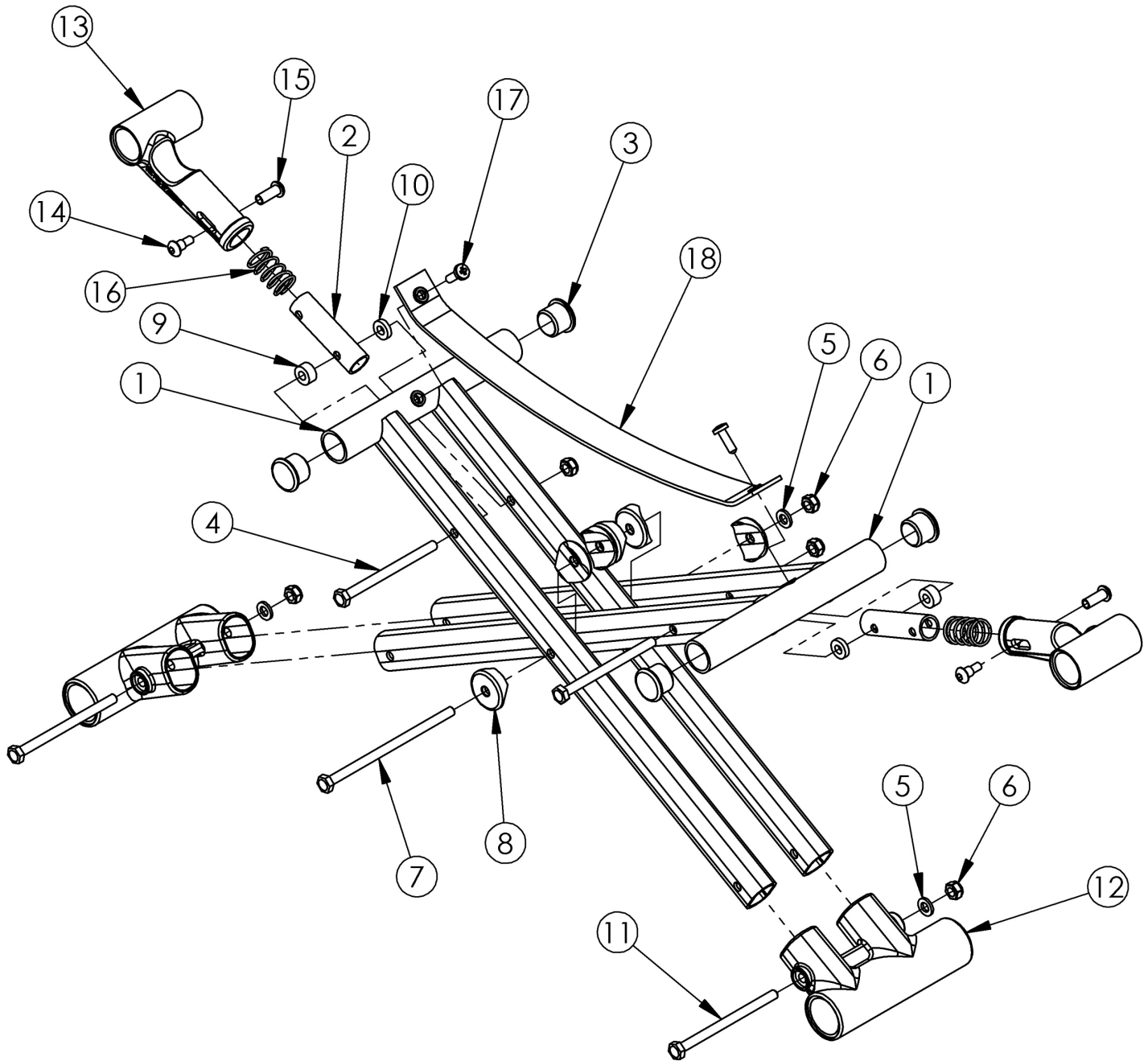

Pictured oval-style cross braces effective 6/15/16 on 14" - 22" wide chairs starting with serial numbers C5032544, C5037980, C5HD002834 and CVX0091137.

<p>

When changing from round-style to oval-style cross braces, lower frame pivots and cross brace hardware are required. Order kit 502177.

Item Number	Part Number	Description	Quantity	Retail
2a, 13a, 14-16	500024	Upper Pivot Assembly 14" Width	2	\$44.77

Item Number	Part Number	Description	Quantity	Retail
2b, 13b, 14-16	500025	Upper Pivot Assembly 15" Width	2	\$44.77
2c, 13b, 14-16	500026	Upper Pivot Assembly 16" Width	2	\$44.77
2d, 13b, 14-16	500027	Upper Pivot Assembly 17" Width	2	\$44.77
2e, 13b, 14-16	500028	Upper Pivot Assembly 18" Width	2	\$44.77
2f, 13b, 14-16	500029	Upper Pivot Assembly 19" Width	2	\$44.77
2g, 13b, 14-16	500030	Upper Pivot Assembly 20" Width	2	\$44.77
2h, 13b, 14-16	501479	Upper Pivot Assembly 21/22" Width	2	\$44.77
4-8, 11, 12	107296	Lower Frame Pivot and Catalyst Oval-Style Cross Brace Hardware Kit 14" Wide (Pair)	1	\$37.55
4-12	502177	Lower Frame Pivot and Catalyst Oval-Style Cross Brace Hardware Kit (Pair)	1	\$44.97
1a, 3	003353	Seating System Cross Brace 14" Wide CAT	1	\$150.57
1b, 3	003354	Seating System Cross Brace 15" Wide CAT	1	\$150.57
1c, 3	003355	Seating System Cross Brace 16" Wide CAT	1	\$150.57
1d, 3	003356	Seating System Cross Brace 17" Wide CAT	1	\$150.57
1f, 3	003357	Seating System Cross Brace 18" Wide CAT	1	\$150.57
1g, 3	003358	Seating System Cross Brace 19" Wide CAT	1	\$150.57
1h, 3	003359	Seating System Cross Brace 20" Wide CAT	1	\$150.57
1i, 3	003360	Seating System Cross Brace 21" Wide CAT	1	\$150.57
1j, 3	003361	Seating System Cross Brace 22" Wide CAT	1	\$150.57
3	101162	7/8 OD End Plug BLK	2	\$1.71
4	100660	M6 x 80mm CLS8.8 DIN931 HHCS ZC	2	\$1.58
5	100746	M6 Flat Washer Blk Zn	3	\$1.58
6	100658	M6 Nylock Nut Blk Zn	5	\$1.05
7	100678	M6 x 110 CLS8.8 DIN931 HHCS ZC	1	\$2.63
8	002807	Cross Tube Saddle CAT	6	\$1.58
9	100880	1/4" Plastic Spacer <i>Not used on 14" wide chairs</i>	2	\$2.63
10	105019	3/16" Plastic Spacer <i>Not used on 14" wide chairs</i>	2	\$0.53

Item Number	Part Number	Description	Quantity	Retail
11	100704	M6 x 90 CLS8.8 DIN931 HHCS ZC	2	\$1.58
4-11	503320	Catalyst Cross Brace Hardware Kit	1	\$23.96
12	002806	Lower Frame Pivot CAT	1	\$10.51
17	100734	M5 x 16 PHMS ZC	2	\$1.58
18a	200233	Seat Strap Open Frame 14" Wide CAT	1	\$6.30
18b	200234	Seat Strap Open Frame 15" Wide CAT	1	\$6.30
18c	200235	Seat Strap Open Frame 16/21" Wide CAT	1	\$6.30
18d	200236	Seat Strap Open Frame 17/22" Wide CAT	1	\$6.30
18e	200237	Seat Strap Open Frame 18" Wide CAT	1	\$6.30
18f	200238	Seat Strap Open Frame 19" Wide CAT	1	\$6.30
18g	200239	Seat Strap Open Frame 20" Wide CAT	1	\$6.30