
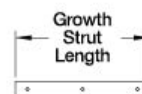


CR45 Rotary Frame - Growth

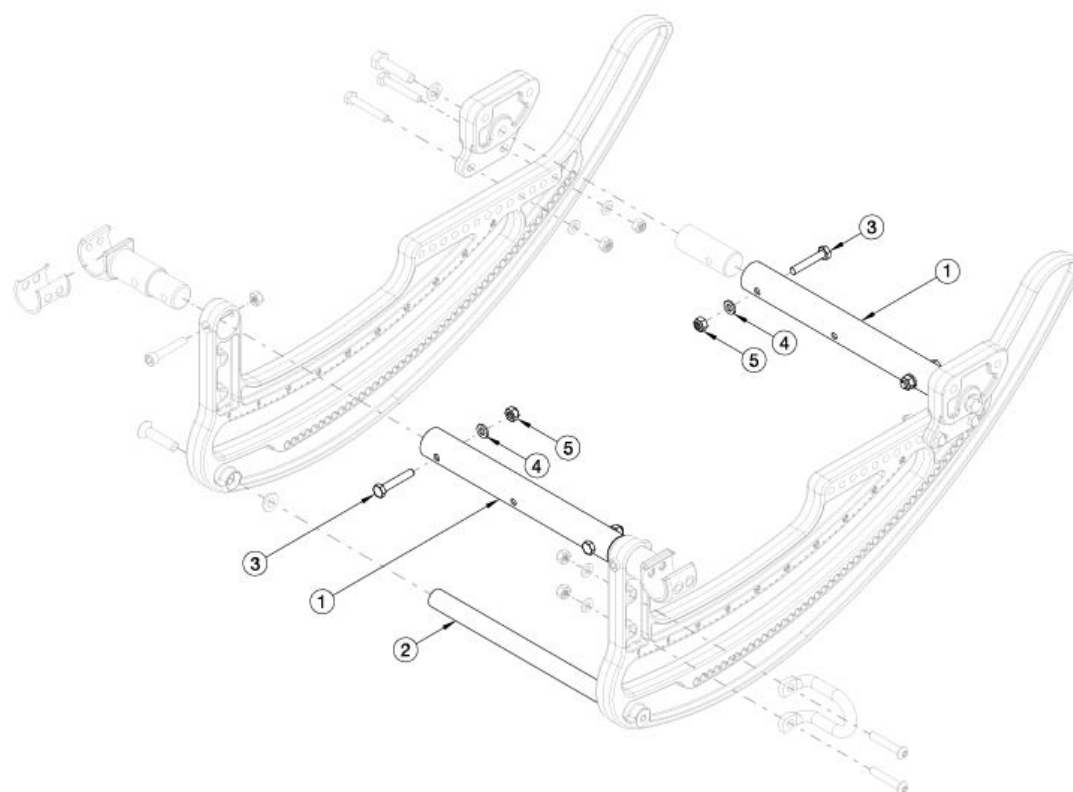

Chair Width	Strut Length
14"	8.46"
15"	9.46"
16"	10.46"
17"	11.46"
18"	12.46"
19"	13.46"
20"	14.46"
21"	15.46"
22"	16.46"

If quoting a Left Frame for Australia or New Zealand, 122921 is required when transit is not present.

Item Number	Part Number	Description	Quantity
1a	112450	Frame Strut Tube 14W	4
1b	112451	Frame Strut Tube 15W	4
1c	112452	Frame Strut Tube 16W	4
1d	112453	Frame Strut Tube 17W	4
1e	112454	Frame Strut Tube 18W	4
1f	112455	Frame Strut Tube 19W	4
1g	112456	Frame Strut Tube 20W	4
1h	112457	Frame Strut Tube 21W	4
1i	112458	Frame Strut Tube 22W	4
2a	112460	Rotary Strut 14W	2
2b	112461	Rotary Strut 15W	2

Item Number	Part Number	Description	Quantity
2c	112462	Rotary Strut 16W	2
2d	112463	Rotary Strut 17W	2
2e	112464	Rotary Strut 18W	2
2f	112465	Rotary Strut 19W	2
2h	112466	Rotary Strut 20W	2
2h	112467	Rotary Strut 21W	2
2i	112468	Rotary Strut 22W	2
3	102386	M6 x 35mm HHCS Blk Zc	4
4	100746	M6 Flat Washer Blk Zn <i>Use 12 with transit</i>	8
5	100658	M6 Nylock Nut Blk Zn <i>Use 14 with transit</i>	10