


Catalyst Titanium Cross Braces - Growth

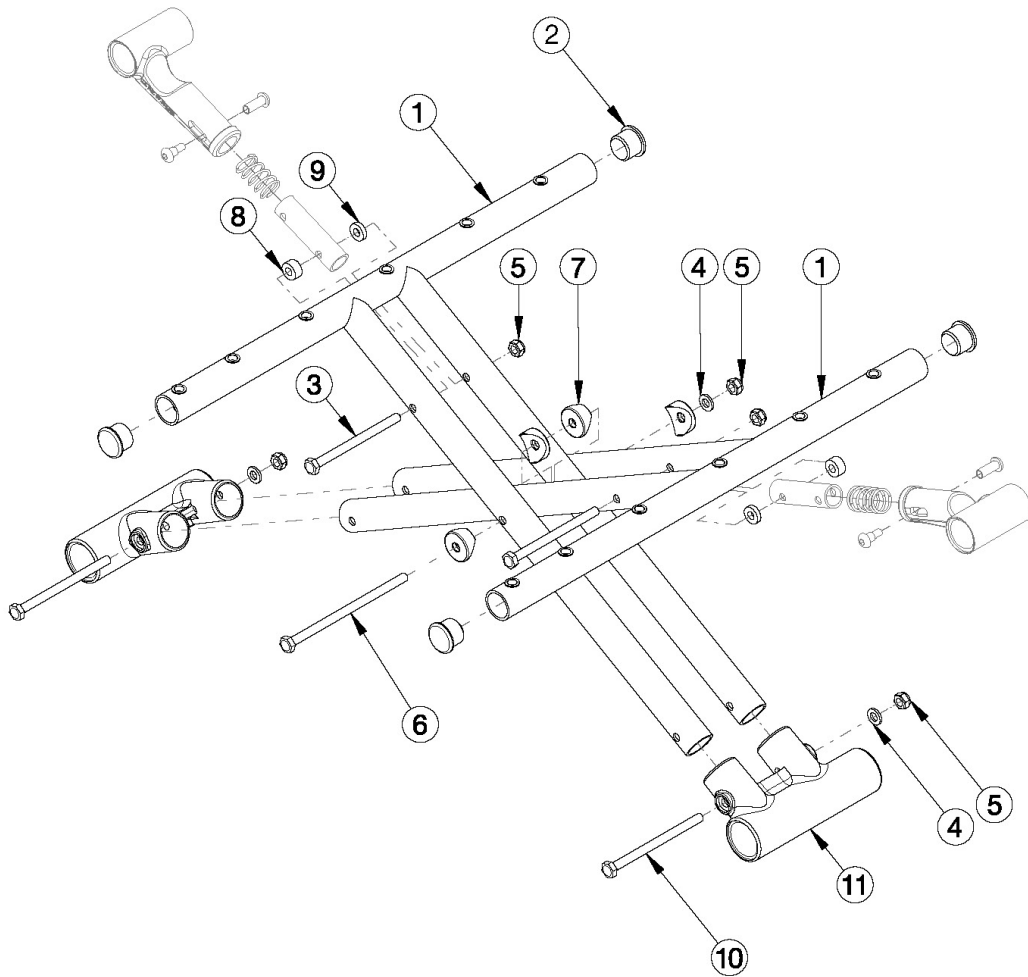

Round-style cross braces If changing from oval-style aluminum cross braces, lower frame pivots and cross brace hardware are required.

Item Number	Part Number	Description	Quantity
3-10	501398	Round-Style Cross Brace Hardware Kit CAT	1
1a, 2	100295	Cross Brace Titanium Right 14 x 14 CAT	1
1b, 2	100338	Cross Brace Titanium Left 14 x 14 CAT	1
1c, 2	100296	Cross Brace Titanium Right 14 x 15 CAT	1
1d, 2	100339	Cross Brace Titanium Left 14 x 15 CAT	1
1e, 2	100297	Cross Brace Titanium Right 14 x 16 CAT	1
1f, 2	100340	Cross Brace Titanium Left 14 x 16 CAT	1
1g, 2	100298	Cross Brace Titanium Right 14 x 17 CAT	1
1h, 2	100341	Cross Brace Titanium Left 14 x 17 CAT	1
1i, 2	100299	Cross Brace Titanium Right 14 x 18 CAT	1
1j, 2	100342	Cross Brace Titanium Left 14 x 18 CAT	1
1k, 2	100300	Cross Brace Titanium Right 15 x 14 CAT	1
1l, 2	100343	Cross Brace Titanium Left 15 x 14 CAT	1
1m, 2	100301	Cross Brace Titanium Right 15 x 15 CAT	1
1n, 2	100344	Cross Brace Titanium Left 15 x 15 CAT	1
1o, 2	100302	Cross Brace Titanium Right 15 x 16 CAT	1
1p, 2	100345	Cross Brace Titanium Left 15 x 16 CAT	1
1q, 2	100303	Cross Brace Titanium Right 15 x 17 CAT	1
1r, 2	100346	Cross Brace Titanium Left 15 x 17 CAT	1
1s, 2	100304	Cross Brace Titanium Right 15 x 18 CAT	1
1t, 2	100347	Cross Brace Titanium Left 15 x 18 CAT	1
1u, 2	100305	Cross Brace Titanium Right 15 x 19 CAT	1
1v, 2	100348	Cross Brace Titanium Left 15 x 19 CAT	1
1w, 2	100306	Cross Brace Titanium Right 15 x 20 CAT	1
1x, 2	100349	Cross Brace Titanium Left 15 x 20 CAT	1
1y, 2	100307	Cross Brace Titanium Right 16 x 14 CAT	1

Item Number	Part Number	Description	Quantity
1z, 2	100350	Cross Brace Titanium Left 16 x 14 CAT	1
1aa, 2	100308	Cross Brace Titanium Right 16 x 15 CAT	1
1ab, 2	100351	Cross Brace Titanium Left 16 x 15 CAT	1
1ac, 2	100309	Cross Brace Titanium Right 16 x 16 CAT	1
1ad, 2	100352	Cross Brace Titanium Left 16 x 16 CAT	1
1ae, 2	100310	Cross Brace Titanium Right 16 x 17 CAT	1
1af, 2	100353	Cross Brace Titanium Left 16 x 17 CAT	1
1ag, 2	100311	Cross Brace Titanium Right 16 x 18 CAT	1
1ah, 2	100354	Cross Brace Titanium Left 16 x 18 CAT	1
1ai, 2	100312	Cross Brace Titanium Right 16 x 19 CAT	1
1aj, 2	100355	Cross Brace Titanium Left 16 x 19 CAT	1
1ak, 2	100313	Cross Brace Titanium Right 16 x 20 CAT	1
1al, 2	100356	Cross Brace Titanium Left 16 x 20 CAT	1
1am, 2	100314	Cross Brace Titanium Right 17 x 14 CAT	1
1an, 2	100357	Cross Brace Titanium Left 17 x 14 CAT	1
1ao, 2	100315	Cross Brace Titanium Right 17 x 15 CAT	1
1ap, 2	100358	Cross Brace Titanium Left 17 x 15 CAT	1
1aq, 2	100316	Cross Brace Titanium Right 17 x 16 CAT	1
1ar, 2	100359	Cross Brace Titanium Left 17 x 16 CAT	1
1as, 2	100317	Cross Brace Titanium Right 17 x 17 CAT	1
1at, 2	100360	Cross Brace Titanium Left 17 x 17 CAT	1
1au, 2	100318	Cross Brace Titanium Right 17 x 18 CAT	1
1av, 2	100361	Cross Brace Titanium Left 17 x 18 CAT	1
1aw, 2	100319	Cross Brace Titanium Right 17 x 19 CAT	1
1ax, 2	100362	Cross Brace Titanium Left 17 x 19 CAT	1
1ay, 2	100320	Cross Brace Titanium Right 17 x 20 CAT	1
1az, 2	100363	Cross Brace Titanium Left 17 x 20 CAT	1
1ba, 2	100321	Cross Brace Titanium Right 18 x 14 CAT	1

Item Number	Part Number	Description	Quantity
1bb, 2	100364	Cross Brace Titanium Left 18 x 14 CAT	1
1bc, 2	100322	Cross Brace Titanium Right 18 x 15 CAT	1
1bd, 2	100365	Cross Brace Titanium Left 18 x 15 CAT	1
1be, 2	100323	Cross Brace Titanium Right 18 x 16 CAT	1
1bf, 2	100366	Cross Brace Titanium Left 18 x 16 CAT	1
1bg, 2	100324	Cross Brace Titanium Right 18 x 17 CAT	1
1bh, 2	100367	Cross Brace Titanium Left 18 x 17 CAT	1
1bi, 2	100325	Cross Brace Titanium Right 18 x 18 CAT	1
1bj, 2	100368	Cross Brace Titanium Left 18 x 18 CAT	1
1bk, 2	100326	Cross Brace Titanium Right 18 x 19 CAT	1
1bl, 2	100369	Cross Brace Titanium Left 18 x 19 CAT	1
1bm, 2	100327	Cross Brace Titanium Right 18 x 20 CAT	1
1bn, 2	100370	Cross Brace Titanium Left 18 x 20 CAT	1
1bo, 2	100328	Cross Brace Titanium Right 19 x 16 CAT	1
1bp, 2	100371	Cross Brace Titanium Left 19 x 16 CAT	1
1bq, 2	100329	Cross Brace Titanium Right 19 x 17 CAT	1
1br, 2	100372	Cross Brace Titanium Left 19 x 17 CAT	1
1bs, 2	100330	Cross Brace Titanium Right 19 x 18 CAT	1
1bt, 2	100373	Cross Brace Titanium Left 19 x 18 CAT	1
1bu, 2	100331	Cross Brace Titanium Right 19 x 19 CAT	1
1bv, 2	100374	Cross Brace Titanium Left 19 x 19 CAT	1
1bw, 2	100332	Cross Brace Titanium Right 19 x 20 CAT	1
1bx, 2	100375	Cross Brace Titanium Left 19 x 20 CAT	1
1by, 2	100333	Cross Brace Titanium Right 20 x 16 CAT	1
1bz, 2	100376	Cross Brace Titanium Left 20 x 16 CAT	1
1ca, 2	100334	Cross Brace Titanium Right 20 x 17 CAT	1
1cb, 2	100377	Cross Brace Titanium Left 20 x 17 CAT	1
1cc, 2	100335	Cross Brace Titanium Right 20 x 18 CAT	1

Item Number	Part Number	Description	Quantity
1cd, 2	100378	Cross Brace Titanium Left 20 x 18 CAT	1
1ce, 2	100336	Cross Brace Titanium Right 20 x 19 CAT	1
1cf, 2	100379	Cross Brace Titanium Left 20 x 19 CAT	1
1cg, 2	100337	Cross Brace Titanium Right 20 x 20 CAT	1
1ch, 2	100380	Cross Brace Titanium Left 20 x 20 CAT	1
2	100803	Tube End Plug 7/8"	4
3	100660	M6 x 80mm CLS8.8 DIN931 HHCS ZC	2
4	100746	M6 Flat Washer Blk Zn	1
5	100658	M6 Nylock Nut Blk Zn	3
6	100678	M6 x 110 CLS8.8 DIN931 HHCS ZC	1
7	000054	Saddle 7/8"	4
8	100880	1/4" Plastic Spacer <i>Not used on 14" wide chairs</i>	2
9	105019	3/16" Plastic Spacer <i>Not used on 14" wide chairs</i>	2
10	100704	M6 x 90 CLS8.8 DIN931 HHCS ZC	2
11	000001	Lower Frame Pivot CAT	2