


# CR45 Attendant Foot Lock

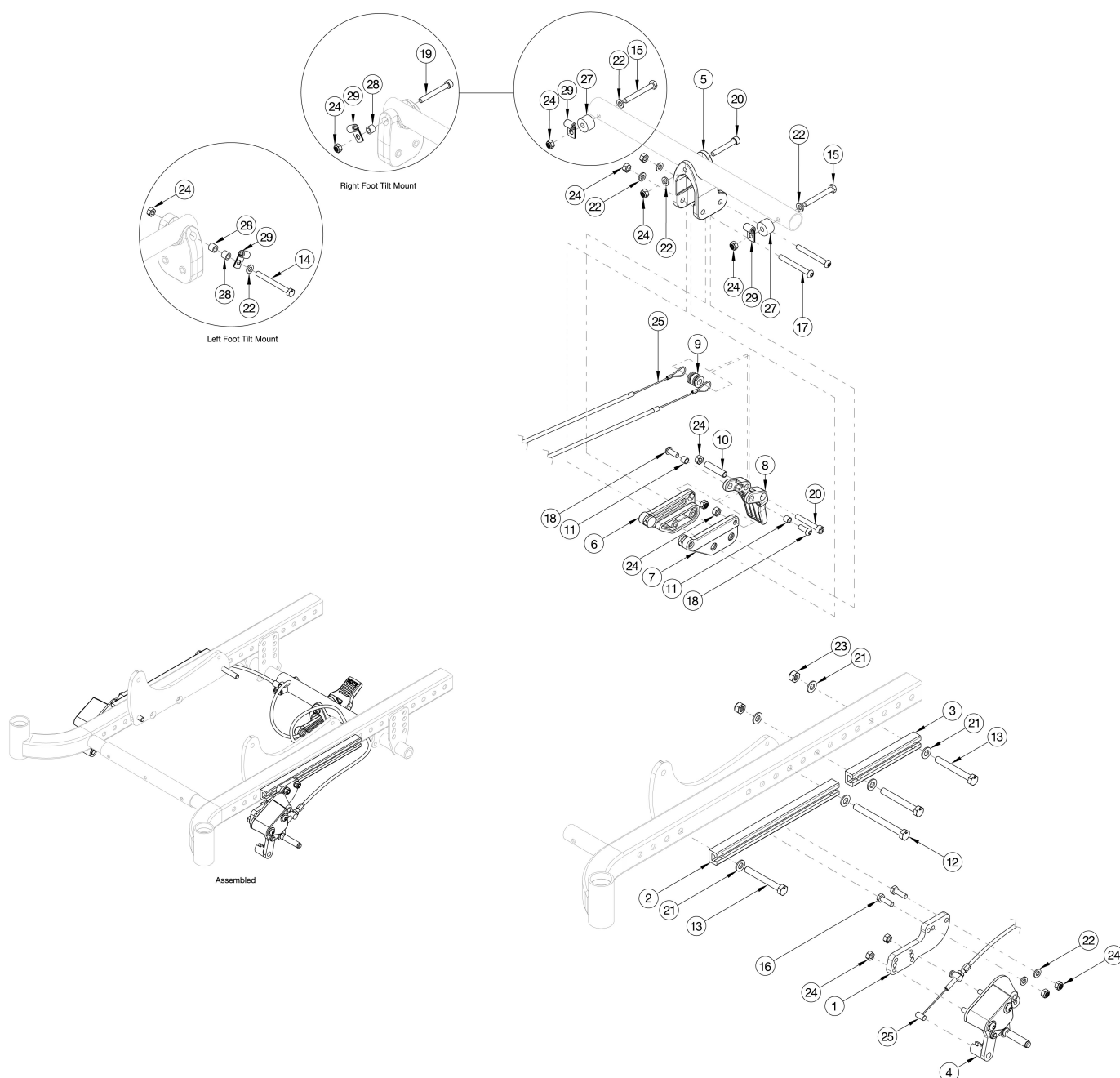

| Item Number                                  | Part Number | Description                                                                  | Quantity |
|----------------------------------------------|-------------|------------------------------------------------------------------------------|----------|
| 1-2, 4a, 4b, 5-11, 14-20, 22, 24, 25a, 26-29 | 116306      | Attendant Foot Lock, 12"-16" Wheels                                          | 1        |
| 1,4a,4b,5-11,14-21,23                        | 116302      | Attendant Foot Lock, 12"-16" Wheels w/ Reclining Backrest                    | 1        |
| 1, 4a, 4b, 5-11, 14-20, 22, 24, 25b, 26-29   | 116307      | Attendant Foot Lock, 20"-24" Wheels<br><i>Can be used with all backrests</i> | 1        |

| Item Number                         | Part Number | Description                                             | Quantity |
|-------------------------------------|-------------|---------------------------------------------------------|----------|
| 1, 4a, 16, 22, 24, 26               | 116304      | Attendant Foot Lock, L                                  | 1        |
| 1, 4b, 16, 22, 24, 26               | 116303      | Attendant Foot Lock, R                                  | 1        |
| 5-11, 17-18, 20, 22, 24, 25a, 27-28 | 116305      | Foot Lock Pedal Assy, 12"-16" Wheel                     | 1        |
| 5-11, 17-18, 20, 22, 24, 25b, 27-28 | 116330      | Foot Lock Pedal Assy, 20-24" Wheel                      | 1        |
| 1                                   | 112507      | Wheel Lock Mount                                        | 2        |
| 2                                   | 112505      | Wheel Lock Channel, CR45                                | 2        |
| 3                                   | 112506      | Short Wheel Lock Channel, CR45                          | 1        |
| 4a                                  | 111591      | Hemi Wheel Lock, Follower Assembly without Mount, Left  | 1        |
| 4b                                  | 111592      | Hemi Wheel Lock, Follower Assembly without Mount, Right | 1        |
| 5                                   | 112508      | Foot Lock Mount Bracket                                 | 1        |
| 6                                   | 111441      | Mount, Pedal, Foot Lock, R                              | 1        |
| 7                                   | 111442      | Mount, Pedal, Foot Lock, L                              | 1        |
| 8                                   | 111440      | Pedal, Foot Lock                                        | 1        |
| 9                                   | 111443      | Spool, Foot Lock                                        | 1        |
| 11                                  | 111445      | Spacer, .313ODx.243IDx.270LG, SS                        | 2        |
| 12                                  | 101832      | M8 x 90 HHCS Blk Zn                                     | 1        |
| 13                                  | 101829      | M8 x 65 HHCS Blk Zn                                     | 2        |
| 14                                  | 100677      | M6 x 60 CLS10.9 DIN931 HHCS ZC                          | 1        |
| 15                                  | 101826      | M6 x 50 HHCS Blk Zn                                     | 1        |
| 16                                  | 101823      | M6 x 20 HHCS Blk Zn                                     | 4        |
| 17                                  | 101125      | M6 x 55 BTN SHCS BLKZC                                  | 2        |
| 18                                  | 102634      | M6 X 1.0 X 16 BHCS BLK ZC                               | 2        |
| 19                                  | 102473      | M6 X 50 CLS12.9 SHCS BLK ZC                             | 1        |
| 20                                  | 101847      | M6 x 40 SHCS Blk Zn                                     | 2        |
| 21                                  | 101858      | Washer Flat M8 Blk Zn                                   | 6        |
| 22                                  | 100746      | M6 Flat Washer Blk Zn                                   | 13       |

| Item Number | Part Number | Description                       | Quantity |
|-------------|-------------|-----------------------------------|----------|
| 23          | 100659      | M8 Nylock Nut Blk Zn              | 2        |
| 24          | 100658      | M6 Nylock Nut Blk Zn              | 16       |
| 25a         | 111436      | Cable, Foot Lock, Short           | 2        |
| 25b         | 111437      | Cable, Foot Lock, Long            | 2        |
| 26          | 115743      | Hemi Wheel Lock Retaining Grommet | 2        |
| 27          | 102047      | Spacer, .25ID x .75OD x .50LG     | 1        |
| 28          | 002086      | Spacer, .38ODx.24IDx.31LG         | 1        |
| 29          | 001239      | P-Clip 3/16"                      | 2        |