


(Discontinued) Aluminum Tsunami Backrest Lockout

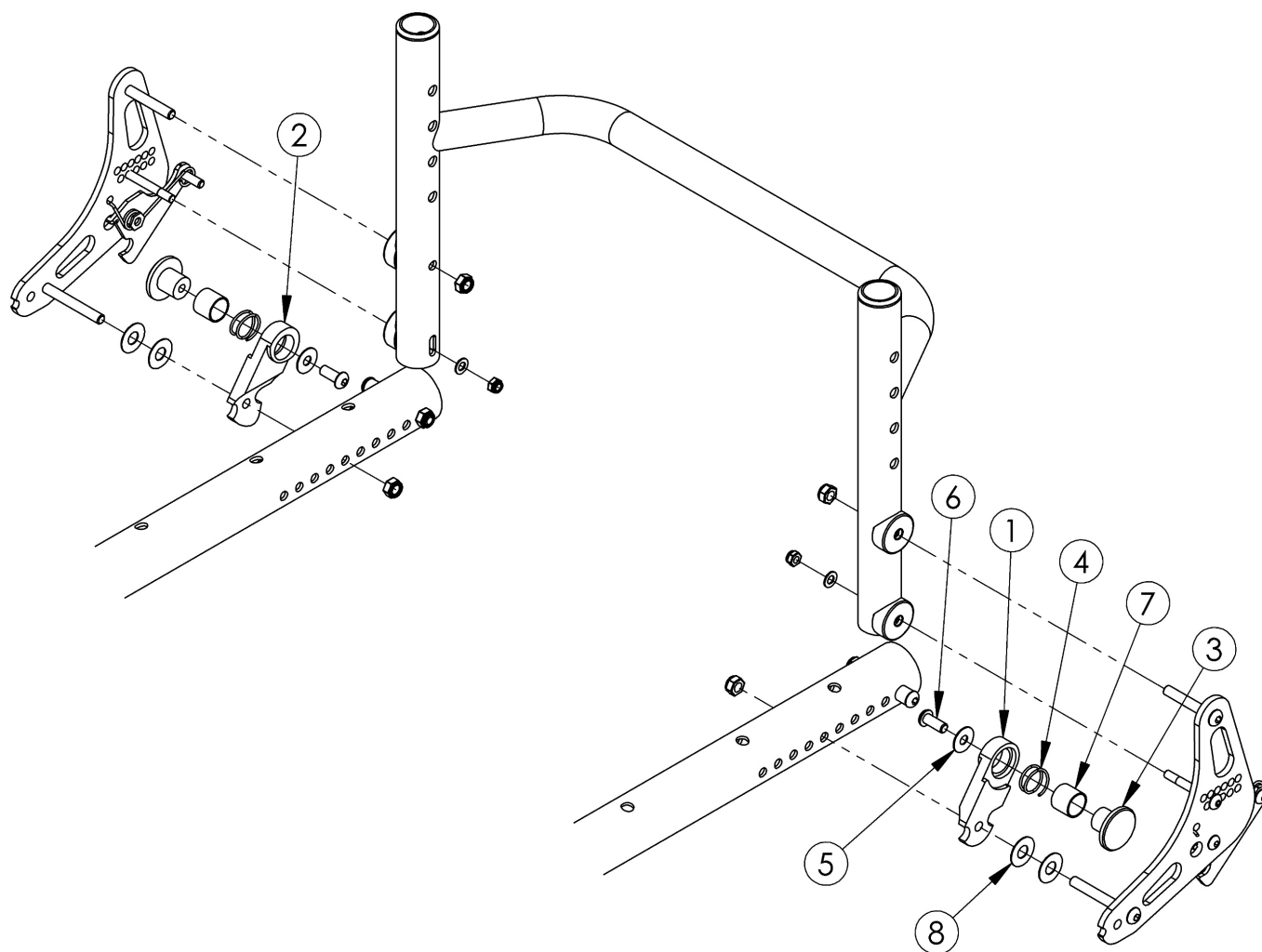

This Tsunami Backrest Lockout can only be used with the Tsunami Style Backrest

The right and left assemblies shouldn't be ordered together. The right and left assemblies indicate what side the lockout bracket is located (desired) on.

Item Number	Part Number	Description	Quantity	Retail
2 - 8	500552	Backrest Lockout Assembly Right AL TSU	1	\$92.14
1, 3 - 8	500745	Backrest Lockout Assembly Left AL TSU	1	\$92.14
1	000459	Backrest Lockout Bracket AL Left TSU/AL	1	\$73.42
2	000449	Backrest Lockout Bracket AL Right TSU/LW	1	\$73.42
3	000435	Backrest Lockout Button TSU	1	\$12.52
4	000447	Backrest Lockout Spring TSU	1	\$4.61

Item Number	Part Number	Description	Quantity	Retail
5	102629	Seastrom 5610-69-062 Washer Nylon Black	1	\$1.05
6	100669	M5 x 10 BTN SHCS BLKZC w/Patch	1	\$1.58
7	000456	Lockout Sleeve	1	\$4.42
8	000428	Wear Washer	2	\$2.63