


Rogue2 Armrest - Swing Away

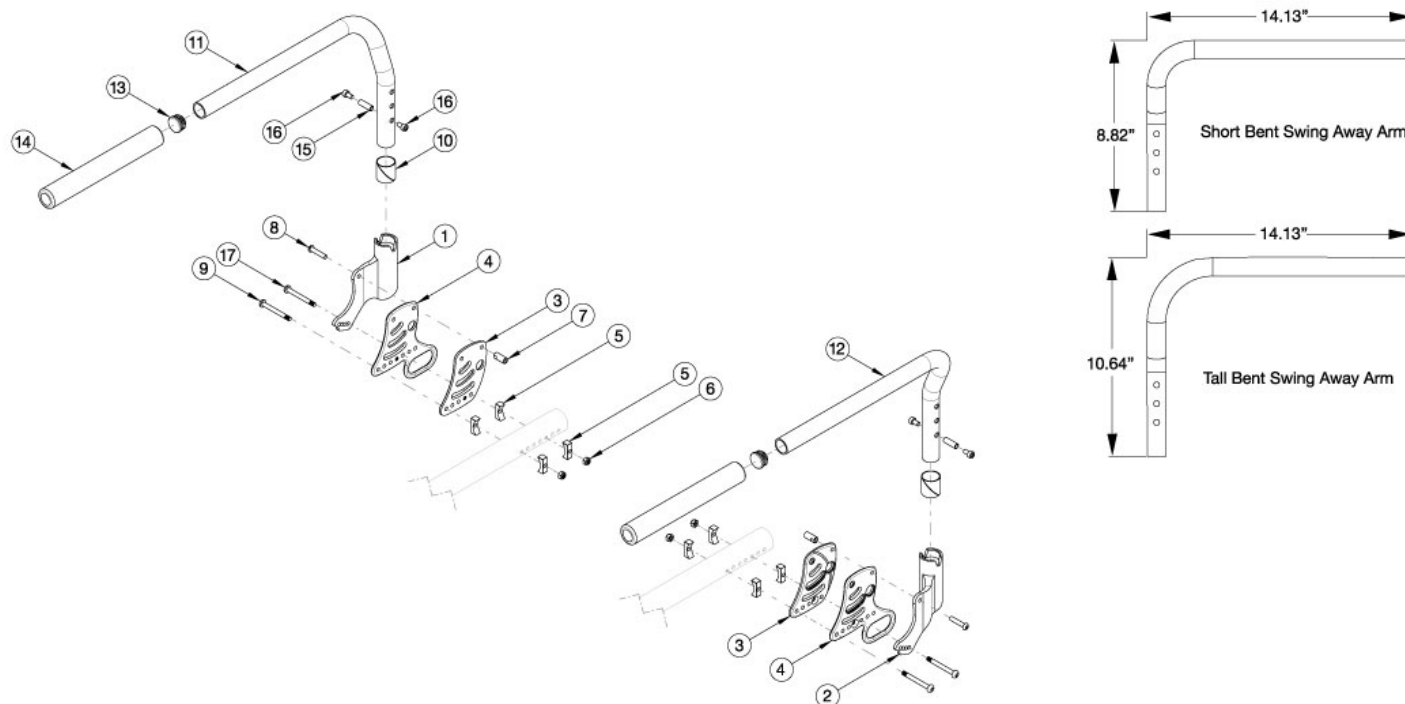

When to use Short vs. Tall Swing Away Arms: 20" Wheel: Rear Seat Height = 12.5" use Tall // Rear Seat Height = 13" or greater use Short 22" Wheel: Rear Seat Height = 13.5-14.5" use Tall // Rear Seat Height = 15" or greater use Short 24" Wheel: Rear Seat Height = 14.5-16.5" use Tall // Rear Seat Height = 17" or greater use Short 25" Wheel: Rear Seat Height = 14.5-17.7" use Tall // Rear Seat Height = 18" or greater use Short 26" Wheel: Rear Seat Height = 15.5-18.5" use Tall // Rear Seat Height = 19" or greater use Short

Item Number	Part Number	Description	Quantity
1, 6, 8 - 10, 11a, 13 - 17	117134	Short Swing Away Arm with Receiver Assembly Right, Rogue	1
2, 6, 8, 10, 12a, 13 - 17	117135	Short Swing Away Arm with Receiver Assembly Left, Rogue	1
1 - 2, 6, 8, 10, 11a, 12a, 13 - 17	117136	Short Swing Away Arm with Receiver Assembly PAIR, Rogue	1
1, 6, 8, 10, 11b, 13 - 17	117137	Tall Swing Away Arm with Receiver Assembly Right, Rogue	1
2, 6, 8, 10, 12b, 13 - 17	117138	Tall Swing Away Arm with Receiver Assembly Left, Rogue	1
1 - 2, 6, 8, 10, 11b, 12b, 13 - 17	117139	Tall Swing Away Arm with Receiver Assembly PAIR, Rogue	1
1, 6, 8, 10, 17	117140	Swing Away Arm Receiver Assembly Right, Rogue	1
2, 6, 8, 10, 17	117141	Swing Away Arm Receiver Assembly Left, Rogue	1
11a, 13 - 16	108166	Short Swing Away Arm Right, Ethos / Rogue	1
12a, 13 - 16	108167	Short Swing Away Arm Left, Ethos / Rogue	1

Item Number	Part Number	Description	Quantity
11a, 12a, 13 - 16	108168	Short Swing Away Arm PAIR, Ethos / Rogue	1
11b, 13 - 16	108169	Tall Swing Away Arm Right, Ethos / Rogue	1
12b, 13 - 16	108170	Tall Swing Away Arm Left, Ethos / Rogue	1
11b, 12b, 13 - 16	108171	Tall Swing Away Arm PAIR, Ethos / Rogue	1
1, 10	117132	Swing Away Arm Receiver w/ Nyliner, Right	1
2, 10	117133	Swing Away Arm Receiver w/ Nyliner, Left	1
3a	115550	Back Plate, Right, Black	1
3b	115550_ _ _	Color Anodized/Color Accent Back Plate, Right	1
3c	115551	Back Plate, Left, Black	1
3d	115551_ _ _	Color Anodized/Color Accent Back Plate, Left	1
4a	115552	Back Plate, Transit, Right, Black	1
4b	115552_ _ _	Color Accent Back Plate, Transit, Right	1
4c	115553	Back Plate, Transit, Left, Black	1
4d	115553_ _ _	Color Accent Back Plate, Transit, Left	1
5	116329	Rigid Backrest Saddle, Narrow, 1.375"	4
6	100658	M6 Nylock Nut Blk Zn	4
7	115546	Strike, Ø.364	2
8	115574	M6 x 30 BHCS, CLS 10.9, NYL, BLZ	2
9	003232	M6 x 55 BHCS Blk Zc	4
10	103517	Nyliner 1" ID x 1.094" OD x 1.125" Long	2
13	100804	Tube End Plug 1"	2
15	000065	M6 x 23.68 Threaded Barrel	2
16	110061	M6 x 10 Stainless Steel Armrest Screw with Patch	4
17	003241	M6 x 60 BHCS P/THD Custom Blk Zc	2