


Ethos Backrest Mount and Hardware

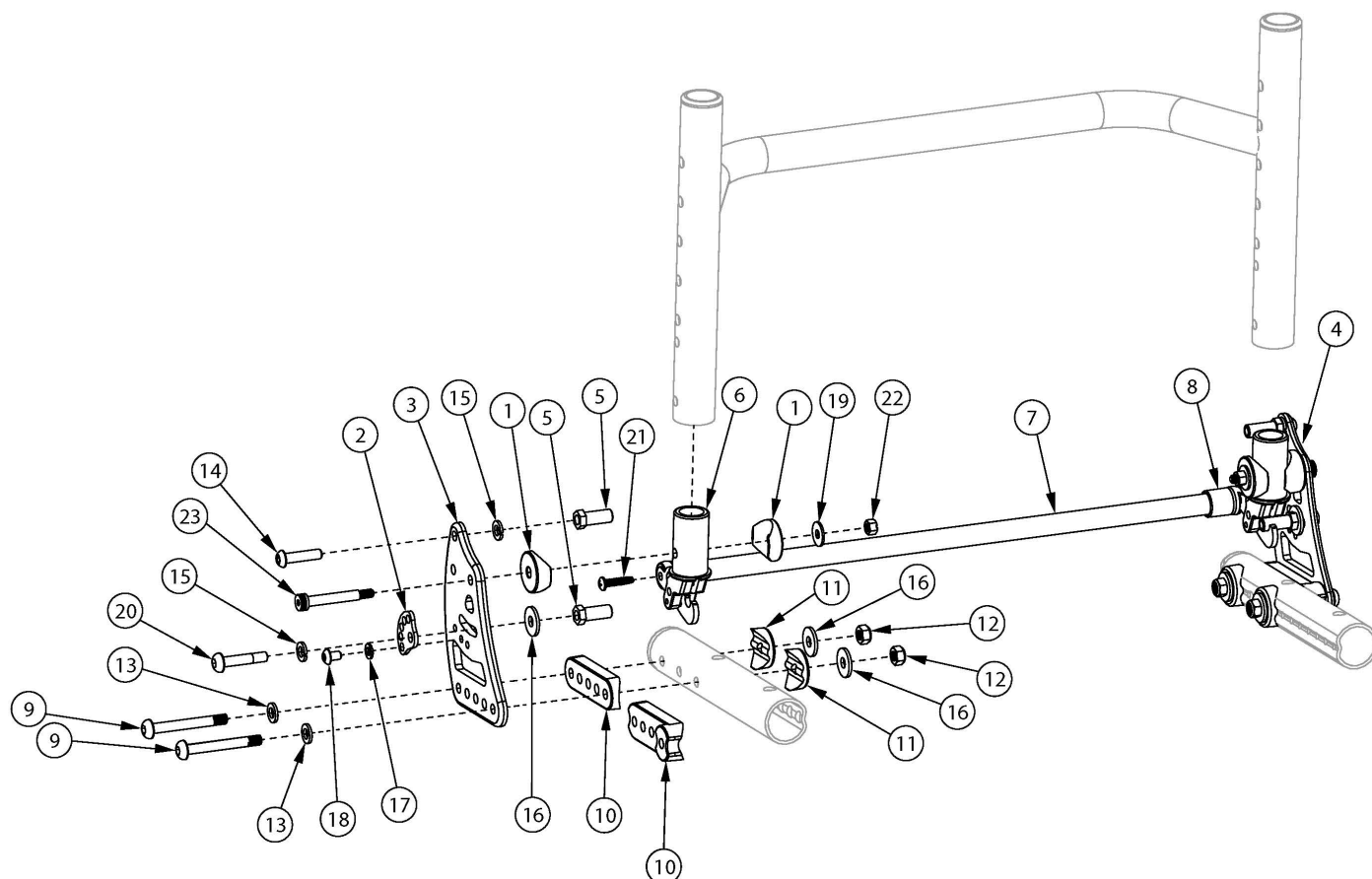

Item Number	Part Number	Description	Quantity	Retail
1a, 1b, 2, 3a, 4a, 5, 6, 9a, 9b, 10a, 10b, 11-23a, 23b	109680	Ethos Backrest Bracket Mount Kit, Black Anodized <i>This kit does not contain a release bar or release bar ends. Select appropriate Backrest Release Bar Kit in addition to this kit for a complete assembly.</i>	1	\$398.12
1a, 1b, 2, 3b, 4b, 5, 6, 9a, 9b, 10a, 10b, 11-23a, 23b	109681_ _ _	Ethos Backrest Bracket Mount Kit, Color Anodized <i>This kit does not contain a release bar or release bar ends. Select appropriate Backrest Release Bar Kit in addition to this kit for a complete assembly.</i>	1	\$422.42
1a, 1b, 2, 3c, 4c, 5, 6, 9a, 9b, 10a, 10b, 11-23a, 23b	109682	Ethos Transit Backrest Bracket Mount Kit, Black Anodized <i>This kit does not contain a release bar or release bar ends. Select appropriate Backrest Release Bar Kit in addition to this kit for a complete assembly.</i>	1	\$339.14
1a	000055	Saddle 1"	4	\$1.55

Item Number	Part Number	Description	Quantity	Retail
		<i>If swing away arms are not present, use quantity 4. Use quantity 2 if swing away arms are present.</i>		
1b	004117	Saddle M6 ID x 1 OD x .25 T <i>If swing away arms are present, use quantity 2 of 000055 and quantity 2 of 004117</i>	2	\$0.46
2	001762	Arc Adjustment Cam	2	\$3.90
3a	003711	Back Bracket Right, Black Anodized	1	\$55.69
3b	003711_ _ _	Color Anodized Back Bracket, Right	1	\$67.84
3c	109296	Back Bracket with Transit, Black, Rigid	1	\$26.20
4a	003712	Back Bracket Left, Black Anodized	1	\$55.69
4b	003712_ _ _	Color Anodized Back Bracket, Left	1	\$67.84
4c	109296	Back Bracket with Transit, Black, Rigid	1	\$26.20
5	003713	Strike Hex	4	\$5.15
6	003734	Pivot Assembly	2	\$57.23
6, 21	108107	Backrest Pivot Replacement Kit w/Instruction (PAIR)	1	\$114.46
7a	003739	Backrest Release Bar 12" Wide, Black Anodized	1	\$20.60
7b	003740	Backrest Release Bar 13" Wide, Black Anodized	1	\$20.60
7c	003741	Backrest Release Bar 14" Wide, Black Anodized	1	\$20.60
7d	003742	Backrest Release Bar 15" Wide, Black Anodized	1	\$20.60
7e	003743	Backrest Release Bar 16" Wide, Black Anodized	1	\$20.60
7f	003744	Backrest Release Bar 17" Wide, Black Anodized	1	\$20.60
7g	003745	Backrest Release Bar 18" Wide, Black Anodized	1	\$20.60
7h	003746	Backrest Release Bar 19" Wide, Black Anodized	1	\$20.60
7i	003747	Backrest Release Bar 20" Wide, Black Anodized	1	\$20.60
7j	109642_ _ _	Ethos Backrest Release Bar 12" Wide, Painted	1	\$50.00
7k	109643_ _ _	Ethos Backrest Release Bar 13" Wide, Painted	1	\$50.00
7l	109644_ _ _	Ethos Backrest Release Bar 14" Wide, Painted	1	\$50.00
7m	109645_ _ _	Ethos Backrest Release Bar 15" Wide, Painted	1	\$50.00
7n	109646_ _ _	Ethos Backrest Release Bar 16" Wide, Painted	1	\$50.00
7o	109647_ _ _	Ethos Backrest Release Bar 17" Wide, Painted	1	\$50.00

Item Number	Part Number	Description	Quantity	Retail
7p	109648_ _ _	Ethos Backrest Release Bar 18" Wide, Painted	1	\$50.00
7q	109649_ _ _	Ethos Backrest Release Bar 19" Wide, Painted	1	\$50.00
7r	109650_ _ _	Ethos Backrest Release Bar 20" Wide, Painted	1	\$50.00
7a, 8	107190	Backrest Release Bar Kit, 12" Wide, Black Anodized with Backrest Release Bar Ends	1	\$28.12
7b, 8	107191	Backrest Release Bar Kit, 13" Wide, Black Anodized with Backrest Release Bar Ends	1	\$28.12
7c, 8	107192	Backrest Release Bar Kit, 14" Wide, Black Anodized with Backrest Release Bar Ends	1	\$28.12
7d, 8	107193	Backrest Release Bar Kit, 15" Wide, Black Anodized with Backrest Release Bar Ends	1	\$28.12
7e, 8	107194	Backrest Release Bar Kit, 16" Wide, Black Anodized with Backrest Release Bar Ends	1	\$28.12
7f, 8	107195	Backrest Release Bar Kit, 17" Wide, Black Anodized with Backrest Release Bar Ends	1	\$28.12
7g, 8	107196	Backrest Release Bar Kit, 18" Wide, Black Anodized with Backrest Release Bar Ends	1	\$28.12
7h, 8	107197	Backrest Release Bar Kit, 19" Wide, Black Anodized with Backrest Release Bar Ends	1	\$28.12
7i, 8	107198	Backrest Release Bar Kit, 20" Wide, Black Anodized with Backrest Release Bar Ends	1	\$28.12
7j, 8	109654_ _ _	Ethos Backrest Release Bar Kit 12" Wide, Painted with Backrest Release Bar Ends	1	\$55.72
7k, 8	109655_ _ _	Ethos Backrest Release Bar Kit 13" Wide, Painted with Backrest Release Bar Ends	1	\$55.72
7l, 8	109656_ _ _	Ethos Backrest Release Bar Kit 14" Wide, Painted with Backrest Release Bar Ends	1	\$55.72
7m, 8	109657_ _ _	Ethos Backrest Release Bar Kit 15" Wide, Painted with Backrest Release Bar Ends	1	\$55.72
7n, 8	109658_ _ _	Ethos Backrest Release Bar Kit 16" Wide, Painted with Backrest Release Bar Ends	1	\$55.72
7o, 8	109659_ _ _	Ethos Backrest Release Bar Kit 17" Wide, Painted with Backrest Release Bar Ends	1	\$55.72
7p, 8	109660_ _ _	Ethos Backrest Release Bar Kit 18" Wide, Painted with Backrest Release Bar Ends	1	\$55.72
7q, 8	109661_ _ _	Ethos Backrest Release Bar Kit 19" Wide, Painted with Backrest Release Bar Ends	1	\$55.72
7r, 8	109662_ _ _	Ethos Backrest Release Bar Kit 20" Wide, Painted with Backrest Release Bar Ends	1	\$55.72
8	003750	Backrest Release Bar End	2	\$2.86
9a	003846	M6 x 50 BHCS, Custom, Blk Zc <i>Used if Swing Away Arms are not present</i>	4	\$14.33
9b	003232	M6 x 55 BHCS Blk Zc <i>Used if Swing Away Arms are present</i>	4	\$8.76
10a	003848	Back Bracket Saddle 1 1/8" <i>Used if Swing Away Arms are not present</i>	2	\$1.06

Item Number	Part Number	Description	Quantity	Retail
10b	004116	Saddle, Swing Away Arm, 1 1/8" Tube <i>Used if Swing Away Arms are present</i>	2	\$1.11
11	003849	Saddle For Dimple 1 1/8"	4	\$0.43
12	100658	M6 Nylock Nut Blk Zn	4	\$1.03
13	100746	M6 Flat Washer Blk Zn	4	\$1.55
14	101593	M6 x 25 BTN SHCS BLKZC	2	\$1.55
15	101835	M6 Lock Washer Blk Zn	4	\$0.52
16	101853	M6 Washer Blk Zn	2	\$0.52
17	102614	M5 Split Lock Washer BLK ZC	2	\$0.52
18	102621	M5 x 8 BHCS BLK ZC	2	\$0.52
19	103022	M5 Washer, Blk Zn	2	\$0.52
20	103227	M6 x 30 Long Shoulder BHCS BZ	2	\$4.12
21	103279	#8-16 Tri-Lobular Plastic Screw BZ	2	\$0.52
22	103283	M5 Top Lock Nut BZ	2	\$0.52
23a	103284	M6 x 35 Shoulder Bolt BZ <i>Used if Swing Away Arms are not present</i>	2	\$4.64
23b	103397	M6 x 40 Shoulder Screw BLK ZC <i>Used if Swing Away Arms are present</i>	2	\$2.50

Matte Finish	
Code	Color Name
CGM	Charcoal Gray
EGM	Emerald Green
GGM	Graphite Gray*
IBM	Iridescent Burgundy
INM	Iridescent Navy
AGM	Military Green**
RRM	Retro Red
RBM	Royal Blue
SBM	Shadow Black
SWM	Smooth White

* Discontinued Color

** Formerly called Army Green

Gloss Finish	
Code	Color Name
BLK	Black
BLU	Blue
SKY	Blue Skies
BRO	Bright Orange*
PNK	Bubblegum Pink
BUR	Burgundy*
CBL	Candy Blue
PUR	Candy Purple
RED	Candy Red
COP	Copper
DPR	Deep Purple*
EBL	Electric Blue*
GRN	Forest Green*
FUS	Fuchsia*
GSM	Granny Smith
MTF	Metallic Fuchsia
MOR	Metallic Orange
ORG	Orange*
SAP	Sapphire*
SIL	Silver
TEA	Teal
YLW	Yellow
FLM	Graphic Image Transfer

Color Anodizing Samples

