


Catalyst 5 Performance Caster Housing for Fixed Front

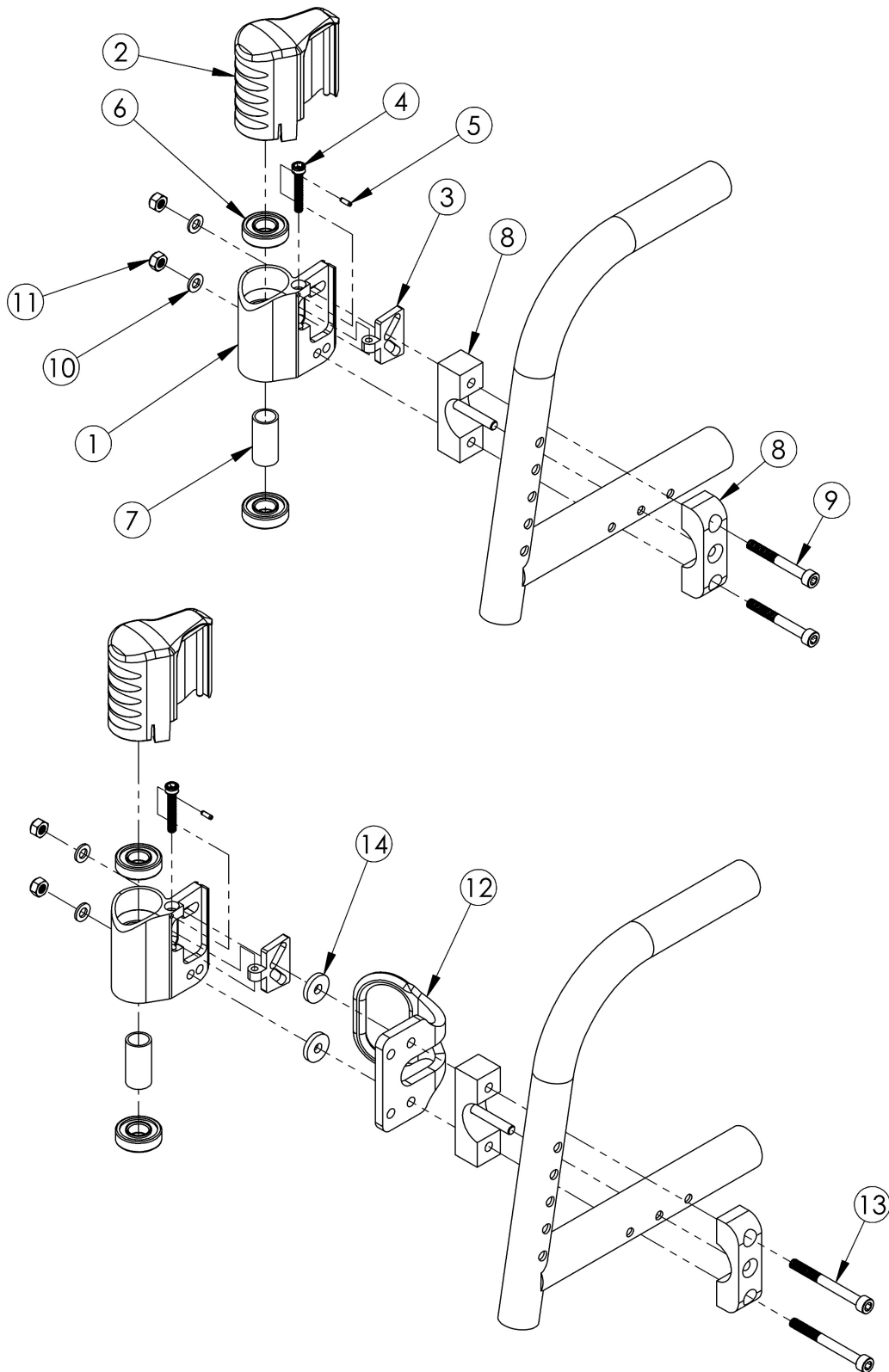

Not Available with Heavy Duty Option Please Note: The shown orientation of the caster housing above is in the "Forward" position. If only 1 side caster housing is needed, customer must confirm the orientation of the caster housing.

Item Number	Part Number	Description	Quantity
1a, 2a, 3-11	502423	Performance Caster Housing Kit for Fixed Front Right	1
1b, 2b, 3-11	502424	Performance Caster Housing Kit for Fixed Front Left	1
1a, 1b, 2a, 2b, 3-11	502425	Performance Caster Housing Kit for Fixed Front PAIR	1
1a, 2a, 3-8, 10-14	502420	Performance Caster Housing Kit for Fixed Front with Transit Right	1
1b, 2b, 3-8, 10-14	502421	Performance Caster Housing Kit for Fixed Front with Transit Left	1
1a, 1b, 2a, 2b, 3-8, 10-14	502422	Performance Caster Housing Kit for Fixed Front with Transit PAIR	1
1a, 3, 4, 5	502522	Caster Barrel RP Right w/ Adjustment Plate CAT	1
1b, 3, 4, 5	502523	Caster Barrel RP Left w/ Adjustment Plate CAT	1
2a	002442	Cover Caster Barrel Right CAT	1
2b	002443	Cover Caster Barrel Left CAT	1
6	100902	Bearing 1 1/8" x 1/2" R8RS	2
7	002447	Spacer Caster Barrel	1
8	002775	Performance Caster Housing Fixed Front Clamp CAT	1
9	102473	M6 X 50 CLS12.9 SHCS BLK ZC	2
10	100746	M6 Flat Washer Blk Zn	2
11	100658	M6 Nylock Nut Blk Zn	2
13	102474	M6 X 60 CLS12.9 SHCS BLK ZC	2
14	102814	Washer Nylon 1/4 x 3/4 x 1/8	2
1a, 6, 7	002888	RP Tuning Caster Barrel w/ Bearings Right	1
1b, 6, 7	002889	RP Tuning Caster Barrel w/ Bearings Left	1