


Discontinued Flip Attendant Foot Lock

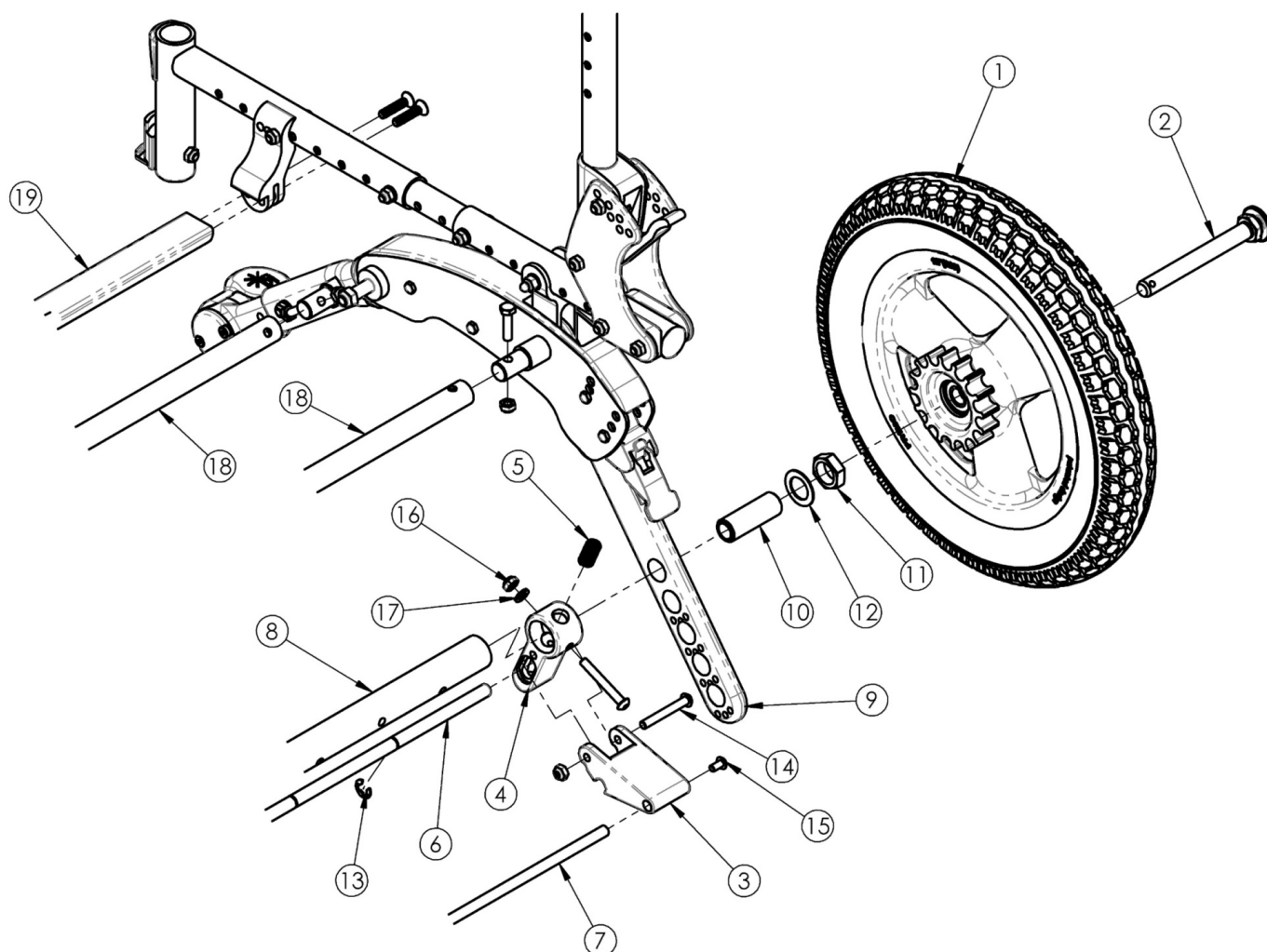

This Flip Attendant Foot Lock is discontinued and was used on serial numbers prior to FLP007536.

Item Number	Part Number	Description	Quantity
1a	502853	Flip Attendant Foot Lock Wheel, 12" Pneumatic EACH	1
1b	502854	Flip Attendant Foot Lock Wheel, 12" Airless EACH	1
1c	502855	Flip Attendant Foot Lock Wheel, 12" Full Poly EACH	1
1d	502856	Flip Attendant Foot Lock Wheel, 16" Pneumatic EACH	1
1e	502857	Flip Attendant Foot Lock Wheel, 16" Airless EACH	1
1f	502858	Flip Attendant Foot Lock Wheel, 16" Full Poly EACH	1
2	101108	Axle Quick Release w/ Button 1/2" x 99MM	2

Item Number	Part Number	Description	Quantity
3	001928	Lever, Hub Lock	2
5	001929	Spring, Hub Lock	2
7a	001939	Link Rod Hub Lock 10 Wide	1
7b	001940	Link Rod Hub Lock 11 Wide	1
7c	001941	Link Rod Hub Lock 12 Wide	1
7d	001942	Link Rod Hub Lock 13 Wide	1
7e	001943	Link Rod Hub Lock 14 Wide	1
7f	001944	Link Rod Hub Lock 15 Wide	1
7g	001945	Link Rod Hub Lock 16 Wide	1
7h	001946	Link Rod Hub Lock 17 Wide	1
7i	001947	Link Rod Hub Lock 18 Wide	1
8a	001840	Axle Tube, 10W	1
8b	001841	Axle Tube, 11W	1
8c	001842	Axle Tube, 12W	1
8d	001843	Axle Tube, 13W	1
8e	001844	Axle Tube, 14W	1
8f	001845	Axle Tube, 15W	1
8g	001846	Axle Tube, 16W	1
8h	001847	Axle Tube, 17W	1
8i	001848	Axle Tube, 18W	1
9a	001838	Axle Bar, Standard	2
9b	001839	Axle Bar, Self Propel	2
10	100756	Axle Receiver 1/2" Diameter	2
11	100514	Axle Receiver Nut M18 x D12 x 8mm	2
12	100583	Axle Plate Washer 18mm	2
13	102811	E-Ring, 3/8"	2
14	100670	M6 x 45 BTN SHCS BLK ZC	4
15	100669	M5 x 10 BTN SHCS BLKZC w/Patch	2

Item Number	Part Number	Description	Quantity
16	100658	M6 Nylock Nut Blk Zn	4
17	100746	M6 Flat Washer Blk Zn	2
18a	002719	Frame Growth Strut 6.5" <i>Used on 10" wide chairs</i>	1
18b	002629	Frame Growth Strut 7.5" <i>Used on 11" wide chairs</i>	1
18c	002630	Frame Growth Strut 8.5" <i>Used on 12" wide chairs</i>	1
18d	002631	Frame Growth Strut 9.5" <i>Used on 10" and 13" wide chairs</i>	1
18e	002632	Frame Growth Strut 10.5" <i>Used on 11" and 14" wide chairs</i>	1
18f	002633	Frame Growth Strut 11.5" <i>Used on 12" and 15" wide chairs</i>	1
18g	002634	Frame Growth Strut 12.5" <i>Used on 13" and 16" wide chairs</i>	1
18h	002635	Frame Growth Strut 13.5" <i>Used on 14" and 17" wide chairs</i>	1
18i	002636	Frame Growth Strut 14.5" <i>Used on 15" and 18" wide chairs</i>	1
18j	002637	Frame Growth Strut 15.5" <i>Used on 16" wide chairs</i>	1
18k	002638	Frame Growth Strut 16.5" <i>Used on 17" wide chairs</i>	1
18l	002639	Frame Growth Strut 17.5" <i>Used on 18" wide chairs</i>	1
19a	001879	Strut Tube Seat 10 Wide	1
19b	001880	Strut Tube Seat 11 Wide	1
19c	001881	Strut Tube Seat 12 Wide	1
19d	001882	Strut Tube Seat 13 Wide	1
19e	001883	Strut Tube Seat 14 Wide	1
19f	001884	Strut Tube Seat 15 Wide	1
19g	001885	Strut Tube Seat 16 Wide	1

Item Number	Part Number	Description	Quantity
19h	001886	Strut Tube Seat 17 Wide	1
19i	001887	Strut Tube Seat 18 Wide	1