


Liberty Folding Back Cross Brace

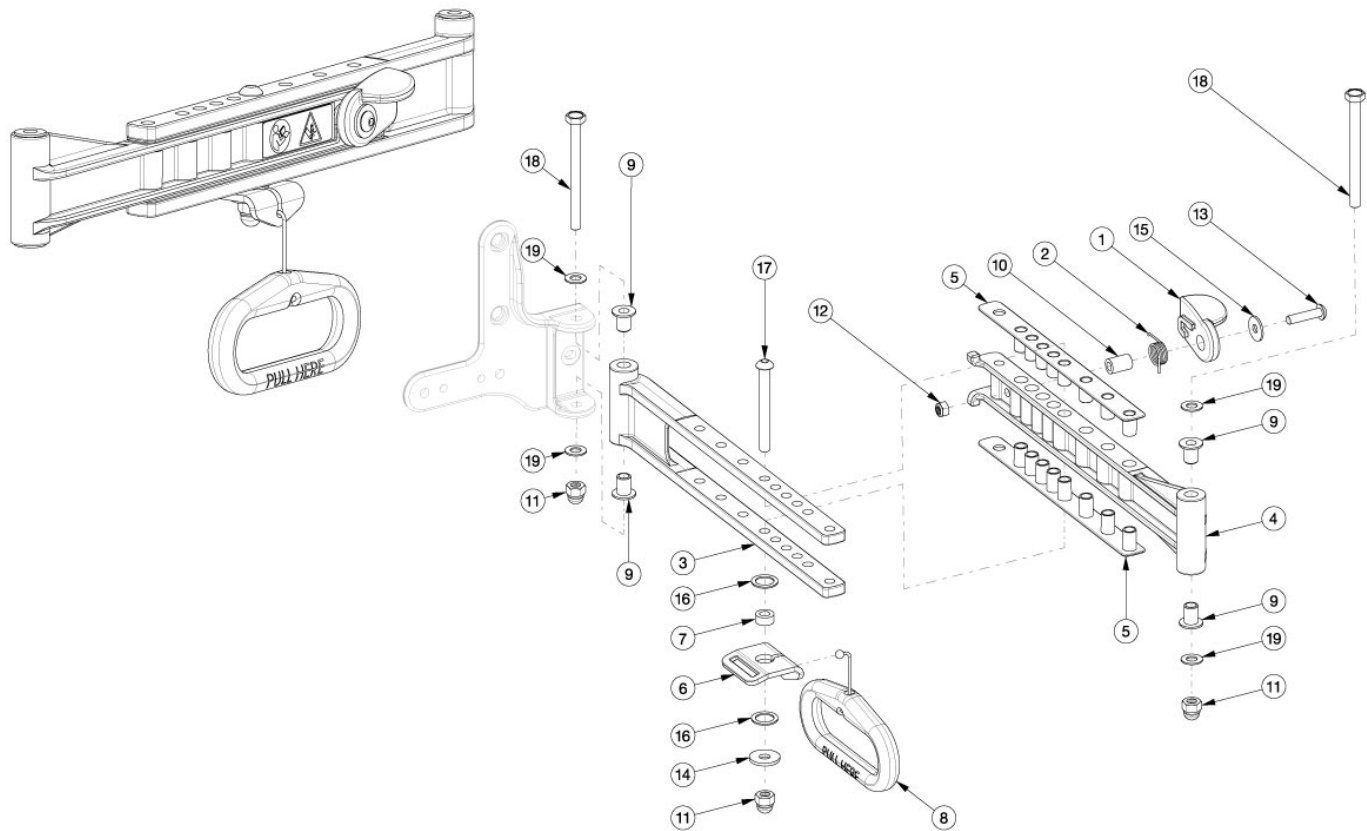

For Non-Folding Cross Brace hardware for a Liberty Folding Frame, see Liberty Non-Folding Cross Brace page.

Item Number	Part Number	Description	Quantity
1, 2, 10, 12, 13, 15	506656	Fold Lever Assembly with Instruction	1
1, 2, 3a, 4a, 5-17	110083	Cross Brace Assembly, Back, Small, 14W with Instruction <i>Used on 14 - 15" wide with folding frame. Has the ability to achieve all small width range settings (14" - 18").</i>	1
1, 2, 3a, 4a, 5-17	110084	Cross Brace Assembly, Back, Small, 16W with Instruction <i>Used on 16 - 18" wide with a small width range and folding frame</i>	1
1, 2, 3b, 4b, 5-17	110085	Cross Brace Assembly, Back, Medium, 16W with Instruction <i>Used on 16 - 17" wide with medium width range and folding frame. Has the ability to achieve all medium width range settings (16" - 20").</i>	1
1, 2, 3b, 4b, 5-17	110086	Cross Brace Assembly, Back, Medium, 18W with Instruction <i>Used on 18 - 20" wide with medium width range and folding frame</i>	1
1, 2, 3c, 4c, 5-17	110087	Cross Brace Assembly, Back, Large, 18W with Instruction	1

Item Number	Part Number	Description	Quantity
		<i>Used on 18 - 19" wide with large width range and folding frame. Has the ability to achieve all large width range settings (18" - 22").</i>	
1, 2, 3c, 4c, 5-17	110088	Cross Brace Assembly, Back, Large, 20W with Instruction <i>Used on 20 - 22" wide with large width range and folding frame</i>	1
1, 2, 3a, 4a, 5-19	506142	Cross Brace Kit, Back, Folding, Small	1
1, 2, 3b, 4b, 5-19	506143	Cross Brace Kit, Back, Folding, Medium	1
1, 2, 3c, 4c, 5-19	506144	Cross Brace Kit, Back, Folding, Large	1
3a, 4a, 5, 9, 18, 19	110095	Cross Brace Kit, Back, Non-Folding, Small	1
3b, 4b, 5, 9, 18, 19	110096	Cross Brace Kit, Back, Non-Folding, Medium	1
3c, 4c, 5, 9, 18, 19	110097	Cross Brace Kit, Back, Non-Folding, Large	1
3a, 4a, 5, 9	110089	Cross Brace Assembly, Back, Non-Folding, Small, 14W with Instruction <i>Used on 14 - 15" wide with non-folding frame. Has the ability to achieve all small width range settings (14" - 18"). Includes non-folding frame hardware.</i>	1
3a, 4a, 5, 9	110090	Cross Brace Assembly, Back, Non-Folding, Small, 16W with Instruction <i>Used on 16 - 18" wide with a small width range and non-folding frame. Includes non-folding frame hardware.</i>	1
3b, 4b, 5, 9	110091	Cross Brace Assembly, Back, Non-Folding, Medium, 16W with Instruction <i>Used on 16 - 17" wide with medium width range and non-folding frame. Has the ability to achieve all medium width range settings (16" - 20"). Includes non-folding frame hardware.</i>	1
3b, 4b, 5, 9	110092	Cross Brace Assembly, Back, Non-Folding, Medium, 18W with Instruction <i>Used on 18 - 20" wide with medium width range and non-folding frame. Includes non-folding frame hardware.</i>	1
3c, 4c, 5, 9	110093	Cross Brace Assembly, Back, Non-Folding, Large, 18W with Instruction <i>Used on 18 - 19" wide with large width range and non-folding frame. Has the ability to achieve all large width range settings (18" - 22"). Includes non-folding frame hardware.</i>	1
3c, 4c, 5, 9	110094	Cross Brace Assembly, Back, Non-Folding, Large, 20W with Instruction <i>Used on 20 - 22" wide with large width range and non-folding frame. Includes non-folding frame hardware.</i>	1
1	506655	Fold Lever with Instruction <i>Not included with Non-folding frame</i>	1
2	003267	Spring, Latch <i>Not included with Non-folding frame</i>	1
5	003326	Pivot Pad, Backrest	2
6, 8	506313	Back Cross Brace Handle and Mount	1

Item Number	Part Number	Description	Quantity
		<i>Not included with Non-folding frame</i>	
7	003437	Spacer, .430ODx.242IDx.200L <i>Not included with Non-folding frame</i>	1
8	003433	Handle Assembly, Liberty <i>Not included with Non-folding frame</i>	1
9	003666	Bearing, Flg, 6mm ID x 12mm Lg	4
10	003683	Spacer, .350ODx.201IDx.520L <i>Not included with Non-folding frame</i>	1
11	100558	M6 Nyl Ins Domed Cap L/N Zc <i>Not included with Non-folding frame</i>	1
12	100657	M5 Nylock Nut Blk Zn <i>Not included with Non-folding frame</i>	1
13	101117	M5 x 22 BTN SHCS BLK ZC <i>Not included with Non-folding frame</i>	1
14	101853	M6 Washer Blk Zn <i>Not included with Non-folding frame</i>	1
15	103022	M5 Washer, Blk Zn <i>Not included with Non-folding frame</i>	1
16	103024	Washer, Nylon, .428IDx.625ODx.031T <i>Not included with Non-folding frame</i>	2
17	103141	M6 x 55 BHCS, Shldr, BLK Zn <i>Not included with Non-folding frame</i>	1
18	103028	M6 x 70 HHCS, Shldr, Blk Zn	2
19	102803	Washer Flat .250 X .500 X .027 Hx Blz	4