


# Catalyst 5Vx Transit

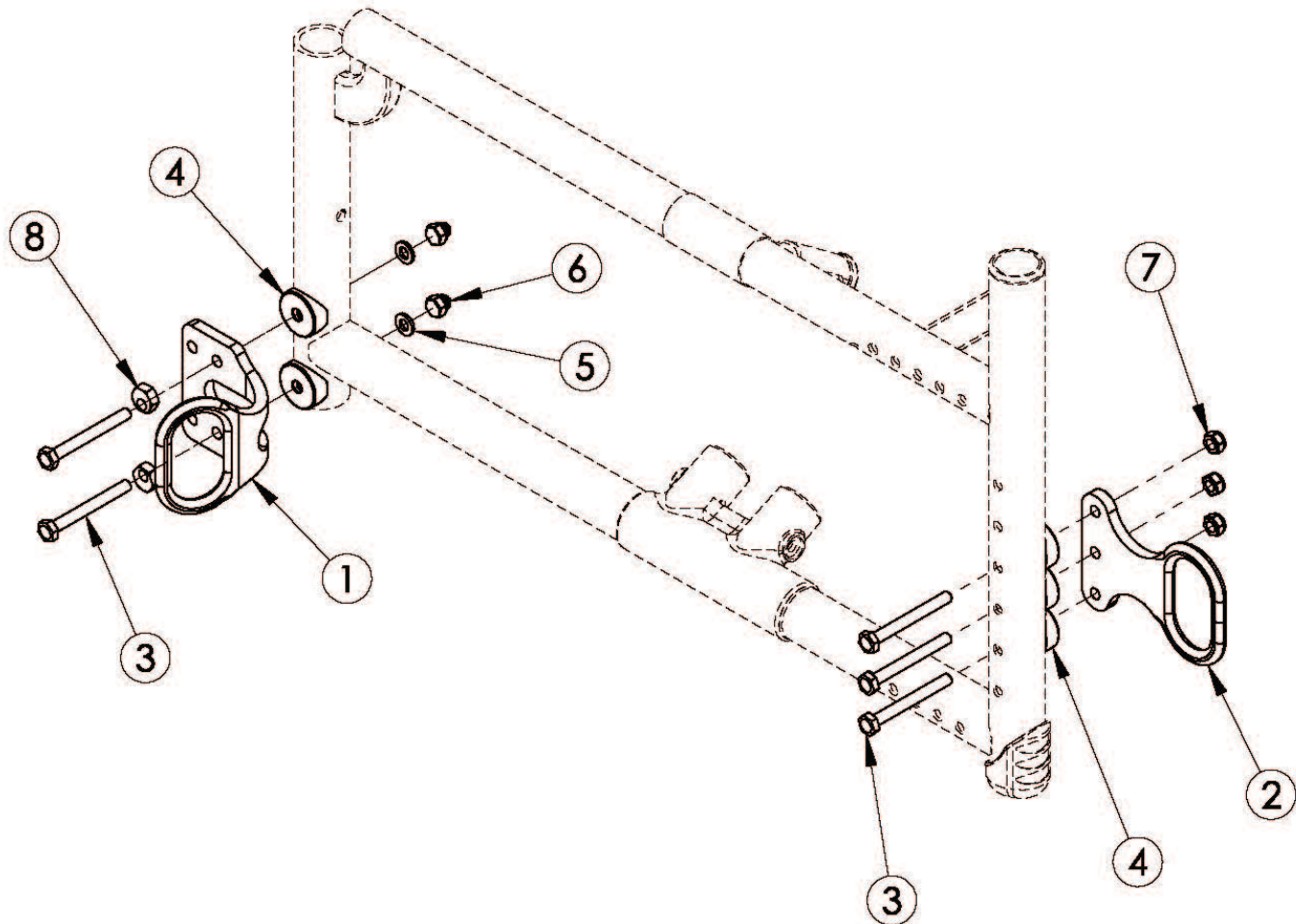

Notice: Chair configuration review prior to parts being quoted or ordered. <p>  
Not available with Heavy Duty Option or Removable Seat Pan.

Item Number	Part Number	Description	Quantity	Retail
1-2, 3a-3c, 4-8	500548	Transit Bracket Retrofit Kit CAT5 / SPARK <i>Not available with Heavy Duty Option. This kit is a complete retrofit, containing a pair of front and rear transit brackets and hardware.</i>	1	\$401.86
3a	101826	M6 x 50 HHCS Blk Zn <i>Use quantity 2 if only Transit is present; Use quantity 1 if Transit and Anti-Tip bracket are present</i>	4	\$2.21
3b	100676	M6 x 55 CLS10.9 DIN931 HHCS ZC <i>Use quantity 2 if Transit and Anti-Tip bracket are present. Use quantity 1 if Swing Away Arm bracket, Transit, and Anti-Tip bracket are present</i>	4	\$1.66

Item Number	Part Number	Description	Quantity	Retail
3c	100677	M6 x 60 CLS10.9 DIN931 HHCS ZC <i>Used to attach Front Transit Bracket. Use quantity 2 if Transit, Anti-Tip bracket and Swing Away Arm bracket are present</i>	4	\$1.66
4	000056	Saddle 1 1/8"	10	\$1.66
5	100746	M6 Flat Washer Blk Zn	4	\$1.66
6	100558	M6 NYL INS Domed Cap L/N ZC	4	\$1.10
7	100658	M6 Nylock Nut Blk Zn	6	\$1.10
8	100510	Eccentric Washer	4	\$1.73