


Flip Attendant Foot Lock

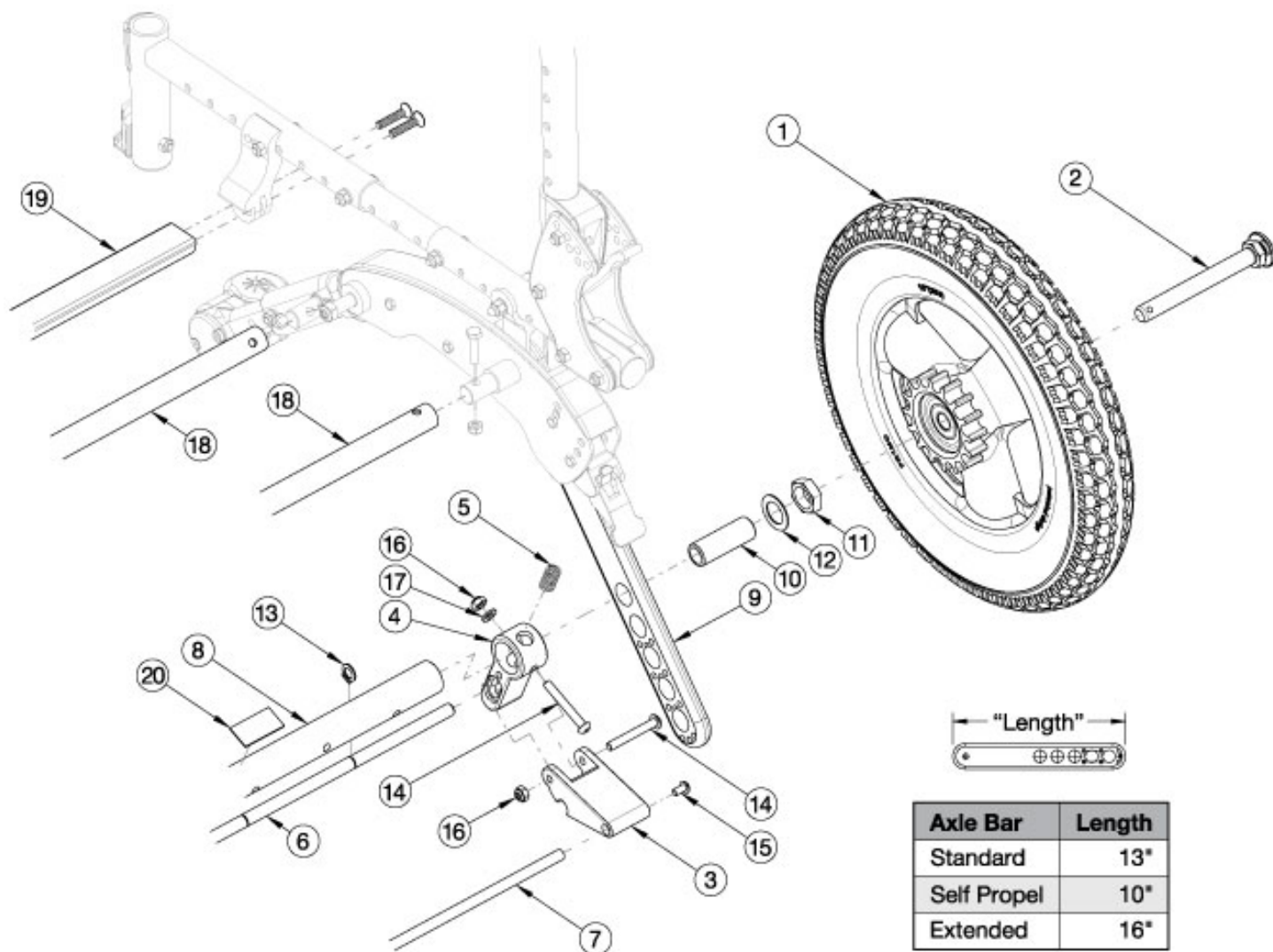

This design effective starting with serial number FLP007536. If retrofitting/replacing, for a full assembly please select a wheel size, axle tube and the Attendant Foot Lock Kit.

Item Number	Part Number	Description	Quantity
3-5, 6a, 7a, 13-17	505470	Attendant Foot Lock Kit 10 Wide FLIP	1
3-5, 6b, 7b, 13-17	505471	Attendant Foot Lock Kit 11 Wide FLIP	1
3-5, 6c, 7c, 13-17	505472	Attendant Foot Lock Kit 12 Wide FLIP	1
3-5, 6d, 7d, 13-17	505473	Attendant Foot Lock Kit 13 Wide FLIP	1
3-5, 6e, 7e, 13-17	505474	Attendant Foot Lock Kit 14 Wide FLIP	1
3-5, 6f, 7f, 13-17	505475	Attendant Foot Lock Kit 15 Wide FLIP	1
3-5, 6g, 7g, 13-17	505476	Attendant Foot Lock Kit 16 Wide FLIP	1

Item Number	Part Number	Description	Quantity
3-5, 6h, 7h, 13-17	505477	Attendant Foot Lock Kit 17 Wide FLIP <i>Used with axle receiver 100756 (Discontinued 3/3/2023)</i>	1
3-5, 6i, 7i, 13-17	505478	Attendant Foot Lock Kit 18 Wide FLIP <i>Used with axle receiver 114480</i>	1
4, 6a, 7a, 8a, 13, 18a, 18d, 19a, 20	505479	Strut Kit 10" Wide Flip w/ Attendant Foot Lock	1
4, 6b, 7b, 8b, 13, 18b, 18e, 19b, 20	505480	Strut Kit 11" Wide Flip w/ Attendant Foot Lock	1
4, 6c, 7c, 8c, 13, 18c, 18f, 19c, 20	505481	Strut Kit 12" Wide Flip w/ Attendant Foot Lock	1
4, 6d, 7d, 8d, 13, 18d, 18g, 19d, 20	505482	Strut Kit 13" Wide Flip w/ Attendant Foot Lock	1
4, 6e, 7e, 8e, 13, 18e, 18h, 19e, 20	505483	Strut Kit 14" Wide Flip w/ Attendant Foot Lock	1
4, 6f, 7f, 8f, 13, 18f, 18i, 19f, 20	505484	Strut Kit 15" Wide Flip w/ Attendant Foot Lock	1
4, 6g, 7g, 8g, 13, 18g, 18j, 19g, 20	505485	Strut Kit 16" Wide Flip w/ Attendant Foot Lock	1
4, 6h, 7h, 8h, 13, 18h, 18k, 19h, 20	505486	Strut Kit 17" Wide Flip w/ Attendant Foot Lock	1
4, 6i, 7i, 8i, 13, 18i, 18l, 19i, 20	505487	Strut Kit 18" Wide Flip w/ Attendant Foot Lock	1
1a	502853	Flip Attendant Foot Lock Wheel, 12" Pneumatic EACH	1
1b	502854	Flip Attendant Foot Lock Wheel, 12" Airless EACH	1
1c	502855	Flip Attendant Foot Lock Wheel, 12" Full Poly EACH	1
1d	502856	Flip Attendant Foot Lock Wheel, 16" Pneumatic EACH	1
1e	502857	Flip Attendant Foot Lock Wheel, 16" Airless EACH	1
1f	502858	Flip Attendant Foot Lock Wheel, 16" Full Poly EACH	1
1g	113604	Flip Attendant Foot Lock Wheel, 16" Low Poly EACH	1
2a	101108	Axle Quick Release w/ Button 1/2" x 99MM	2
2b	101191	Axle Quick Release w/ Button 1/2" x 110MM	2
3	001928	Lever, Hub Lock	2
5	001929	Spring, Hub Lock	2
4, 6a, 13	505461	Locking Rod Hub Lock 10 Wide Kit	1

Item Number	Part Number	Description	Quantity
4, 6b, 13	505462	Locking Rod Hub Lock 11 Wide Kit	1
4, 6c, 13	505463	Locking Rod Hub Lock 12 Wide Kit	1
4, 6d, 13	505464	Locking Rod Hub Lock 13 Wide Kit	1
4, 6e, 13	505465	Locking Rod Hub Lock 14 Wide Kit	1
4, 6f, 13	505466	Locking Rod Hub Lock 15 Wide Kit	1
4, 6g, 13	505467	Locking Rod Hub Lock 16 Wide Kit	1
4, 6h, 13	505468	Locking Rod Hub Lock 17 Wide Kit	1
4, 6i, 13	505469	Locking Rod Hub Lock 18 Wide Kit	1
7a	001939	Link Rod Hub Lock 10 Wide	1
7b	001940	Link Rod Hub Lock 11 Wide	1
7c	001941	Link Rod Hub Lock 12 Wide <i>Used with the reclining backrest.</i>	1
7d	001942	Link Rod Hub Lock 13 Wide <i>Replaced part number 100756</i>	1
7e	001943	Link Rod Hub Lock 14 Wide	1
7f	001944	Link Rod Hub Lock 15 Wide	1
7g	001945	Link Rod Hub Lock 16 Wide	1
7h	001946	Link Rod Hub Lock 17 Wide	1
7i	001947	Link Rod Hub Lock 18 Wide	1
8a, 20	115883	Axle Tube with Wear Strip, 10 Wide	1
8b, 20	115884	Axle Tube with Wear Strip, 11 Wide	1
8c, 20	115885	Axle Tube with Wear Strip, 12 Wide	1
8d, 20	115886	Axle Tube with Wear Strip, 13 Wide	1
8e, 20	115887	Axle Tube with Wear Strip, 14 Wide	1
8f, 20	115888	Axle Tube with Wear Strip, 15 Wide	1
8g, 20	115889	Axle Tube with Wear Strip, 16 Wide	1
8h, 20	115890	Axle Tube with Wear Strip, 17 Wide	1
8i, 20	115891	Axle Tube with Wear Strip, 18 Wide	1
9a	001838	Axle Bar, Standard	2

Item Number	Part Number	Description	Quantity
9b	001839	Axle Bar, Self Propel	2
9c	003838	Axle Bar, Extended	1
10	114480	Axle Receiver, 2.5" Length	2
11	100514	Axle Receiver Nut M18 x D12 x 8mm	2
12	100583	Axle Plate Washer 18mm	2
13	103503	3/8" External Self-Locking Shaft Ring	2
14	100670	M6 x 45 BTN SHCS BLK ZC	4
15	100669	M5 x 10 BTN SHCS BLKZC w/Patch	2
16	100658	M6 Nylock Nut Blk Zn	4
17	100746	M6 Flat Washer Blk Zn	2
18a	002719	Frame Growth Strut 6.5" <i>Used on 10" wide chairs</i>	1
18b	002629	Frame Growth Strut 7.5" <i>Used on 11" wide chairs</i>	1
18c	002630	Frame Growth Strut 8.5" <i>Used on 12" wide chairs</i>	1
18d	002631	Frame Growth Strut 9.5" <i>Used on 10" and 13" wide chairs</i>	1
18e	002632	Frame Growth Strut 10.5" <i>Used on 11" and 14" wide chairs</i>	1
18f	002633	Frame Growth Strut 11.5" <i>Used on 12" and 15" wide chairs</i>	1
18g	002634	Frame Growth Strut 12.5" <i>Used on 13" and 16" wide chairs</i>	1
18h	002635	Frame Growth Strut 13.5" <i>Used on 14" and 17" wide chairs</i>	1
18i	002636	Frame Growth Strut 14.5" <i>Used on 15" and 18" wide chairs</i>	1
18j	002637	Frame Growth Strut 15.5" <i>Used on 16" wide chairs</i>	1
18k	002638	Frame Growth Strut 16.5" <i>Used on 17" wide chairs</i>	1
18l	002639	Frame Growth Strut 17.5"	1

Item Number	Part Number	Description	Quantity
		<i>Used on 18" wide chairs</i>	
19a	001879	Strut Tube Seat 10 Wide	1
19b	001880	Strut Tube Seat 11 Wide	1
19c	001881	Strut Tube Seat 12 Wide	1
19d	001882	Strut Tube Seat 13 Wide	1
19e	001883	Strut Tube Seat 14 Wide	1
19f	001884	Strut Tube Seat 15 Wide	1
19g	001885	Strut Tube Seat 16 Wide	1
19h	001886	Strut Tube Seat 17 Wide	1
19i	001887	Strut Tube Seat 18 Wide	1