


Discontinued Focus CR Caster Forks and Stems

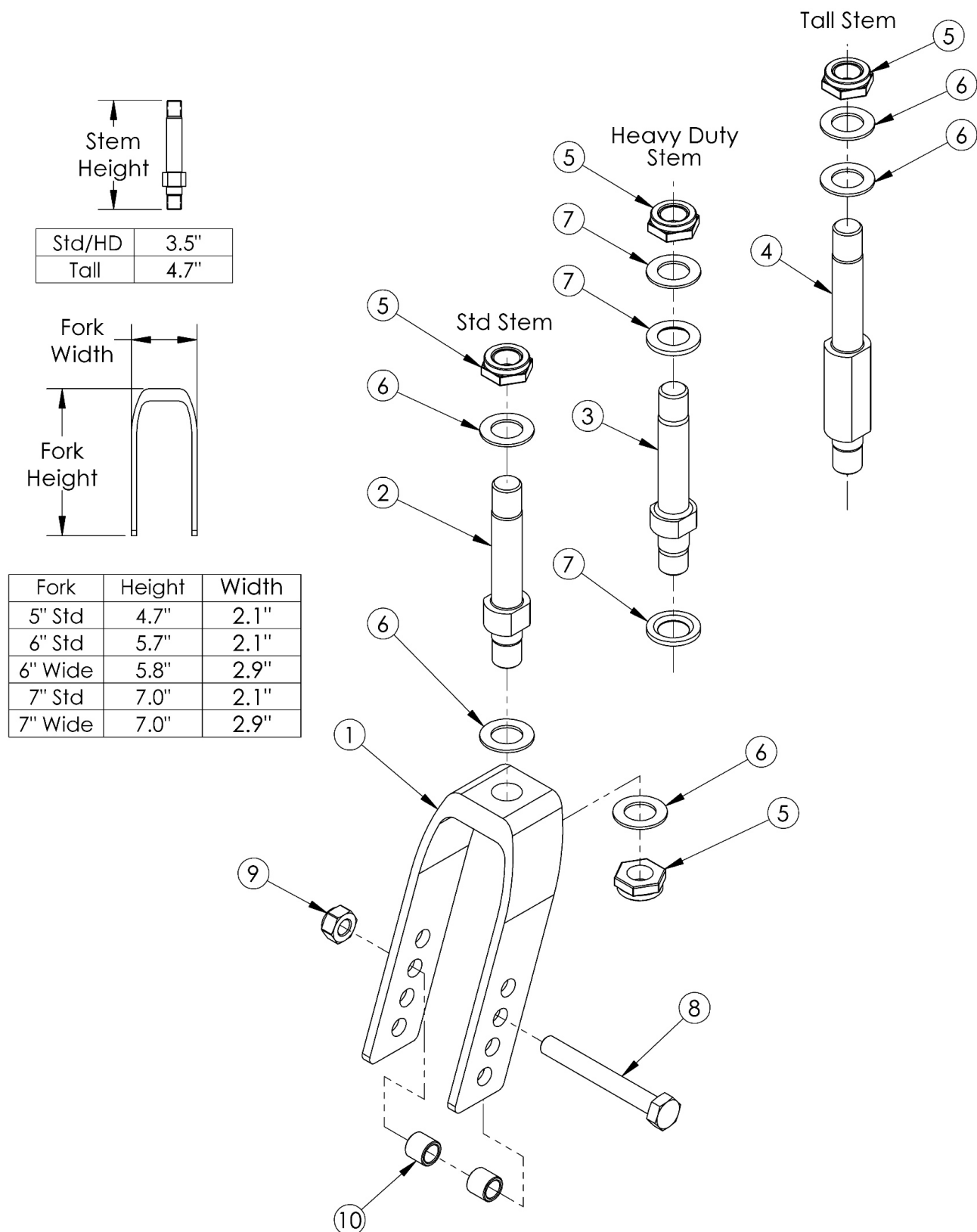

If changing to 2" casters, wide forks and hardware needed.

Item Number	Part Number	Description	Quantity
1a, 2, 5, 6	503816	Fork Assembly 5" Std Stem, Tilt	1
1b, 2, 5, 6	503814	Fork Assembly 6" Std Stem, Tilt	1
1c, 2, 5, 6	503817	Fork Assembly 6" Wide Std Stem, Tilt <i>Only used with 2" wide casters</i>	1
1d, 2, 5, 6	503815	Fork Assembly 7" Std Stem, Tilt	1
1e, 2, 5, 6	503818	Fork Assembly 7" Wide Std Stem, Tilt <i>Only used with 2" wide casters</i>	1
1e, 3, 5-7	503122	Fork Assembly 7" Wide Heavy Duty Std Stem Focus <i>Only used with 2" wide casters</i>	1
8a, 9, 10a	502653	Caster Attachment Hardware Kit CAT/FCS	1
8b, 9, 10b	502654	Caster Attachment Hardware Kit for 6" and 7" Wide Fork, CAT/FCS <i>Only used with 2" wide casters</i>	1
1a, 5	503823	Caster Fork 5" Tilt	1
1b, 5	503824	Caster Fork 6" Tilt	1
1c, 5	503825	Caster Fork 6" Wide Tilt <i>Only used with 2" wide casters</i>	1
1d, 5	503826	Caster Fork 7" Tilt	1
1e, 5	503827	Caster Fork 7" Wide Tilt <i>Only used with 2" wide casters</i>	1
2, 5	503819	Caster Stem Standard, Tilt	1
3, 5	503820	Caster Stem Standard HD, Focus	1
5	100529	M12 x 1.25mm Lock Nut Black	2
6	100682	Washer 1/2" .518 x .875 x .047 F/W Black Zinc	3
7	002593	Caster Stem Washer Heavy Duty <i>Used with Heavy Duty</i>	2
8a	101829	M8 x 65 HHCS Blk Zn	1
8b	101831	M8 x 80 HHCS Blk Zn <i>Used with wide forks</i>	1
9	100659	M8 Nylock Nut Blk Zn	1
10a	100792	Caster Spacer D8 x D12 x 10mm / Close Mount Handrim Spacer <i>Used with all casters except 6" x 2" Poly</i>	2

Item Number	Part Number	Description	Quantity
10b	001246	Caster Spacer Black 6 x 2 Poly .472ODX.319IDX.370LG <i>Only used with 6" x 2" casters</i>	2