


# Catalyst Tension Adjustable Back Upholstery - Standard Backposts

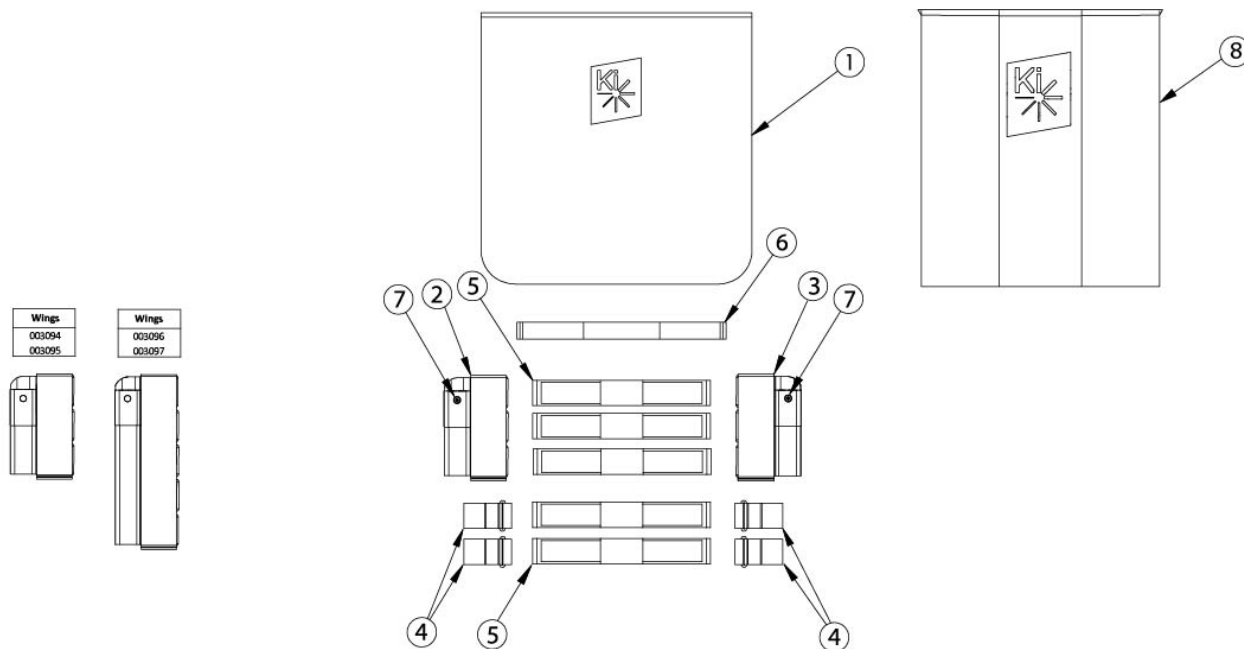

Backrest Range and Height	Wings	Total Strap Qty	Total D-Ring Qty
Short 8.5"	003094	3	0
	003095		
Short 9.5" - 10.5"	003094	4	2
	003095		
Short 11.5"	003094	5	4
	003095		
Medium 12.5"	003096	5	0
	003097		
Medium 13.5" - 14.5"	003096	6	2
	003097		
Medium 15.5"	003096	7	4
Tall 16.5" - 17.5"	003097		
Tall 18.5" - 19.5"	003096	8	6
	003097		

Assemblies contain the cover, amount of straps and d-rings used for the tallest height listed in the assembly. Tension Straps (6a-6d) are based on seat width and used with the combination of Fold Down Push Handle and Tension Adjustable Back Upholstery. Attaching hardware is not included with the back upholstery assemblies.

Item Number	Part Number	Description	Quantity
1c, 2b, 3b, 4, 5a	505519	Back Upholstery Assembly Tension Adjustable with Standard Cover 14W, 12.5" - 15.5" Height, Standard Backposts	1
1e, 2b, 3b, 4, 5a	505520	Back Upholstery Assembly Tension Adjustable with Standard Cover 14W, 16.5" - 19.5" Height, Standard Backposts	1
1h, 2b, 3b, 4, 5b	505522	Back Upholstery Assembly Tension Adjustable with Standard Cover 15W, 12.5" - 15.5" Height, Standard Backposts	1
1j, 2b, 3b, 4, 5b	505523	Back Upholstery Assembly Tension Adjustable with Standard Cover 15W, 16.5" - 19.5" Height, Standard Backposts	1
1m, 2b, 3b, 4, 5c	505525	Back Upholstery Assembly Tension Adjustable with Standard Cover 16W, 12.5" - 15.5" Height, Standard Backposts	1

Item Number	Part Number	Description	Quantity
1o, 2b, 3b, 4, 5c	505526	Back Upholstery Assembly Tension Adjustable with Standard Cover 16W, 16.5" - 19.5" Height, Standard Backposts	1
1r, 2b, 3b, 4, 5d	505528	Back Upholstery Assembly Tension Adjustable with Standard Cover 17W, 12.5" - 15.5" Height, Standard Backposts	1
1t, 2b, 3b,4, 5d	505529	Back Upholstery Assembly Tension Adjustable with Standard Cover 17W, 16.5" - 19.5" Height, Standard Backposts	1
1w, 2b, 3b, 4, 5e	505531	Back Upholstery Assembly Tension Adjustable with Standard Cover 18W, 12.5" - 15.5" Height, Standard Backposts	1
1y, 2b, 3b, 4, 5e	505532	Back Upholstery Assembly Tension Adjustable with Standard Cover 18W, 16.5" - 19.5" Height, Standard Backposts	1
1ab, 2b, 3b, 4, 5f	505534	Back Upholstery Assembly Tension Adjustable with Standard Cover 19W, 12.5" - 15.5" Height, Standard Backposts	1
1ad, 2b, 3b, 4, 5f	505535	Back Upholstery Assembly Tension Adjustable with Standard Cover 19W, 16.5" - 19.5" Height, Standard Backposts	1
1ag, 2b, 3b, 4, 5g	505537	Back Upholstery Assembly Tension Adjustable with Standard Cover 20W, 12.5" - 15.5" Height, Standard Backposts	1
1ai, 2b, 3b, 4, 5g	505538	Back Upholstery Assembly Tension Adjustable with Standard Cover 20W, 16.5" - 19.5" Height, Standard Backposts	1
1al, 2b, 3b, 4, 5h	505540	Back Upholstery Assembly Tension Adjustable with Standard Cover 21W, 12.5" - 15.5" Height, Standard Backposts	1
1an, 2b, 3b, 4, 5h	505541	Back Upholstery Assembly Tension Adjustable with Standard Cover 21W, 16.5" - 19.5" Height, Standard Backposts	1
1ao, 2b, 3b, 4, 5i	505543	Back Upholstery Assembly Tension Adjustable with Standard Cover 22W, 12.5" - 15.5" Height, Standard Backposts	1
1aq, 2b, 3b, 4, 5i	505544	Back Upholstery Assembly Tension Adjustable with Standard Cover 22W, 16.5" - 19.5" Height, Standard Backposts	1
2b, 3b, 4, 5a, 8c	107898	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 14W, 12.5" - 15.5" Height, Standard Backposts	1
2b, 3b, 4, 5a, 8e	107899	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 14W, 16.5" - 19.5" Height, Standard Backposts	1
2b, 3b, 4, 5b, 8h	107901	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 15W, 12.5" - 15.5" Height, Standard Backposts	1
2b, 3b, 4, 5b, 8j	107902	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 15W, 16.5" - 19.5" Height, Standard Backposts	1
2b, 3b, 4, 5c, 8m	107904	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 16W, 12.5" - 15.5" Height, Standard Backposts	1
2b, 3b, 4, 5c, 8o	107905	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 16W, 16.5" - 19.5" Height, Standard Backposts	1

Item Number	Part Number	Description	Quantity
2b, 3b, 4, 5d, 8r	107907	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 17W, 12.5" - 15.5" Height, Standard Backposts	1
2b, 3b,4, 5d, 8t	107908	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 17W, 16.5" - 19.5" Height, Standard Backposts	1
2b, 3b, 4, 5e, 8w	107910	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 18W, 12.5" - 15.5" Height, Standard Backposts	1
2b, 3b, 4, 5e, 8y	107911	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 18W, 16.5" - 19.5" Height, Standard Backposts	1
2b, 3b, 4, 5f, 8ab	107913	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 19W, 12.5" - 15.5" Height, Standard Backposts	1
2b, 3b, 4, 5f, 8ad	107914	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 19W, 16.5" - 19.5" Height, Standard Backposts	1
2b, 3b, 4, 5g, 8ag	107916	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 20W, 12.5" - 15.5" Height, Standard Backposts	1
2b, 3b, 4, 5g, 8ai	107917	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 20W, 16.5" - 19.5" Height, Standard Backposts	1
2b, 3b, 4, 5h, 8al	107919	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 21W, 12.5" - 15.5" Height, Standard Backposts	1
2b, 3b, 4, 5h, 8an	107920	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 21W, 16.5" - 19.5" Height, Standard Backposts	1
2b, 3b, 4, 5i, 8ao	107921	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 22W, 12.5" - 15.5" Height, Standard Backposts	1
2b, 3b, 4, 5i, 8aq	107922	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 22W, 16.5" - 19.5" Height, Standard Backposts	1
1a	002497	Backrest Cover 13" Rigid / 14" Cat x 9-12"	1
1b	002498	Backrest Cover 13" Rigid / 14" Cat x 11-14"	1
1c	002499	Backrest Cover 13" Rigid / 14" Cat x 13-16"	1
1d	002500	Backrest Cover 13" Rigid - 14" CAT/Liberty x 15-18"	1
1e	003043	Backrest Cover 13" Rigid - 14" CAT/Liberty x 17-20"	1
1f	002501	Backrest Cover 14" Rigid / 15" Cat x 9-12"	1
1g	002502	Backrest Cover 14" Rigid / 15" Cat x 11-14"	1
1h	002503	Backrest Cover 14" Rigid / 15" Cat x 13-16"	1
1i	002504	Backrest Cover 14" Rigid - 15" CAT/Liberty x 15-18"	1
1j	003044	Backrest Cover 14" Rigid - 15" CAT/Liberty x 17-20"	1

Item Number	Part Number	Description	Quantity
1k	002505	Backrest Cover 15" Rigid / 16" Cat x 9-12"	1
1l	002506	Backrest Cover 15" Rigid / 16" Cat x 11-14"	1
1m	002507	Backrest Cover 15" Rigid / 16" Cat x 13-16"	1
1n	002508	Backrest Cover 15" Rigid - 16" CAT/Liberty x 15-18"	1
1o	003045	Backrest Cover 15" Rigid - 16" CAT/Liberty x 17-20"	1
1p	002509	Backrest Cover 16" Rigid / 17" Cat x 9-12"	1
1q	002510	Backrest Cover 16" Rigid / 17" Cat x 11-14"	1
1r	002511	Backrest Cover 16" Rigid / 17" Cat x 13-16"	1
1s	002512	Backrest Cover 16" Rigid - 17" CAT/Liberty x 15-18"	1
1t	003046	Backrest Cover 16" Rigid - 17" CAT/Liberty x 17-20"	1
1u	003048	Backrest Cover 17" Rigid / 18" Cat x 9-12"	1
1v	003049	Backrest Cover 17" Rigid / 18" Cat x 11-14"	1
1w	003050	Backrest Cover 17" Rigid / 18" Cat x 13-16"	1
1x	003051	Backrest Cover 17" Rigid - 18" CAT/Liberty x 15-18"	1
1y	003052	Backrest Cover 17" Rigid - 18" CAT/Liberty x 17-20"	1
1z	003054	Backrest Cover 18" Rigid / 19" Cat x 9-12"	1
1aa	003055	Backrest Cover 18" Rigid / 19" Cat x 11-14"	1
1ab	003056	Backrest Cover 18" Rigid / 19" Cat x 13-16"	1
1ac	003057	Backrest Cover 18" Rigid - 19" CAT/Liberty x 15-18"	1
1ad	003058	Backrest Cover 18" Rigid - 19" CAT/Liberty x 17-20"	1
1ae	003060	Backrest Cover 19" Rigid / 20" Cat x 9-12"	1
1af	003061	Backrest Cover 19" Rigid / 20" Cat x 11-14"	1
1ag	003062	Backrest Cover 19" Rigid / 20" Cat x 13-16"	1
1ah	003063	Backrest Cover 19" Rigid - 20" CAT/Liberty x 15-18"	1
1ai	003064	Backrest Cover 19" Rigid - 20" CAT/Liberty x 17-20"	1
1aj	003066	Backrest Cover 20" Rigid / 21" Cat x 9-12"	1
1ak	003067	Backrest Cover 20" Rigid / 21" Cat x 11-14"	1

Item Number	Part Number	Description	Quantity
1al	003068	Backrest Cover 20" Rigid / 21" Cat x 13-16"	1
1am	003069	Backrest Cover 20" Rigid - 21" CAT/Liberty x 15-18"	1
1an	003070	Backrest Cover 20" Rigid - 21" CAT/Liberty x 17-20"	1
1ao	003072	Backrest Cover 21" Rigid / 22" Cat x 13-16"	1
1ap	003073	Backrest Cover 21" Rigid - 22" CAT/Liberty x 15-18"	1
1aq	003074	Backrest Cover 21" Rigid - 22" CAT/Liberty x 17-20"	1
2a	003095	Tension Adjustable Upholstery Wing 5.5" Height Left	1
2b	003097	Tension Adjustable Upholstery Wing 9.5" Height Left	1
3a	003094	Tension Adjustable Upholstery Wing 5.5" Height Right	1
3b	003096	Tension Adjustable Upholstery Wing 9.5" Height Right	1
4	100973	Back Upholstery D-Ring	2
5a	003487	Back Upholstery Strap 15.3" <i>Used on 14" chair widths</i>	3
5b	002532	Back Upholstery Strap 17.3" <i>Used on 15" chair widths</i>	3
5c	002533	Back Upholstery Strap 19.3" <i>Used on 16" chair widths</i>	3
5d	002534	Back Upholstery Strap 21.3" <i>Used on 17" chair widths</i>	3
5e	002535	Back Upholstery Strap 23.3" <i>Used on 18" chair widths</i>	3
5f	002536	Back Upholstery Strap 25.3" <i>Used on 19" chair widths</i>	3
5g	002537	Back Upholstery Strap 27.3" <i>Used on 20" chair widths</i>	3
5h	002538	Back Upholstery Strap 29.3" <i>Used on 21" chair widths</i>	3
5i	002539	Back Upholstery Strap 31.3" <i>Used on 22" chair widths</i>	3
6a	001750	Tension Strap Folding Handle 14-15 Wide	1
6b	001751	Tension Strap Folding Handle 16-17 Wide	1

Item Number	Part Number	Description	Quantity
6c	001752	Tension Strap Folding Handle 18-19 Wide	1
6d	001753	Tension Strap Folding Handle 20-22 Wide	1
7	100734	M5 x 16 PHMS ZC	2
8a	003896	Air Mesh Backrest Cover 14C 13R Width x 9-12" Height	1
8b	003897	Air Mesh Backrest Cover 14C 13R Width x 11-14" Height	1
8c	003898	Air Mesh Backrest Cover 14C 13R Width x 13-16" Height	1
8d	003899	Air Mesh Backrest Cover 14C 13R Width x 15-18" Height	1
8e	003913	Air Mesh Backrest Cover 14C 13R Width x 17-20" Height	1
8f	003900	Air Mesh Backrest Cover 15C 14R Width x 9-12" Height	1
8g	003901	Air Mesh Backrest Cover 15C 14R Width x 11-14" Height	1
8h	003902	Air Mesh Backrest Cover 15C 14R Width x 13-16" Height	1
8i	003903	Air Mesh Backrest Cover 15C 14R Width x 15-18" Height	1
8j	003914	Air Mesh Backrest Cover 15C 14R Width x 17-20" Height	1
8k	003904	Air Mesh Backrest Cover 16C 15R Width x 9-12" Height	1
8l	003905	Air Mesh Backrest Cover 16C 15R Width x 11-14" Height	1
8m	003906	Air Mesh Backrest Cover 16C 15R Width x 13-16" Height	1
8n	003907	Air Mesh Backrest Cover 16C 15R Width x 15-18" Height	1
8o	003915	Air Mesh Backrest Cover 16C 15R Width x 17-20" Height	1
8p	003909	Air Mesh Backrest Cover 17C 16R Width x 9-12" Height	1
8q	003910	Air Mesh Backrest Cover 17C 16R Width x 11-14" Height	1
8r	003911	Air Mesh Backrest Cover 17C 16R Width x 13-16" Height	1
8s	003912	Air Mesh Backrest Cover 17C 16R Width x 15-18" Height	1
8t	003916	Air Mesh Backrest Cover 17C 16R Width x 17-20" Height	1
8u	003918	Air Mesh Backrest Cover 18C 17R Width x 9-12" Height	1
8v	003919	Air Mesh Backrest Cover 18C 17R Width x 11-14" Height	1
8w	003920	Air Mesh Backrest Cover 18C 17R Width x 13-16" Height	1
8x	003921	Air Mesh Backrest Cover 18C 17R Width x 15-18" Height	1

Item Number	Part Number	Description	Quantity
8y	003922	Air Mesh Backrest Cover 18C 17R Width x 17-20" Height	1
8z	003924	Air Mesh Backrest Cover 19C 18R Width x 9-12" Height	1
8aa	003925	Air Mesh Backrest Cover 19C 18R Width x 11-14" Height	1
8ab	003926	Air Mesh Backrest Cover 19C 18R Width x 13-16" Height	1
8ac	003927	Air Mesh Backrest Cover 19C 18R Width x 15-18" Height	1
8ad	003928	Air Mesh Backrest Cover 19C 18R Width x 17-20" Height	1
8ae	003930	Air Mesh Backrest Cover 20C 19R Wide x 9-12" Height	1
8af	003931	Air Mesh Backrest Cover 20C 19R Width x 11-14" Height	1
8ag	003932	Air Mesh Backrest Cover 20C 19R Width x 13-16" Height	1
8ah	003933	Air Mesh Backrest Cover 20C 19R Width x 15-18" Height	1
8ai	003934	Air Mesh Backrest Cover 20C 19R Width x 17-20" Height	1
8aj	003936	Air Mesh Backrest Cover 21C 20R Width x 9-12" Height	1
8ak	003937	Air Mesh Backrest Cover 21C 20R Width x 11-14" Height	1
8al	003938	Air Mesh Backrest Cover 21C 20R Width x 13-16" Height	1
8am	003939	Air Mesh Backrest Cover 21C 20R Width x 15-18" Height	1
8an	003940	Air Mesh Backrest Cover 21C 20R Width x 17-20" Height	1
8ao	003942	Air Mesh Backrest Cover 22C 21R Width x 13-16" Height	1
8ap	003943	Air Mesh Backrest Cover 22C 21R Width x 15-18" Height	1
8aq	003944	Air Mesh Backrest Cover 22C 21R Width x 17-20" Height	1