
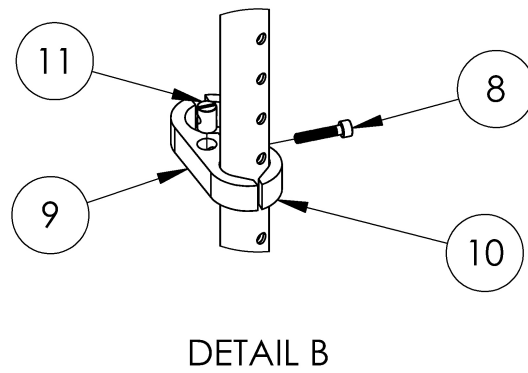
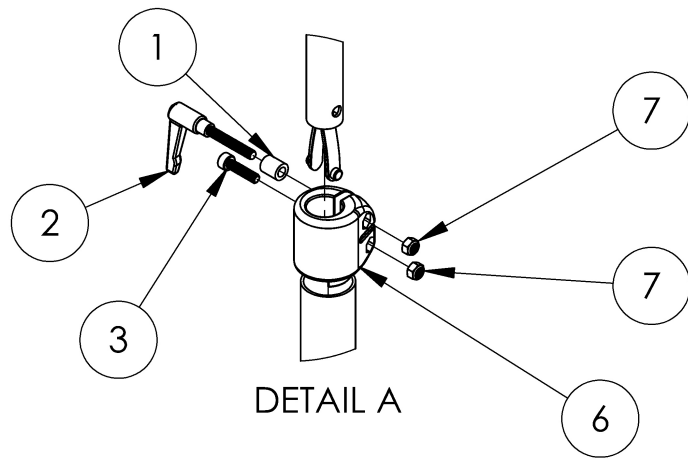


# Tilt IV Pole

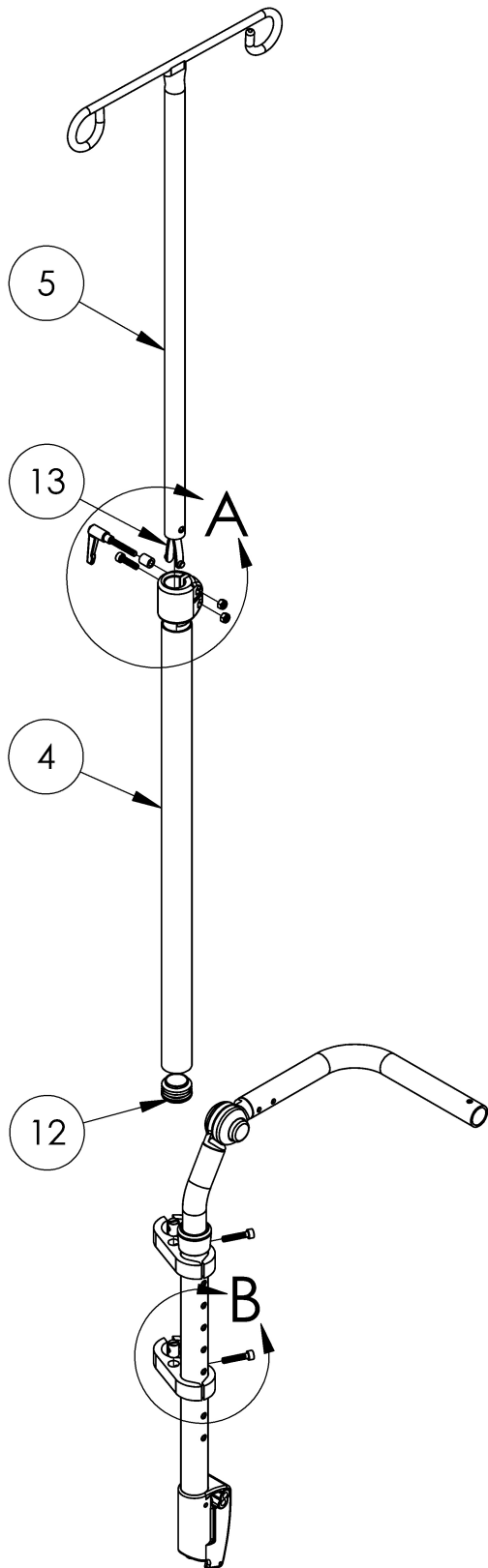

Item Number	Part Number	Description	Quantity
1-13	502409	IV Pole Assembly Focus/Flip	1
1, 2, 3, 6, 7	502401	IV Pole Top Clamp Assembly	1
8, 9, 10, 11	502408	IV Pole Bottom Clamp Assembly Focus	1
1	102766	Spacer 3/8R. X 7/16 x .218 ID.	1
2	102767	Handle wM5 x 32L Threaded Stud	1
3	101843	M5 x 20 SHCS Blk Zn	1
4	002581	IV Pole Outer Tube 28" Height	1
5	002585	IV Pole Upper	1
6	002395	Footrest Clamp Left LW	1
7	100657	M5 Nylock Nut Blk Zn	2
8	100780	M6 x 25 SHCS ZC	2
9	002685	IV Pole Clamp 1-1/8" 1" Thru 1-1/4" w/ Cross Dowel Hole	2
10	002686	IV Pole Clamp 1-1/8" 1" Thru 1-1/4"	2
11	102768	M6 Barrel Nut	2
12	100532	Tube End Plug 1 1/8"	1
13	102769	Snap Button Plastic	1