


Liberty Seat Frame - Growth

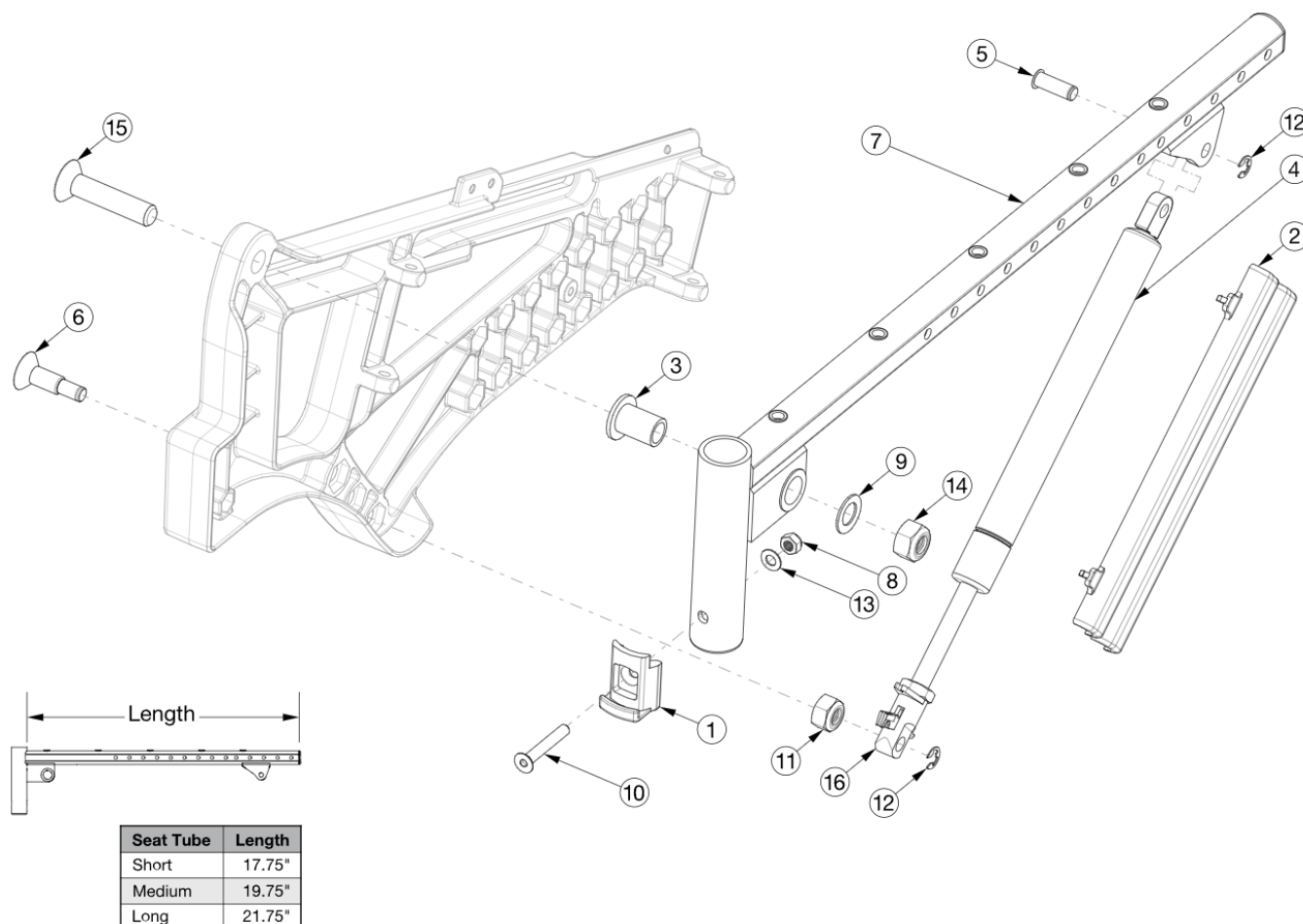

Can you achieve your new depth within the existing depth range? Short = 14" - 16" Medium = 16" - 18" Long = 18" - 20" If no, please choose the appropriate kit below.

Item Number	Part Number	Description	Quantity
2, 4b, 5, 6, 11, 12, 16	506149	Gas Spring Kit, 300N, Liberty	1
2, 4a, 5, 6, 11, 12, 16	506150	Gas Spring Kit, 450N, Liberty	1
3, 7a, 9, 14, 15	506146	Seat Tube Assembly, Short, Liberty	1
3, 7b, 9, 14, 15	506147	Seat Tube Assembly, Medium, Liberty	1
3, 7c, 9, 14, 15	506148	Seat Tube Assembly, Long, Liberty <i>Updated to longer clevis on seat tube 10/21/2020. Not compatible with Transit Wheels.</i>	1
1	000336	Latch Block	1
2, 4a, 16	506508	Gas Spring, 95mm Stroke, 450N with Cover, Liberty <i>Used on Depths 18" and Greater (Default Setting)</i>	1

Item Number	Part Number	Description	Quantity
2, 4b, 16	506509	Gas Spring, 95mm Stroke, 300N with Cover, Liberty <i>Used on Depths Less than 18" (Default Setting)</i>	1
3	003278	Pivot Bushing	1
5	003282	Clevis Pin, 8mm x .830Lg	1
6	003283	Gas Spring Mount Screw	1
8	100658	M6 Nylock Nut Blk Zn	1
9	100682	Washer 1/2" .518 x .875 x .047 F/W Black Zinc	1
10	101819	M6 x 40 FHCS Blk Zn	1
11	101840	M10 Nylock Nut Blk Zn	1
12	102335	E-Ring 5/16" SS	2
13	102803	Washer Flat .250 X .500 X .027 Hx Blz	1
14	103021	M12 X 1.75mm Nylock Nut Blk Zn	1
15	103031	M12 x 55 FHCS, 10.9, Blk Zn	1
16	111877	Gas Strut Lug, Liberty <i>Includes replacement instructions</i>	1