


# Ethos Seat Upholstery

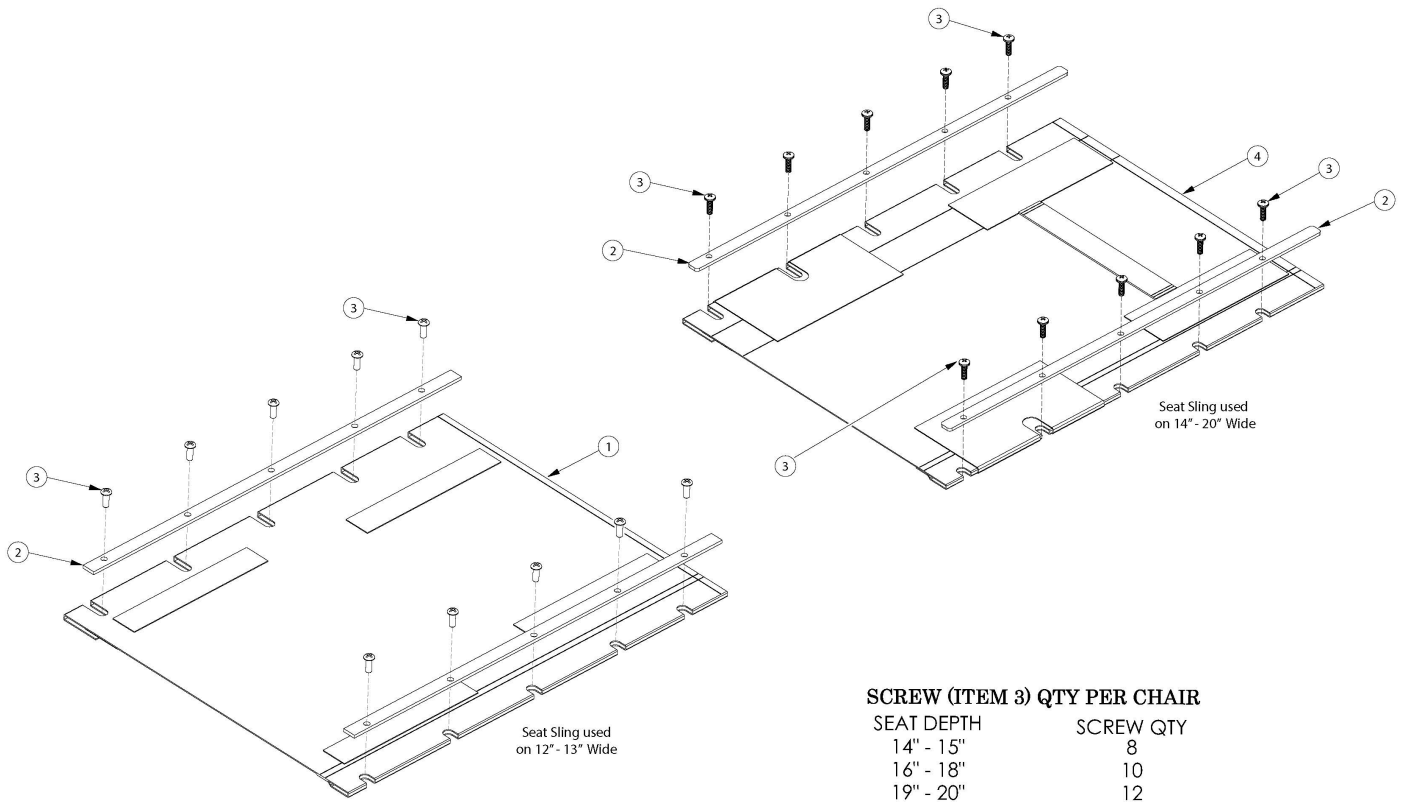

Seat upholstery width measures approximately 1" less than the chair size. Seat upholstery depth measures true to the chair size.

Item Number	Part Number	Description	Quantity
1a	101516	Seat Upholstery Rigid 1214	1
1b	101517	Seat Upholstery Rigid 1215	1
1c	101518	Seat Upholstery Rigid 1216	1
1d	101519	Seat Upholstery Rigid 1217	1
1e	101520	Seat Upholstery Rigid 1218	1
1f	101569	Seat Upholstery Rigid 1219	1
1g	101523	Seat Upholstery Rigid 1314	1
1h	101524	Seat Upholstery Rigid 1315	1
1i	101525	Seat Upholstery Rigid 1316	1
1j	101526	Seat Upholstery Rigid 1317	1

Item Number	Part Number	Description	Quantity
1k	101527	Seat Upholstery Rigid 1318	1
1l	101571	Seat Upholstery Rigid 1319	1
1m	101572	Seat Upholstery Rigid 1320	1
2a	000241	Seat Rail 14" Deep Rigid/Liberty	2
2b	000242	Seat Rail 15" Deep RIGID	2
2c	000243	Seat Rail 16" Deep Rigid/Liberty	2
2d	000244	Seat Rail 17" Deep RIGID	2
2e	000245	Seat Rail 18" Deep Rigid/Liberty	2
2f	000246	Seat Rail 19" Deep RIGID	2
2g	000247	Seat Rail 20" Deep Rigid/Liberty	2
3	100734	M5 x 16 PHMS ZC	12
4a	101000	Seat Upholstery Rigid/Liberty 1414	1
4b	101005	Seat Upholstery Rigid 1415	1
4c	101012	Seat Upholstery Rigid/Liberty 1416	1
4d	101019	Seat Upholstery Rigid 1417	1
4e	101026	Seat Upholstery Rigid/Liberty 1418	1
4f	101033	Seat Upholstery Rigid 1419	1
4g	101074	Seat Upholstery Rigid/Liberty 1420	1
4h	101001	Seat Upholstery Rigid/Liberty 1514	1
4i	101006	Seat Upholstery Rigid 1515	1
4j	101013	Seat Upholstery Rigid/Liberty 1516	1
4k	101020	Seat Upholstery Rigid 1517	1
4l	101027	Seat Upholstery Rigid/Liberty 1518	1
4m	101034	Seat Upholstery Rigid 1519	1
4n	101040	Seat Upholstery Rigid/Liberty 1520	1
4o	101002	Seat Upholstery Rigid/Liberty 1614	1
4p	101007	Seat Upholstery Rigid 1615	1
4q	101014	Seat Upholstery Rigid/Liberty 1616	1

Item Number	Part Number	Description	Quantity
4r	101021	Seat Upholstery Rigid 1617	1
4s	101028	Seat Upholstery Rigid/Liberty 1618	1
4t	101035	Seat Upholstery Rigid 1619	1
4u	101041	Seat Upholstery Rigid/Liberty 1620	1
4v	101003	Seat Upholstery Rigid/Liberty 1714	1
4w	101008	Seat Upholstery Rigid 1715	1
4x	101015	Seat Upholstery Rigid/Liberty 1716	1
4y	101022	Seat Upholstery Rigid 1717	1
4z	101029	Seat Upholstery Rigid/Liberty 1718	1
4aa	101036	Seat Upholstery Rigid 1719	1
4ab	101042	Seat Upholstery Rigid/Liberty 1720	1
4ac	101004	Seat Upholstery Rigid/Liberty 1814	1
4ad	101009	Seat Upholstery Rigid 1815	1
4ae	101016	Seat Upholstery Rigid/Liberty 1816	1
4af	101023	Seat Upholstery Rigid 1817	1
4ag	101030	Seat Upholstery Rigid/Liberty 1818	1
4ah	101037	Seat Upholstery Rigid 1819	1
4ai	101043	Seat Upholstery Rigid/Liberty 1820	1
4aj	101010	Seat Upholstery Rigid 1915	1
4ak	101017	Seat Upholstery Rigid/Liberty 1916	1
4al	101024	Seat Upholstery Rigid 1917	1
4am	101031	Seat Upholstery Rigid/Liberty 1918	1
4an	101038	Seat Upholstery Rigid 1919	1
4ao	101044	Seat Upholstery Rigid/Liberty 1920	1
4ap	101011	Seat Upholstery Rigid 2015	1
4aq	101018	Seat Upholstery Rigid/Liberty 2016	1
4ar	101025	Seat Upholstery Rigid 2017	1
4as	101032	Seat Upholstery Rigid/Liberty 2018	1

<b>Item Number</b>	<b>Part Number</b>	<b>Description</b>	<b>Quantity</b>
4at	101039	Seat Upholstery Rigid 2019	1
4au	101045	Seat Upholstery Rigid/Liberty 2020	1